Serie **SILLAS** S-127 / S-127 B

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

Sillas mecánicas S-127 / S-127 B

Instruction manual

Mechanical chairs S-127 / S-127 B

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Chaises mécaniques S-127 / S-127 B

ESPAÑOL

ENGLISH

FRANÇAIS

S-127 / S-127 B

Silla de rescate

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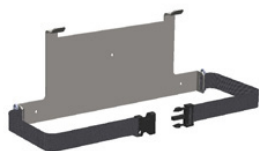
S-127 / S-127 B

Rescue chair

Read these maintenance instructions before using the product and keep them for future reference.

CONTENTS

1. INTRODUCTION	4
1.1 Foreword	4
1.2 Intended use of the product	4
1.3 Liability and warranty	5
1.4 Specifications	5
1.5 Warning	6
1.6 Summary of safety precautions	6
2. INSTRUCTION MANUAL	8
2.1 Technical features	8
2.2 Folding and unfolding the wheelchair	10
2.3 Handles	11
2.4 Straps	11
3. CARE AND MAINTENANCE	12
3.1 Contact information	12
3.2 Environmental conditions	12
3.3 Cleaning	12
3.4 Maintenance	13
3.4.1 Precautionary maintenance	13
3.4.2 Service maintenance	14
3.5 Transport and storage	14
3.6 Disposal	14
4. SPARE PARTS	15
5. WHEELCHAIR ACCESSORIES	16

1. INTRODUCTION

1.1 Foreword

The **S-127** and **S-127 B** wheelchairs have been specifically designed for patient rescue and transport purposes. The product complies with EU Directive 2017/745.

These wheelchairs have been tested in accordance with standard UNE-EN 1789. All information on treatment, disinfection and maintenance is based on our experience and current understanding. We reserve the right to make technical variations to the wheelchairs in order to improve the product.

1.2. Intended use of the product

Read the instruction manual before use.

Maximum load 150 kg.

Class 1 medical product.

Safety warning and instructions.

- Do not dispose of the product as general waste. Send to a recycling centre.
- The packaging should be treated as reusable waste.
- Metals should be intended as old material.
- Plastic products should be treated as recycling material.
- Waste management should follow the rules applying in individual countries.
- Please contact the local municipality for information about recycling and waste.

The **S-127** and **S-127 B** wheelchairs are designed for the transport and support of patients in hospital and pre-hospital settings. The product is designed to place patients in a seated position.

This wheelchair has a maximum capacity of 150 kg and is intended for use by trained professional operators, including emergency medical services and care home medical staff, and first aid medical teams. Rescue wheelchairs are designed for patient transport and not for prolonged stays, or for use in devices that modify atmospheric pressure, such as hyperbaric chambers.

1.3. Liability and warranty

The wheelchair should be checked when handed over to the rescue organisation. All of the features must be explained in detail. The rescue organisation is responsible for training all employees on how to use the equipment correctly.

The product comes with a warranty period of 24 months from the date on the purchase invoice (see the Warranty Terms and Conditions and the Warranty Certificate enclosed with the product).

The Warranty shall not cover breakdown due to incorrect installation, mishandling or misuse of the product. Repairs must be carried out by a service technician authorised by Kartsana or its authorised representative.

The manufacturer shall not be liable for any damage to the stretcher caused by the use of non-original Kartsana products. Changes or modifications made to the unit that have not been expressly approved by Kartsana may result in errors in the equipment's operation.

1.4 Specifications

Maximum workload (unassisted)	150Kg
Main regulations	UNE-EN-1865-4 EN-1789+A1
Differences between S-127 and S-127 B	Both models have the same structure and specifications, though the material of the seat, backrest and straps differs. While the S-127 has a thermoformed backrest and seat, the B version has canvas ones.
Total height	1039 mm
Total width	489 mm
Total length	483 mm
Wheel diameter	125 mm
Recommended operators for loading/unloading an occupied wheelchair	1 operator or 2 operators in the event that the wheelchair is used by a very large person

Kartsana reserves the right to change the specifications without prior notice

1.5. Warning

Throughout this manual, there are 3 types of symbols. As a **warning symbol**, we have ISO 7010-W001. The **prohibition** is indicated by ISO 7010-P001 and the **obligation** by ISO 7010-M001.

Using these symbols, we provide information on important safety measures for the correct use of the stretcher and to avoid possible accidents.

Warnings alert the reader to situations that, if not avoided, may lead to death or serious injury. Furthermore, prohibitions indicate actions that must not be carried out under any circumstance, and obligations indicate measures and/or actions that must be taken to ensure the proper use of the product and to maintain safe functionality.

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WARNINGS

- Improper use of the chair may cause injury to the patient or the operator. Use the chair only as directed in this manual.

- Do not modify the chair or any of its components. Modifying the device may cause unpredictable operation, which could result in injury to the patient or the operator. Device modifications void the warranty.

- Do not allow untrained assistants to help operate the chair. Technicians or assistants without proper training may cause injury to the patient or themselves.

- Always use all restraint belts to secure the patient in the chair. If not properly secured, the patient could fall from the chair and be injured.

- When the chair is occupied, two operators are required to handle it and secure the patient.

- Operators must be trained to lift the total weight of the patient, the chair, and any additional chair components.

- When cleaning, use appropriate personal protective equipment (such as safety goggles or respirators) to avoid the risk of inhaling infectious organisms.

- Some cleaning products are corrosive and may damage the chair if used incorrectly. Measures should be taken to ensure chairs are cleaned with clean water and thoroughly

dried afterward. If units are not properly rinsed and dried, corrosive residues may remain on the surface, potentially causing premature corrosion of critical components.

- Inadequate cleaning increases the risk of exposure to bloodborne pathogens and may cause injury to the patient or the operator.

CAUTIONS

- Any changes or modifications made to the unit that have not been expressly approved by Kartsana may void the user's authority to operate the system.
- Before operating the chair, remove all obstacles that could interfere with its use and cause injury to the operator or the patient.
- Do not clean the unit with steam or ultrasonic cleaners.
- The maximum water temperature must not exceed 80°C (176°F).
- Failure to follow these instructions may result in the voiding of some or all warranties.
- A preventive maintenance program must be established for all Kartsana equipment. Depending on the frequency of use, preventive maintenance may need to be carried out more often.
- For additional maintenance information, please contact our technical department.
- Inadequate maintenance may cause injury or damage to the product. Perform maintenance on the stretcher as described in this manual. Follow the maintenance procedures and use only parts approved by Kartsana. Using unapproved parts or procedures may result in unpredictable operation, cause injury, and void the warranty.

NOTES

- This manual should be considered part of the chair and, as such, must be kept with the product, even if it is sold.
- Kartsana is constantly improving the design and quality of its products. Therefore, although this manual contains the most up-to-date information available at the time of printing, there may be minor discrepancies between the chair and this manual. If you have any questions, please contact Kartsana Customer Service or Technical Support.

2. INSTRUCTION MANUAL

2.1. Technical features of the S-127 / S-127 B rescue wheelchair

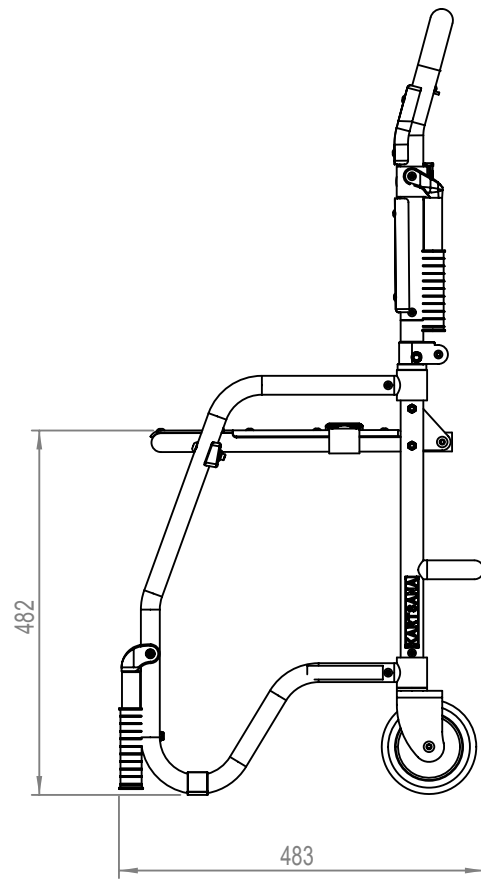
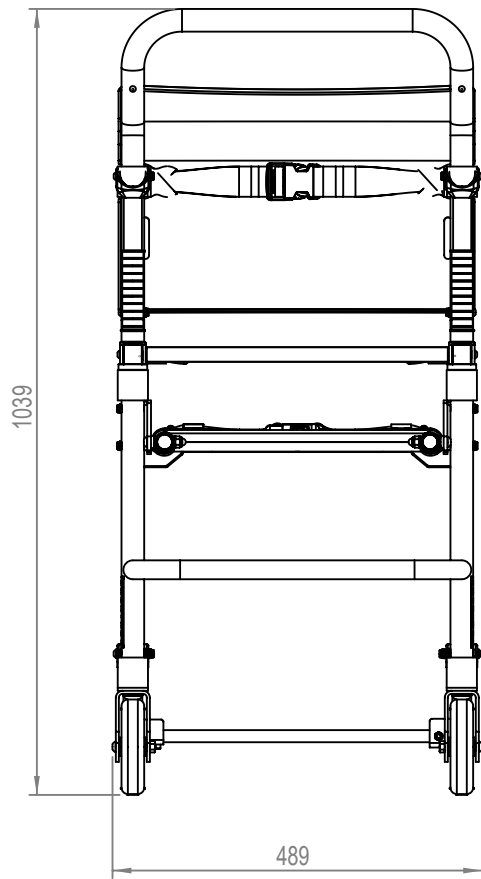
Do not damage the areas where the wheelchair mechanisms are located in order to prevent them from malfunctioning.

Wheelchair weight: 7,2 Kg


Max laod: 150 Kg

Dimensions and measurements in mm


Folded wheelchair:

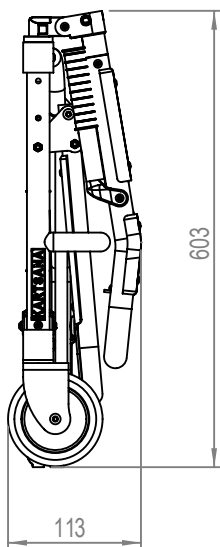2.2. Folding and unfolding the wheelchair

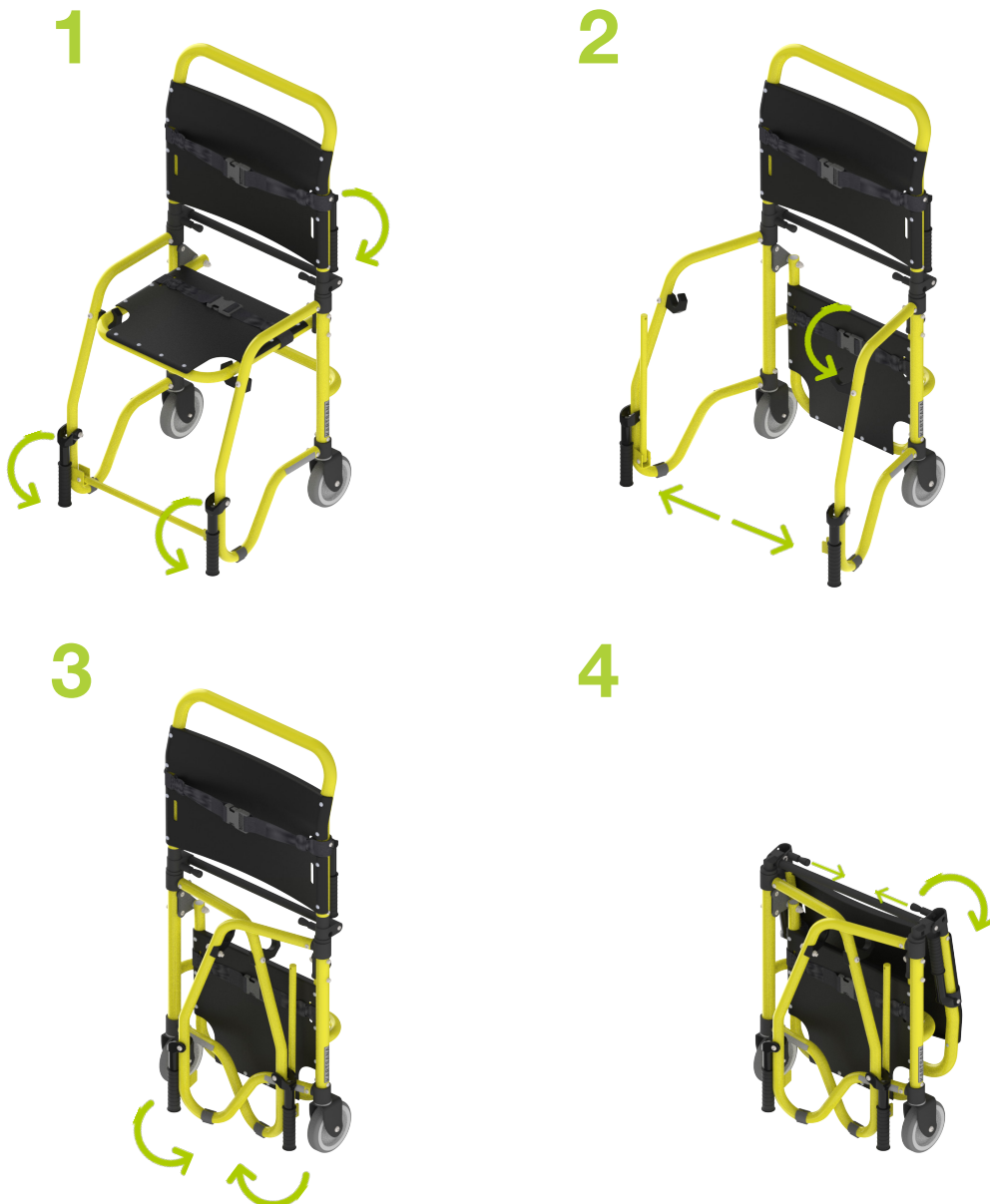To fold the chair, place it upright with the handles in the rest position, lower the seat as far as it will go, fold the crossed legs and lastly fold the backrest back by pulling the knobs on both sides of the backrest (see sequence below).

To unfold it, follow the same steps in reverse order.

To minimise the risk of injury to the patient and attendants in the event of an accident, it is advisable to avoid sharp edges and protruding surfaces inside the ambulance and paying special attention to the areas near the wheelchair. To avoid potential injury to limbs and other body parts, do not place them along the path of the moving parts of the wheelchair.

2.3. Handles

Once the patient is seated in the wheelchair, place the handles in the right position for transport. To use them, simply fold them up as far as they will go.

When descending, for example when going down a flight of stairs, all handles must remain unfolded.

Before folding the wheelchair, remember to place them back in their initial position.

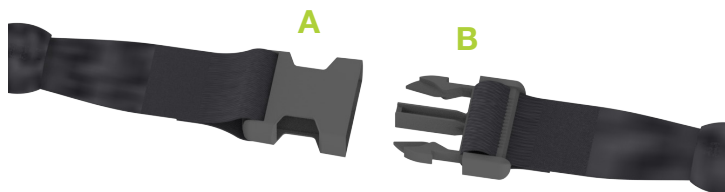
2.4. Straps

Before starting to transport the patient, make sure that the straps are positioned correctly and securely adjusted to the patient.

To secure the strap:

- ◇ Place the female buckle (A) and the male buckle (B) in the position shown in the figure below.
- ◇ Insert the male buckle (B) into the groove of the female buckle (A) until it is locked.
- ◇ Centre the assembly and adjust the strap tension by adjusting its end section.

3. CARE AND MAINTENANCE OF THE PRODUCT

3.1. Contact information

If you have any questions regarding the use, maintenance or installation of a KARTSANA product, please contact our Technical Service Area (SAT). Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient are established.

Telephone: +34 93 715 86 72
Email: sat@kartsana.com
Address: Narcís Monturiol, 34
08192 Sant Quirze del Vallès,
Barcelona (Spain)

3.2. Environmental conditions

Both for use and storage:

Recommended temperature: from -10°C to +60°C

Relative humidity: 5 – 95%

3.3. Cleaning

To clean the stretcher, use a cloth, rag or similar to wipe the surface to be cleaned with products that do not damage the surface of the materials. If the surface to be cleaned is already lubricated, re-grease it after cleaning has been completed.

Please use cleaning agents according to the manufacturer's instructions.

The manufacturer shall not be liable for any damage or anomaly caused by a cleaning product that may damage the surface of the stretcher materials.

For reasons of hygiene and preservation of the product components, the manufacturer recommends cleaning after each use.

3.4. Maintenance

3.4.1 Precautionary maintenance

Correct and regular maintenance guarantees the durability of the product.

Creating a maintenance plan, including regular reviews and establishing a responsible person to carry them out, is recommended.

The person in charge must adhere to the following **requirements**:

- Possess technical knowledge related to the product and the maintenance procedures described in these instructions.
- To have the support of qualified, trained and knowledgeable staff who are familiar with the operation of the product.
- Use original or manufacturer-approved components, spare parts and accessories.
- In accordance with the instructions of the European Council Directive 93/42/EEC which establishes the obligation of the purchaser to provide on request the above mentioned aftersales care record for the purpose of product traceability, records of all product maintenance operations shall be kept.

Check for proper operation **before use**. Prior to service, the following should be checked:

- Functionality.
- Fastenings and hardware.
- Parts that move, such as wheels, belts and mattresses.
- Sensitivity of springs.
- Check the condition of the canvases (S-127 B)

If the device does not appear to be suitable for correct and safe use, **it should be taken out of service** until the device is repaired or restored to full working order.

Do not modify the structure of the device for any reason, as this may cause serious harm to patients and/or operators.

Lubrication intervals of moving parts:

- 1 – 30 services per month: every 3 months.
- 30 – 50 services per month: every 2 months.
- More than 150 services per month: every month.

All maintenance other than lubrication, tightening of nuts and bolts and ordinary cleaning must be carried out by KARTSANA or an authorised service centre.

3.4.2. Service maintenance

The person entrusted with the product review shall ensure the following basic requirements:

- ◇ Adequate knowledge of the product, its technical/constructional characteristics, control points and final tests, packaging, preservation and handling.
- ◇ Knowledge of all product functions and any possible risks, malfunctions or failures of the product.
- ◇ Possession of all necessary tools to carry out any technical service or minor repairs.
- ◇ Possession (or ability to procure) original manufacturer's or manufacturer's authorised spare parts.
- ◇ Use or support of specialised technical personnel trained by the manufacturer to service the product in question.
- ◇ Keeping a record of all maintenance operations carried out on the product, in accordance with the instructions of the European Council Directive 93/42/EEC which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability.

3.5. Transport and storage

Before transporting the device, ensure that it is properly packaged and secured to prevent damage during transportation.

Keep the original packaging for possible further transport. Any damage caused to the device during transport is not covered by the warranty.

Repairs and replacements of damaged parts are the responsibility of the customer. The device should be stored in a dry place.

When storing, do not place any heavy objects on top of the product. It must not be used as a support for any kind of object.

3.6. Disposal

When the product is no longer in a usable condition, do not throw it away but develop a take-back strategy for recycling by an authorised dealer. The packaging shall be managed as reusable waste. Metals are fully recyclable. Plastic products should be treated as recycling material. Waste management should be as established in each country. If you have any questions, please ask your municipal administration.

4. SPARE PARTS

To consult the spare parts for this product, you can visit the **following link:**

https://www.kartsana.com/global/Spare_parts_S-127.pdf

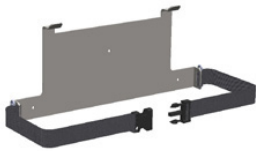
If you have any questions, please contact our technical service.

5. WHEELCHAIR ACCESSORIES

C-013 Stainless steel support for S127 / S129 chairs

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