
Serie **BRAVO**

R-1001 / R-1001 H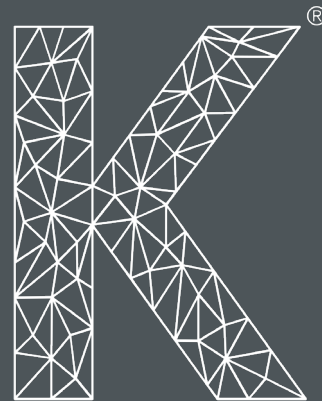

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

Rail eléctrico BRAVO

Instruction manual

Electric rail BRAVO

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ESPAÑOL

ENGLISH

FRANÇAIS

KARTSANA

Life-moving innovation


C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
TEL. 93 714-49-24
info@kartsana.com
www.kartsana.com

Serie Brava

RAIL BRAVO 1

Modelo: R-1001 y R-1001H

MANUAL DE INSTRUCCIONES

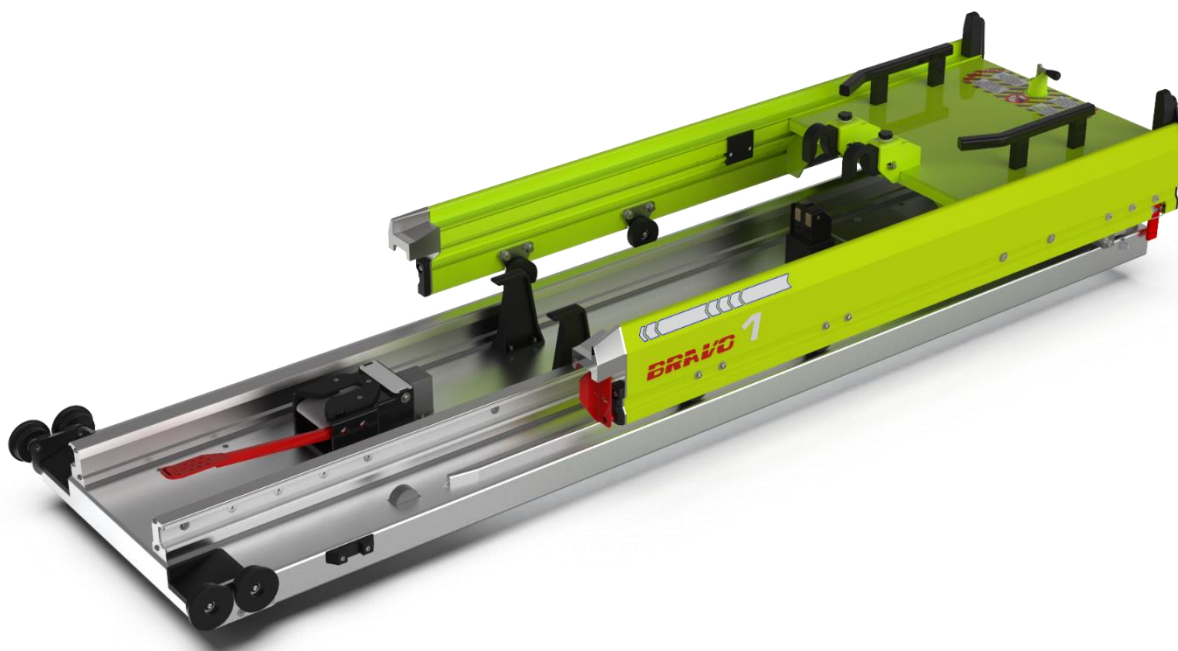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R-1001 and R-1001H Model

INSTRUCTIONS MANUAL

Read these maintenance instructions
before using the product and keep them
for future reference.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

CONTENTS

1.INTRODUCTION	4
1.1. FOREWORD	4
1.2. RESPONSIBILITY AND WARRANTY	4
1.3. SPECIFICATIONS	4
1.4. ATTENTION	5
1.5. SUMMARY OF SAFETY PRECAUTIONS	5
2. MANUAL INSTRUCTIONS	7
2.1. TECHNICAL CHARACTERISTICS	8
2.2. RAIL OPERATION AND USAGE (R-1001 Y R-100H)	8
2.2.1. RED CHARGING TRIGGER	8
2.2.2. UNLOADING LEVER	10
2.3. OPERATION AND HANDLING OF THE R-1001H RAIL ATTACHED TO THE HOVERBOARD	11
3.ASSEMBLY OF THE RAIL IN THE AMBULANCE	12
3.1. SETTING THE RAIL IN THE AMBULANCE	12
3.2. ELECTRICAL CONNECTION	12
3.3. FIXING THE RAIL AND THE HOVERBOARD TO THE AMBULANCE	14
4. INTERMEDIATE POSITION (OPTIONAL)	15
5. DISINFECTION	16
6. MAINTENANCE	17
7. SPARE PARTS	18

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

1. INTRODUCTION

1.1. Foreword

The BRAVO 1 (R-1001) rail has been solely designed for perfectly holding the KARTSANA “BRAVA” stretcher model.

The Bravo rail has been tested in conformity with the regulation UNE-EN 1789.

All the information regarding treatment, disinfection and maintenance is indicated, taking into consideration our current experience and know-how.

We reserve the right to change the specifications of the BRAVO rail to improve the products.

1.2. Responsibility and warranty

The stretcher must be checked at the time of its delivery to the rescue organization. All its functions must be explained in great detail. The assistance organization must take charge of teaching all its employees how to use it correctly.

The warranty is valid for two years following the date indicated on the purchase invoice.

The warranty does not cover defects resulting from incorrect installation, misuse, or improper use of the stretcher. All repairs must be made by a technical service authorized by Kartsana or its respective representative.

The manufacture will not be liable for any anomaly caused to the stretcher due to using any products other than original Kartsana products.

Other warranty clauses, see the Warranty Terms and Conditions that are supplied with the product.

1.3. Specifications

SPECIFICATIONS	BRAVO
Maximum unassisted load:	380Kg
Standards	UNE-EN-1865-5 EN-1789+A1
Total length	2052 mm
Total Width	605 mm
Recommended operatives required to load / unload an occupied stretcher	1 operative, or 2 if the stretcher were occupied by a larger sized patient.
Compatible stretchers	Power Brava

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

1.4. Attention

Through this sign, information is furnished on important safety measures for the correct use of the stretcher, to prevent accidents. The **warnings** alerts the reader about a situation which, if not avoided, could result in death or serious injury. The **cautions** alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur because of use or misuse.

1.5. Summary of safety precautions

Read carefully and strictly follow the warnings and cautions listed on these pages. Service only by qualified personnel.

WARNINGS:

- Improper usage of the stretcher can cause injury to the patient or operator. Operate the stretcher only as described in this manual.
- Do not modify the rail or any components of the rail. Modifying the product can cause unpredictable operation resulting in injury to the patient or operator. Modifying the product also voids its warranty.
- This rail must operate with a compatible stretcher.
- Have the vehicle safety rail compatible installed by a certified mechanic, following instructions in this manual. Improper rail installation can cause injury to the patient or operator and/or damage to the stretcher
- Practice changing height positions and loading the stretcher until operation of the product is fully understood. Improper use can cause injury.
- Do not allow untrained assistants to assist in the operation of the stretcher. Untrained technicians/assistants can cause injury to the patient or themselves.
- Do not ride on the base of the rail. Damage to the product could occur, resulting in injury to the patient or operator.
- Grasping the stretcher improperly can cause injury. Keep hands, fingers and feet away from moving parts. To avoid injury, use extreme caution when placing your hands and feet near the base tubes while raising and lowering the stretcher.
- Always use all restraint straps to secure the patient on the stretcher. An unrestrained patient may fall from the stretcher and be injured.
- The Bravo 1 rail is designed to be compatible with the Power Brava Kartsana stretchers, and is operator responsibility that these products work together.
- Two operators must be present when the stretcher is occupied.
- Operators must be able to lift the total weight of the patient, stretcher and any items on the stretcher.
- When cleaning, use any appropriate personal safety equipment (goggles, respirator, etc.) to avoid the risk of inhaling contagion.
- Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure the stretchers are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive residue on the surface of the stretchers, possibly causing premature corrosion of critical components.
- Failure to properly clean or dispose of contaminated mattress or other stretcher components will increase the risk of bloodborne pathogens and may cause injury to the patient or operator.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

CAUTIONS:

- Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.
- Installation of the safety hook should be done by a certified mechanic familiar with ambulance vehicle construction. Consult the vehicle manufacturer before installing the safety hook and be sure that the installation of the safety hook does not damage or interfere with the brake lines, oxygen lines, fuel lines, fuel tank or electrical wiring of the vehicle.
- Before operating the stretcher, clear any obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment, ensure that the caster wheels are safely set on the ground or damage to the product may occur.
- Ensure that the restraints are not entangled in the base frame when raising and lowering the stretcher.
- Do not steam clean or ultrasonically clean the unit.
- Maximum water temperature should not exceed 80°C.
- Failure to comply with these instructions may invalidate any/all warranties.
- For additional maintenance information, see the preventative maintenance information.
- Improper maintenance can cause injury or damage to the product. Maintain the stretcher as described in this manual. Use only Kartsana approved parts and maintenance procedures. Using unapproved parts and procedures could cause unpredictable operation and/or injury and will void the product warranty.
- Failure to use authorized parts, lubricants, etc. could cause damage to the stretcher and will void the warranty of the product.

NOTES:

- Loose items or debris on the patient compartment floor can interfere with the operation of the safety hook and stretcher fastener. Keep the patient compartment floor clear.
- This manual should be considered a permanent part of the rail and should remain with the product even if the stretcher is subsequently sold.
- Kartsana continually seeks advancements in product design and quality. Therefore, while this manual contains the most current product information available at the time of printing, there may be minor discrepancies between your stretcher and this manual.
- Kartsana recommends that, prior to installation, the certified mechanic plan the placement of the safety hook in the rear of the vehicle.


KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

2. MANUAL INSTRUCTIONS

Only the **BRAVA** stretcher may be put on the BRAVO 1 rail. (Based on the UNE-EN 1789 STANDARD).

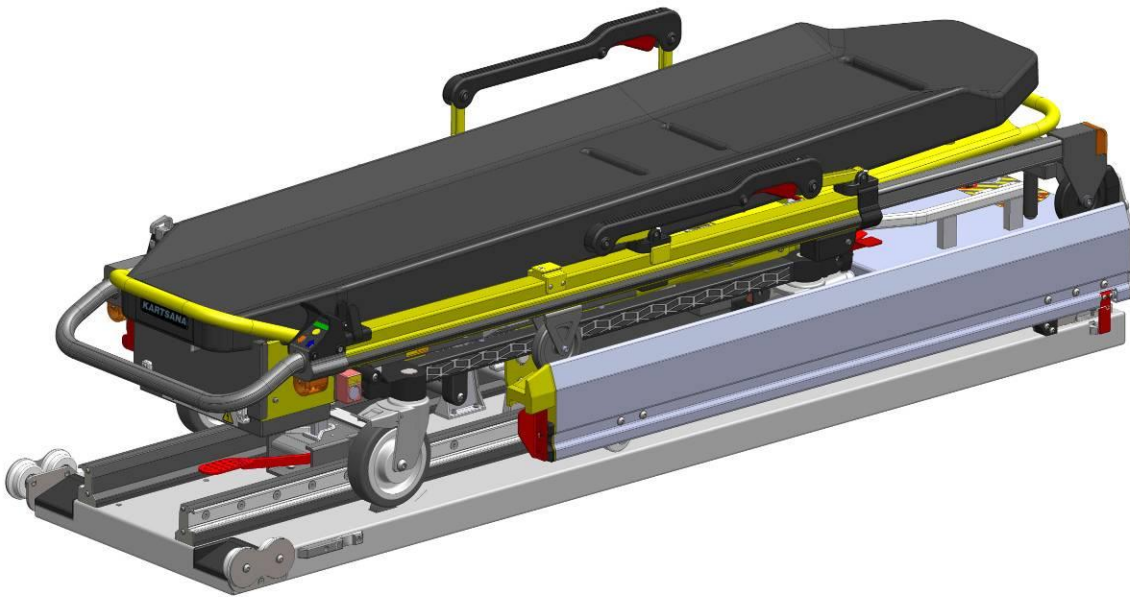To reduce the risk of injuries to the patient and accompanying persons in the event of an accident, it is advisable to avoid sharp edges and projecting parts inside the ambulance, paying special attention to the areas nearest the BRAVO 2 rail.

Make sure that no damage is caused to the areas containing the stretcher mechanisms, to prevent them from malfunctioning.

KARTSANA

Life-moving innovation


OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

2.1. Technical characteristics

Approximate rail weight: **95 Kg**

Maximum load permitted: **300 Kg.**

Approximate measurements of the stretcher (in mm.) based on the drawing below.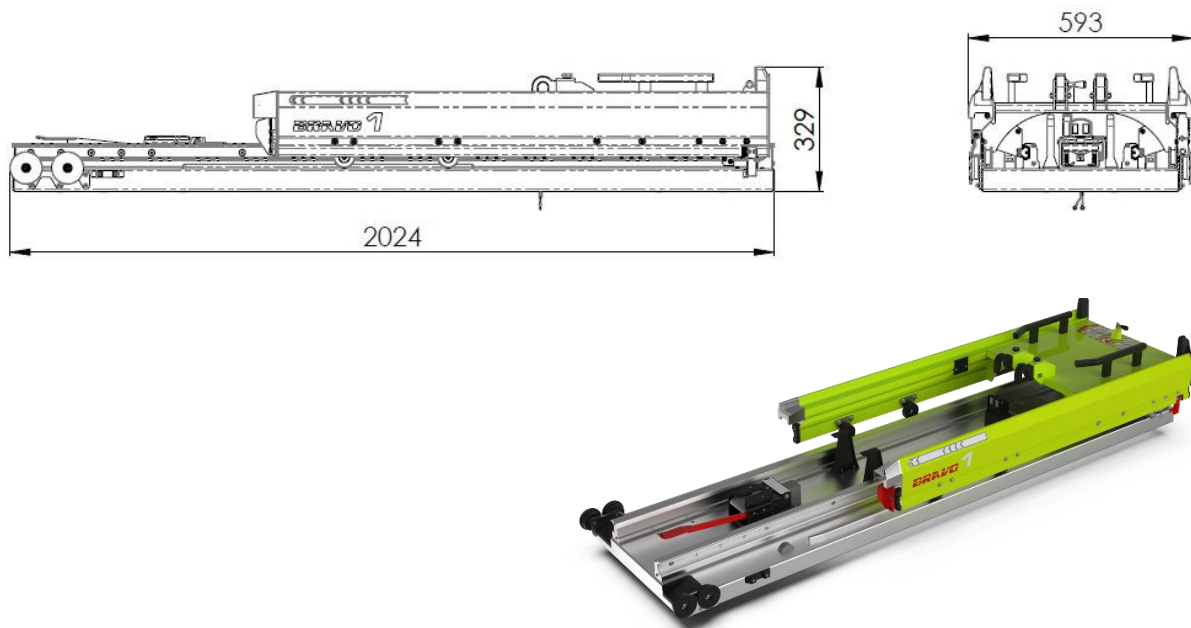

2.2. Rail operation and usage (R-1001 y R-100H)

To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

The basic operation of the R-1001 and R-1001H rails is the same. The added functionality of the R-1001H rail due to the use of the Hoverboard is detailed in section 2.3 of this document.

2.2.1. Red charging trigger

Stretcher lock on the rail:


- Place the stretcher in front of the rail.
- Place the small wheels of the stretcher on the rail cross section. This should be in the exterior position. Position the stretcher onto the rail so that the small wheels of the stretcher are guided by the rail, until the limit is reached at the edge of the cross section.
- Press the left button to raise the stretcher legs until the end of its run. (If this action has been carried out correctly the front axis should be within the inner part of the hooks.

KARTSANA


Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL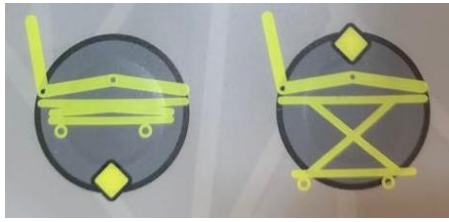


Leg folding and unfolding buttons

- Press the red trigger “B” at the front right of the rail to unlock the mechanism and to push the stretcher towards the inside until the limit has been reached.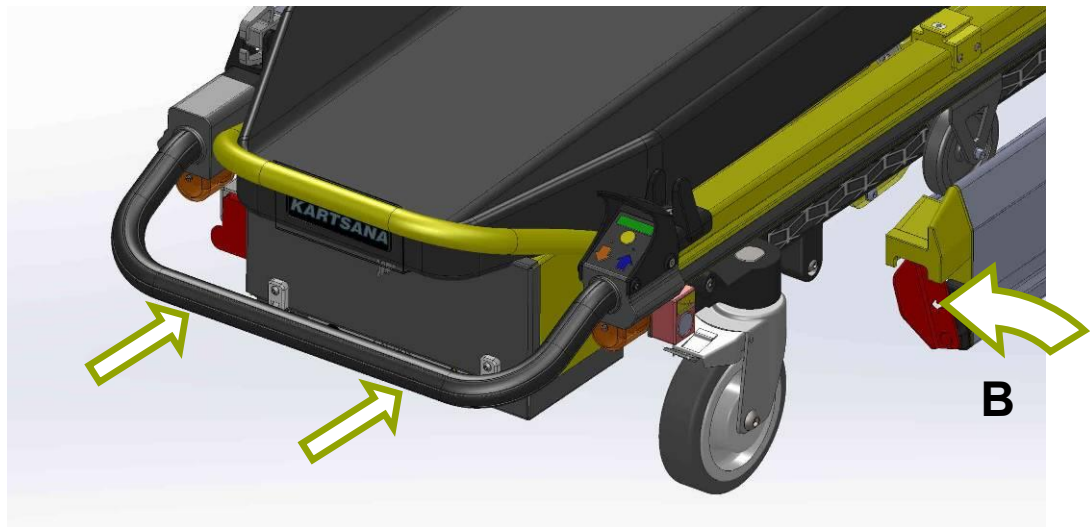

To be sure that the whole stretcher rail is correctly placed make sure that the unload lever is in the horizontal position. And the 3rd point is inside of its compartment.

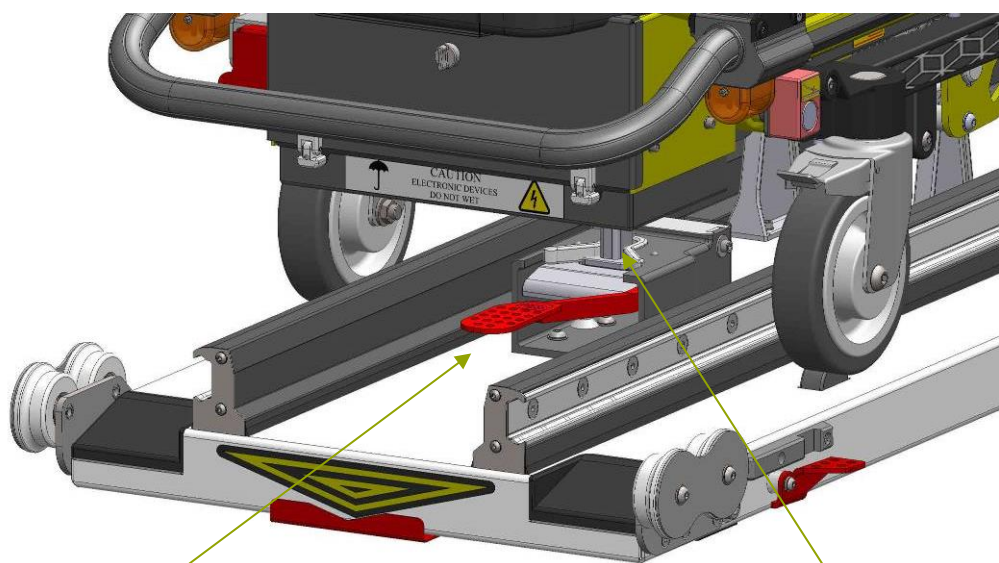Unloading
lever.

Third point

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

IMPORTANT: do not disconnect the supporting battery from the ambulance when it isn't doing any service, to allow the correct charge of the stretcher batteries between services.


2.2.2. Unloading lever

Push down the unloading lever "C" located between the guides at the front of the rail as shown in the figure below. While maintaining the lever in this position, pull the stretcher towards the outside of the ambulance until the mechanical limit has been reached. Once the rail is fully extended, we will be able to open out the stretcher legs by pressing the right button (Leg unfolding). Once the electric stretcher has been raised above the rail level, we can withdraw it.

To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

Note: The rear part of the rail corresponds to the part where the rail start lever is located.

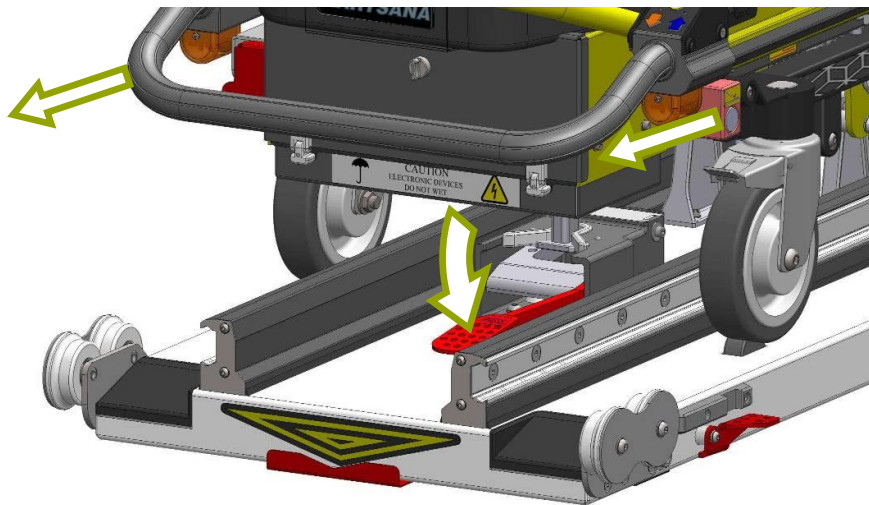IMPORTANT:

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

2.3. Operation and handling of the R-1001H rail attached to the Hoverboard

When the stretcher is loaded and locked, pressing the main switch automatically adjusts the Hoverboard to the patient's weight and raises it to the optimum comfort suspension level, then the switch lights up blue. If the resuscitation switch is also pressed, the device is raised to the highest position and remains in this position, the switch lights up red.

When the main switch of the Hoverboard is turned off, the Hoverboard lowers slowly, also when the stretcher is unlocked so that the patient only must be pushed and never lifted by hand.

By pressing either of the push buttons (front or rear), the lateral movement of the Hoverboard is unlocked. While pressing the button, the Hoverboard can be moved laterally in 8 different positions, each 32 mm from the previous one. When you release the button, the Hoverboard locks in the closest position.

KARTSANA

Life-moving innovation


OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

3. ASSEMBLY OF THE RAIL IN THE AMBULANCE


3.1. Setting the rail in the ambulance

The Bravo 1 rail must be secured to the ambulance using 10 screws DIN 912 M10 – 10.9.

The drilling situation is as follows: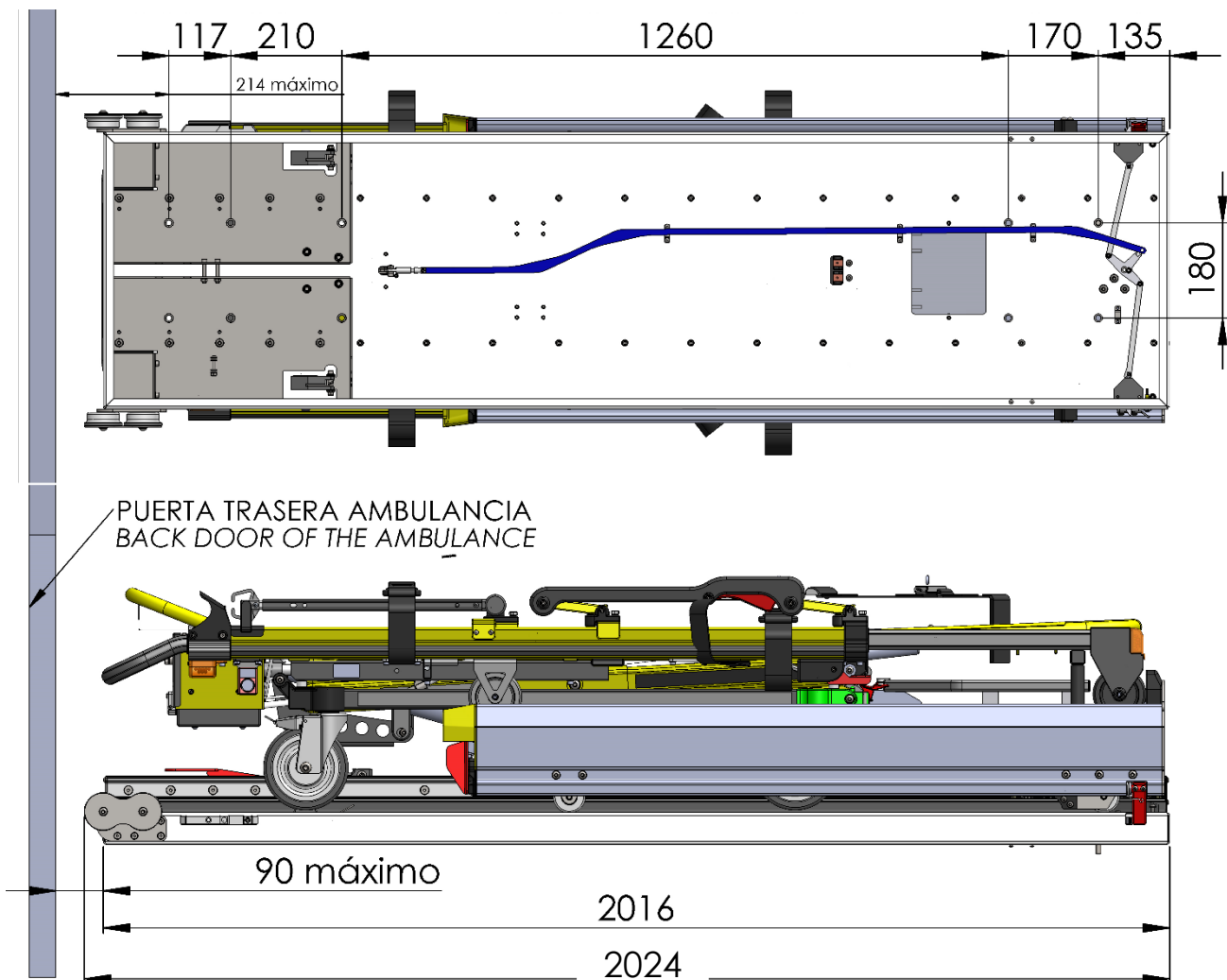

FLOOR FLATNESS TOLERANCE

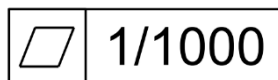KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

3.2. Electrical connection


The electronics is prepared to admit a current of **12 V**.


The BRAVO rail must be connected to the electricity network of the vehicle using section 4 mm² electric cable as a minimum (paying special attention to the poles + o -). The aim of this connection is that the Brava stretcher charges the stretcher batteries when it is inside the ambulance with the rail closed and the third position lever in horizontal position.
L

It's highly recommended to protect externally the wires with a plastic cover to prevent electrical problems (scratches, short-circuits...)

The electrical system must be secured with a 20A fuse.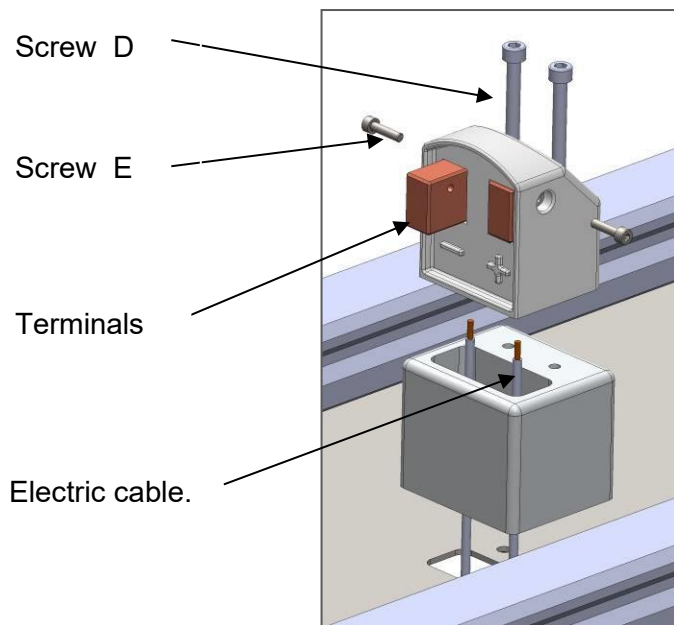

To make the electrical connection is necessary to remove charger outlet box with screws "D" and then using the screw terminals "E" fix the power cord into the respective contact. After reassembling the whole charger plug.

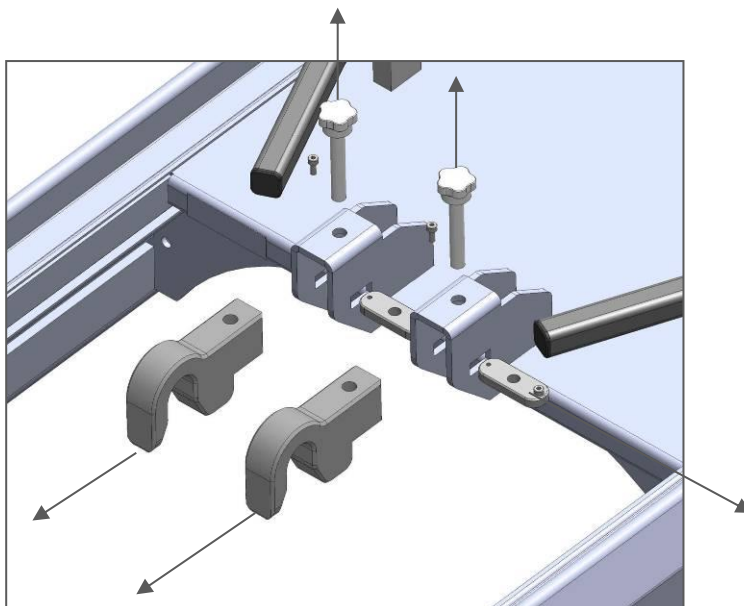In case the electric stretcher breaks down, take down the rail hooks according to the sketch, take off the stretcher and replace said hooks in their original position.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

3.3. Fixing the rail and the Hoverboard to the ambulance

For the correct installation of the Hoverboard device and the R-1001H rail, please refer to the documentation concerning the Hoverboard device.


KARTSANA

Life-moving innovation


OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

4. INTERMEDIATE POSITION (OPTIONAL)

If you want to rise the stretcher inside the ambulance, you must remove the trigger of the slot as you can see below and move the mobile part of the rail until the vertical trigger stays inside the black piece of plastic. When you want to return at locked position of the stretcher with the rail or if you want to take out the stretcher, you must lift the trigger into the slot.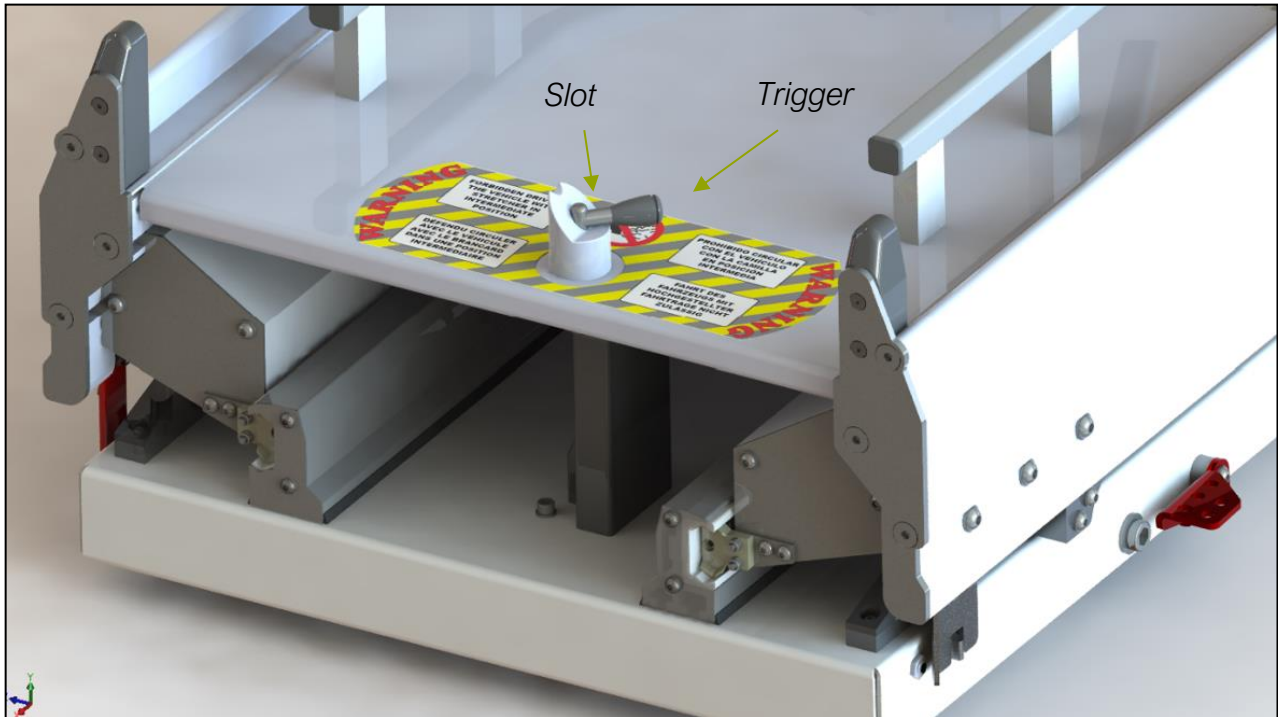

*Vertical trigger
inside de black
piece*

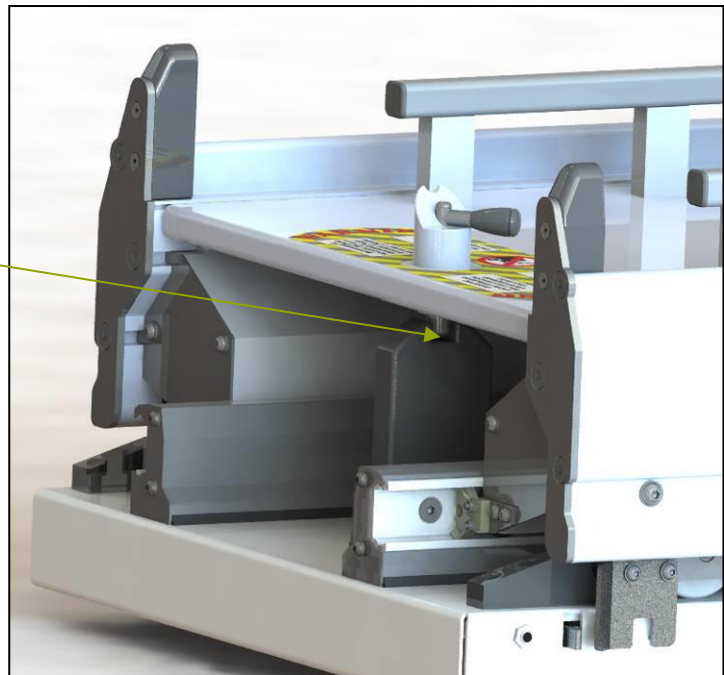KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

IMPORTANT:

5. DISINFECTION

When disinfecting the stretcher, use products that will not damage the surface of the materials and wipe with a cloth or something similar. If the surface to be disinfected is greased, re-greased it after disinfecting it.

All disinfectants must be used in accordance with the manufacturer's instructions.

The manufacturer will not be held responsible for any anomaly or damage caused by using a cleaning product that could damage the surface of the rail materials.

To ensure the hygiene and proper conservation of the product components, the manufacturer recommends that they be disinfected after each use.

6. MAINTENANCE

Contact information

For any questions related to the use, maintenance or installation of a KARTSANA product, you can contact our Technical Support Service (TSS). You can also visit our website for any issue related to spare parts or corrective actions.

- Telephone: +34 93 715 86 72
- E-mail: sat@kartsana.com
- Address:
Narcís Monturiol, 34
08192 - Sant Quirze del Vallès
Barcelona (España)

Maintenance

Check and grease the inside of the side guides and the locking systems every six months. We recommend maintenance by a specialist from our company every 6 months. During this inspection, carry out a general check to replace any parts that may be deteriorated or worn due to use.

The rail must be treated regularly (especially in cold winter temperatures) with grease to protect against corrosion or external agents such as salt and disinfectants. And that facilitates a good movement of the guide rail system, e.g. WD-40 without silicone.

Transport and storage

Before transporting the device, ensure that it is properly packaged and secured to prevent damage during transport.

Keep the original packaging for possible further transport. Any damage caused to the device during transport is not covered by the warranty.

Replacement and replacement of damaged parts are at the customer's expense. The device must be stored in a dry place.

During storage, do not place any heavy objects on top of the product. It must not be used as a support for any kind of object.

Disposal

When the product is no longer in a usable condition, if it has not been contaminated by any corrosive or contagious agents, it can be disposed of as any metal-based waste, following the local regulations in force regarding the disposal of metal-based waste.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING OF R-1001 AND R-100H BRAVO 1 RAIL

7. SPARE PARTS

For spare parts for this product, please visit the following link:

https://www.kartsana.com/global/Spare_parts_R-1001_R1002.pdf

If you have any questions or queries, please contact our technical service.

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