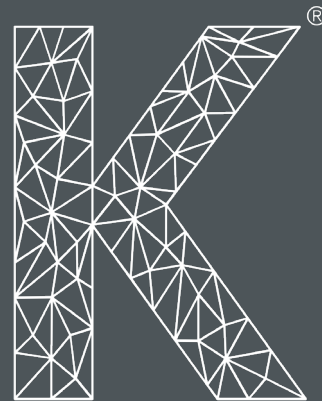
Serie CS 200

R-800 CS

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

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Instruction manual

Mechanical rail R-800 CS

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Ancrage mécanique R-800 CS

ESPAÑOL

ENGLISH

FRANÇAIS

R-800 CS

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Modelo R-800 CS

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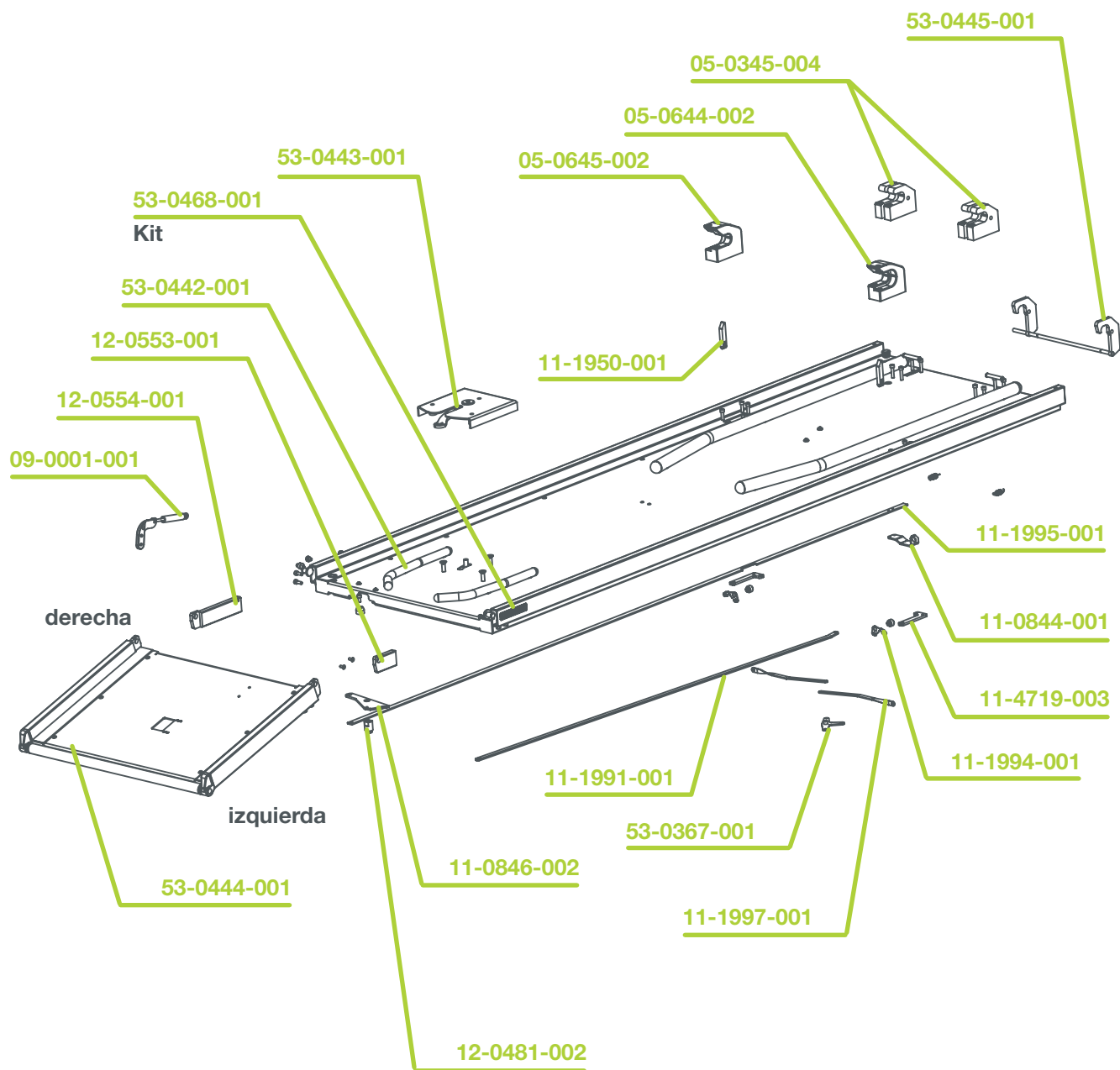
Mechanical rail

Model R-800 CS

Please read these maintenance instructions carefully before using the product and keep them for future reference.

CONTENTS

1. INTRODUCTION	20
1.1 Preface	20
1.2 Liability and warranty	20
1.3 - Specifications	20
1.4 Precautions	21
1.5 Summary of safety precautions	21
2. INSTRUCTION MANUAL	23
2.1 Technical characteristics of the rail	23
2.2 Mounting the stretcher on the rail	24
3. MOUNTING THE RAIL ON THE AMBULANCE	28
3.1 Vehicle configuration	28
3.2 Fixing the rail on the ambulance	28
4. CARE AND MAINTENANCE	29
4.1 Contact information	29
4.2 Environmental conditions	29
4.3 Cleaning	29
4.4 Maintenance	30
4.4.1 Precautionary maintenance	30
4.4.2 Service maintenance	31
4.5 Transportation and storage	31
4.6 Disposal	31
5. SPARE PARTS	32

1. INTRODUCTION

1.1 Preface

The **R-800 CS** rail has been specifically designed for perfect attachment to the Kartsana CS 200 stretcher model.

The R-800 CS rail has been tested in accordance with the UNE-EN 1789 and UNE-EN 1865-5 standards.

All information on treatment, disinfection and maintenance is based on our experience and current knowledge.

We reserve the right to make changes in the technical specifications of the rail in order to improve the product.

1.2 Liability and warranty

The rail must be checked at the time of delivery to the relief organisation. All of the functions must be explained in detail. The relief organisation is responsible for training all employees in the correct use of the equipment.

The product has a warranty period of 24 months from the date indicated on the purchase invoice. (see Warranty Conditions and Warranty Certificate enclosed with the product).

The Warranty will not cover breakdown due to incorrect installation, mishandling or misuse of the product. Repairs must be carried out by a service technician authorised by Kartsana or its authorised representative.

The manufacturer shall not be liable for any damage to the rail caused by the use of non-original Kartsana products. Changes or modifications made to the unit that have not been expressly approved by Kartsana may result in errors in the equipment's operation.

1.3 - Specifications

Maximum workload (unassisted):	350 Kg
Main regulations	UNE-EN-1865-5 EN-1789+A1
Total length	2028/2457 mm
Total width	638 mm
Height	499 mm
Recommended operators for loading/unloading an occupied stretcher	1 operator or 2 operators in the event that the stretcher is used by a very large person
Compatible stretchers	Stretcher CS 200

1.4 Precautions

Throughout this manual, there are 3 types of symbols. As a **warning symbol**, we have ISO 7010-W001. The **prohibition** is indicated by ISO 7010-P001 and the **obligation** by ISO 7010-M001.

We use these signs to provide information on important safety measures for the correct use of the rail and to avoid possible accidents.

Warnings alert the reader to situations that, if not avoided, may lead to death or serious injury. Furthermore, prohibitions indicate actions that must not be carried out under any circumstance, and obligations indicate measures and/or actions that must be taken to ensure the proper use of the product and to maintain safe functionality.

1.5 Summary of safety precautions

Read and follow the warnings and cautions on these pages carefully. Maintenance and repairs may only be carried out by qualified personnel.

PRECAUTIONS

- Improper use of the rail may cause injury to the patient or the handler. Use the rail only as described in this manual.
- Do not modify the rail or any of its components. Modification of the device may result in an unpredictable operation that could cause injury to the patient or operator. Modification of the device voids the warranty.
- This rail must work with a compatible stretcher.
- The fastening system to the emergency vehicle compatible with our rail must be correctly installed for the correct fastening of the stretcher. Otherwise injury to patient, operator or vehicle could occur.
- Request the correct installation of the fixing rail to the vehicle floor. The position of this rail will determine the correct functioning of the stretcher. After installation, practice changing the rail stretcher entry and exit positions until you are familiar with the product. Improper use can lead to injury.
- Do not allow untrained assistants to help you operate the rail. Inadequately trained technicians and assistants can cause injury to the patient and to themselves.
- Do not climb on the base of the rail as this may damage the product, unbalance the patient and/or cause injury.
- Incorrect fastening of the stretcher may result in injury. Avoid placing hands, fingers or feet on the moving parts of both the rail and the stretcher. To avoid injury, when raising and lowering the stretcher, be very careful when placing hands and feet near the moving parts (inlet tray).
- The R-800 CS rail is designed to be compatible with the Kartsana CS 200 stretcher.
- When cleaning, use appropriate personal protective equipment (protective goggles or respirators) to avoid the risk of inhaling infectious organisms.
- Some cleaning products are corrosive and can damage the product if used incorrectly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure that the rails are cleaned with clean water and thoroughly dried after cleaning. Failure to properly rinse and dry the units may leave a corrosive residue on the surface of the units, which may cause premature corrosion of critical components.

PRECAUTIONS

- Changes or modifications to the unit not expressly approved by Kartsana may void the user's authority to operate the system.
- The installation of the stretcher-compatible fixing rail must be carried out by a certified mechanic familiar with the ambulance structure. Consult the vehicle manufacturer before installing the rail and ensure that the installation does not damage or interfere with the vehicle's brake, oxygen or fuel hoses, fuel tank or electrical wiring.
- Before putting the stretcher or the rail into operation, remove all obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment (from the rail), make sure that the wheels of the stretcher are placed securely on the floor, otherwise the product may be damaged.
- Ensure that the belts do not become entangled in the rail when loading and unloading the stretcher.
- Do not use steam or ultrasonic cleaners to clean the unit.
- Failure to comply with these instructions may result in some or all of the warranties becoming void.
- A preventive maintenance programme should be established for all Kartsana equipment. Depending on the product's frequency of use, preventive maintenance may need to be carried out more frequently.
- For further information on maintenance, please refer to our technical department.
- Improper maintenance may cause injury or damage to the product. The rail should be maintained as described in this manual. Follow the maintenance procedures and use only Kartsana approved parts. The use of unapproved parts or procedures may result in an unpredictable operation and may cause injury and void the warranty.
- Failure to use authorised spare parts, lubricants, etc. may damage the rail and void the product warranty.

NOTES

- Loose elements or dirt accumulation on the floor of the ambulance compartment can interfere with the operation of the rail and the stretcher retaining system. Keep the floor of the patient compartment clean.
- This manual should be considered as part of the rail and as such should be kept with the product even if it is sold.
- Kartsana works constantly to improve the design and quality of its products. Therefore, although this manual contains the most current information available at the time of printing, there may be minor discrepancies between the rail and this manual. If you have any questions, please contact Kartsana's customer service or technical service area.
- Kartsana recommends that, prior to installation, a certified mechanic should plan the placement of the mounting rail on the inside of the emergency vehicle.

2. INSTRUCTION MANUAL

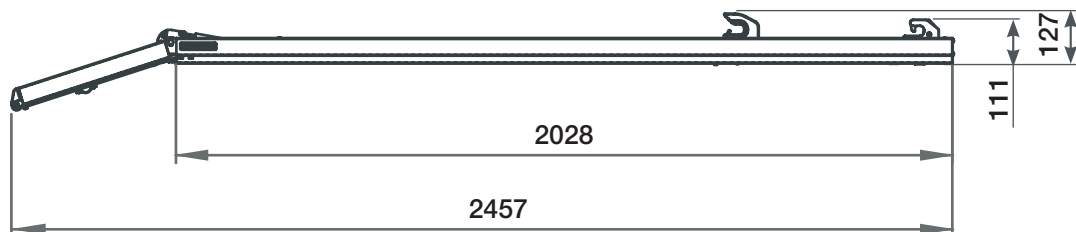
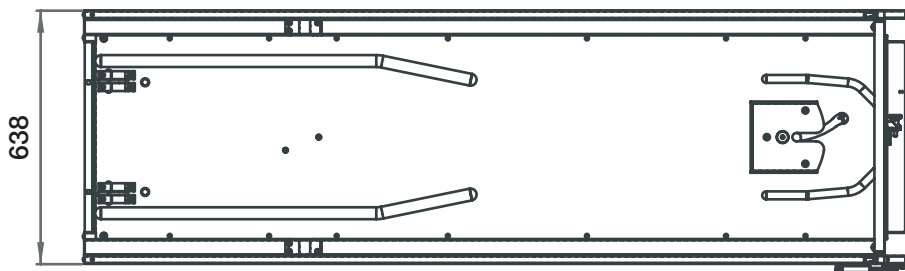
2.1. Technical characteristics of the R-800 CS rail (CS-200)

Only the CS-200 stretcher can be mounted on the R-800 CS rail.

Rail weight: 30 Kg
Maximum permissible load: 350 kg
Rail open length: 2457 mm



(Dimensions and measurements
in mm)

2.2. Assembly of the CS 200 stretcher with R-800 CS rail



Note that the minimum loading height for the stretcher legs to unlock must be **700 mm**.

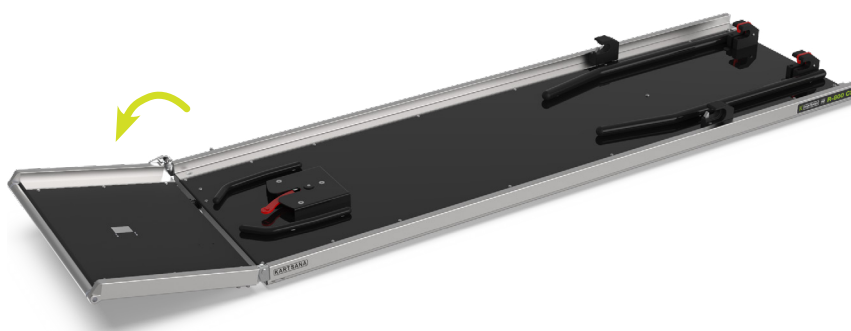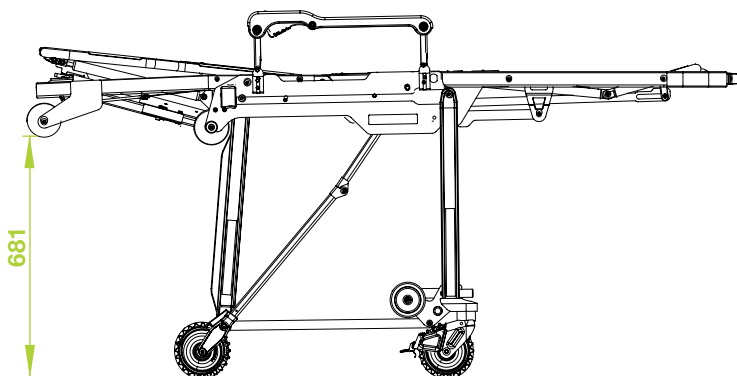In order to make folding possible, the front wheels of the stretcher need to be raised.


Place the rail tray in a horizontal (open) position. This way, the red fixing hooks remain open.

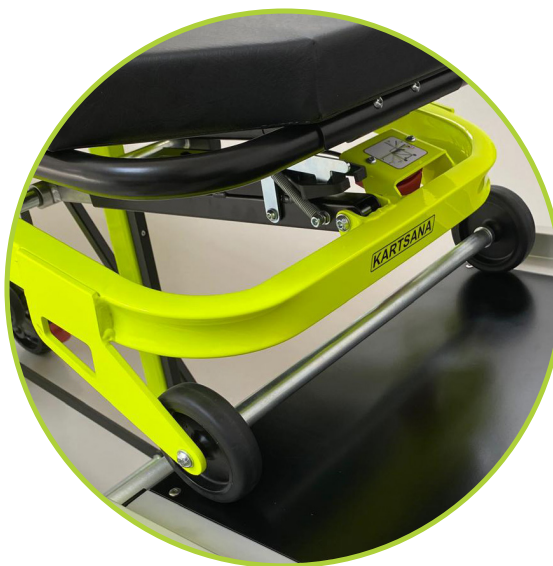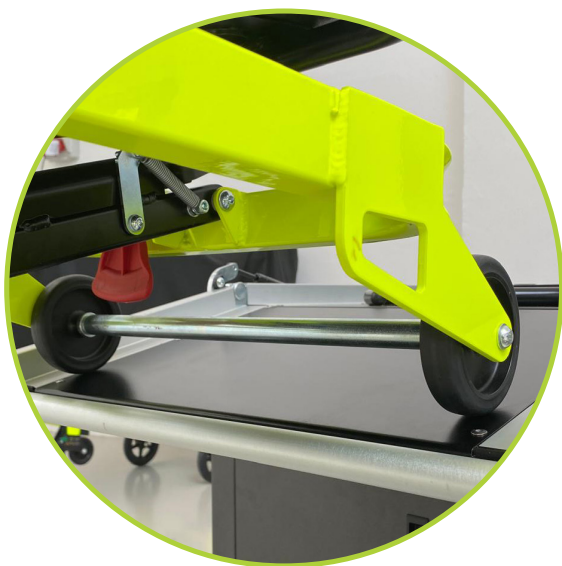Align the front wheels on the rail so that they slide over the tray, always guided by the tubes to prevent lateral movement.


Loading the stretcher on the rail

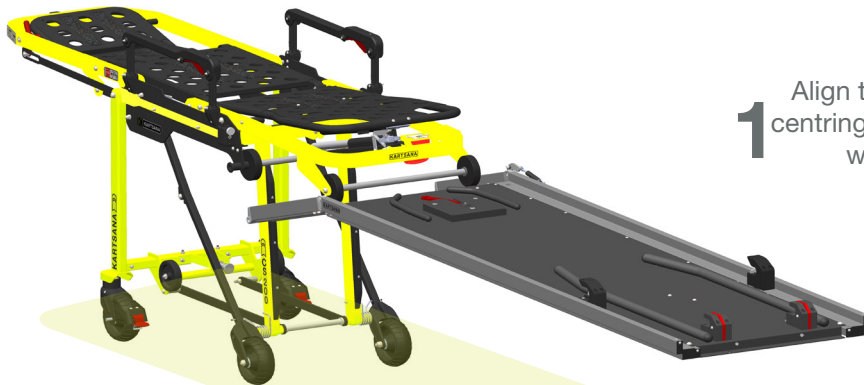1 Align the stretcher by centring the front wheels with the rail.

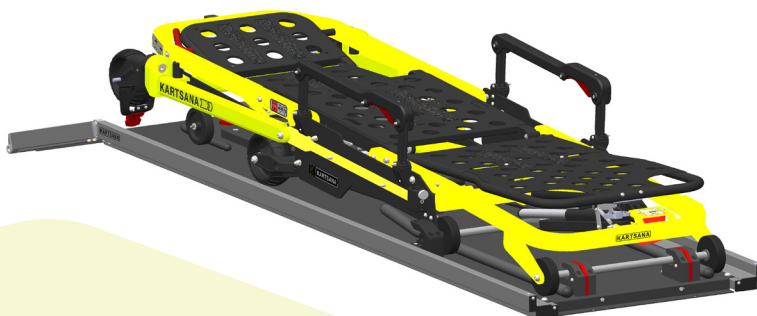2 In order to fold the legs and fit on the rail, the red trigger on the underside of the leg must be pressed. Using the hips, help to push and load the stretcher into the ambulance.

3

Align the stretcher by centring the front wheels with the rail. Unlock the rear leg, and push the stretcher until the front wheel axle hits the two front hooks

4

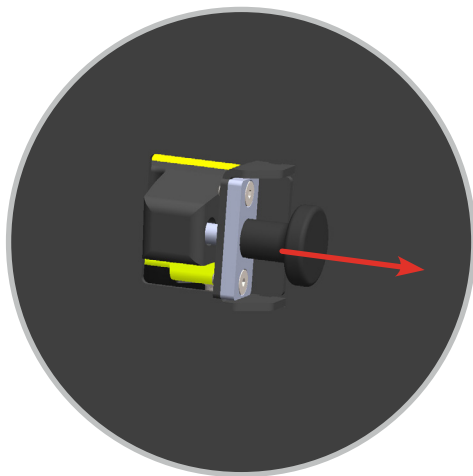
When the wheels reach the end of the rail, the rear pin will enter the stainless steel structure, preventing vertical movement of the stretcher.

5

To ensure the red hooks close and secure the stretcher at the front, lift the rail entry tray.

To ensure the entire assembly is properly locked, once the tray is raised, insert the manual lock that is located on it.



Unloading of the stretcher when on the rail

To unload the stretcher, the procedure is as follows:



1

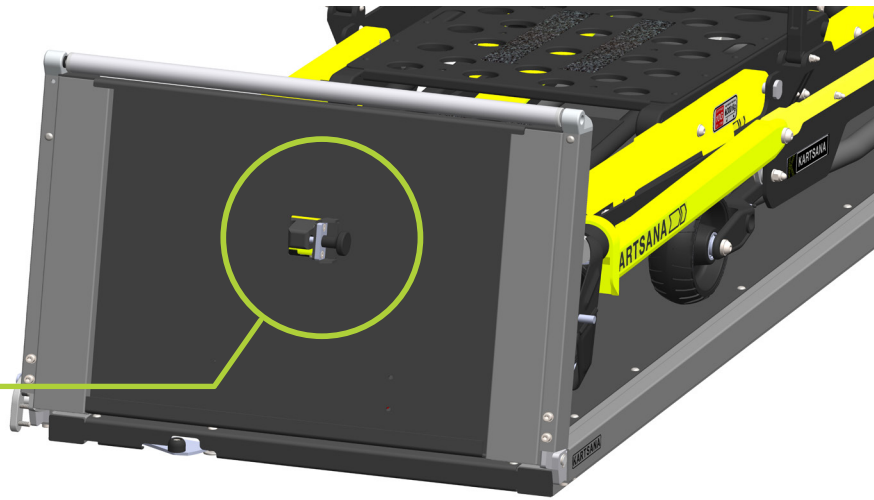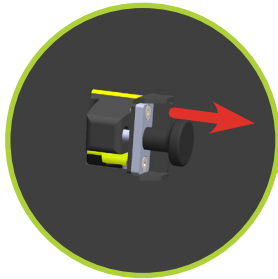Remove the safety lock and lower the tray to release the front hooks.

2

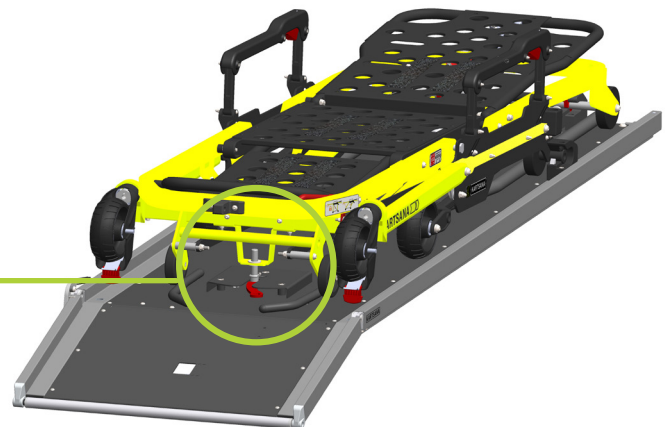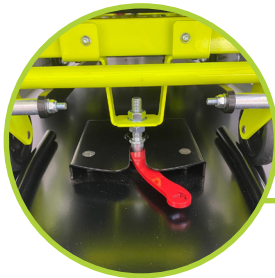
Unlock the red lever to remove the stretcher.

3

Pull the stretcher so that it unfolds on its own.

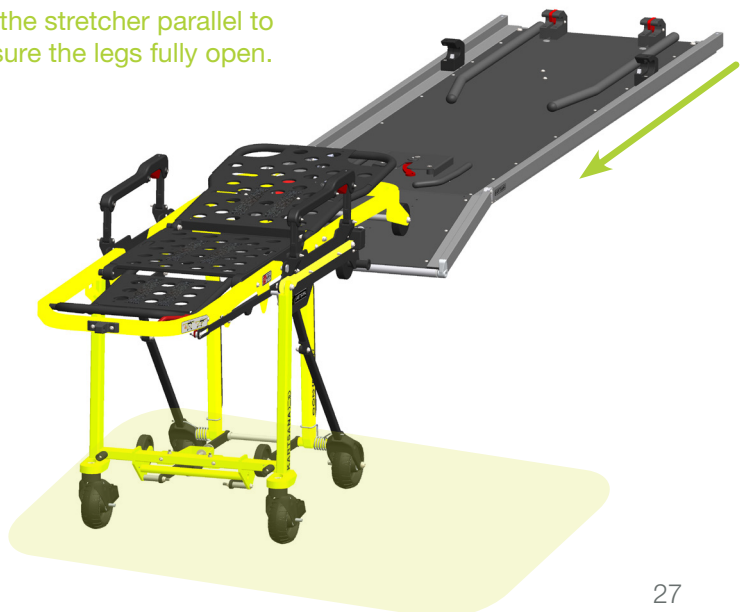Always keep the stretcher parallel to the rail to ensure the legs fully open.

3. MOUNTING THE RAIL ON THE AMBULANCE

3.1 Vehicle configuration

These installation instructions are for securing the CS 200 assembly to the Kartsana R-800 CS rail.

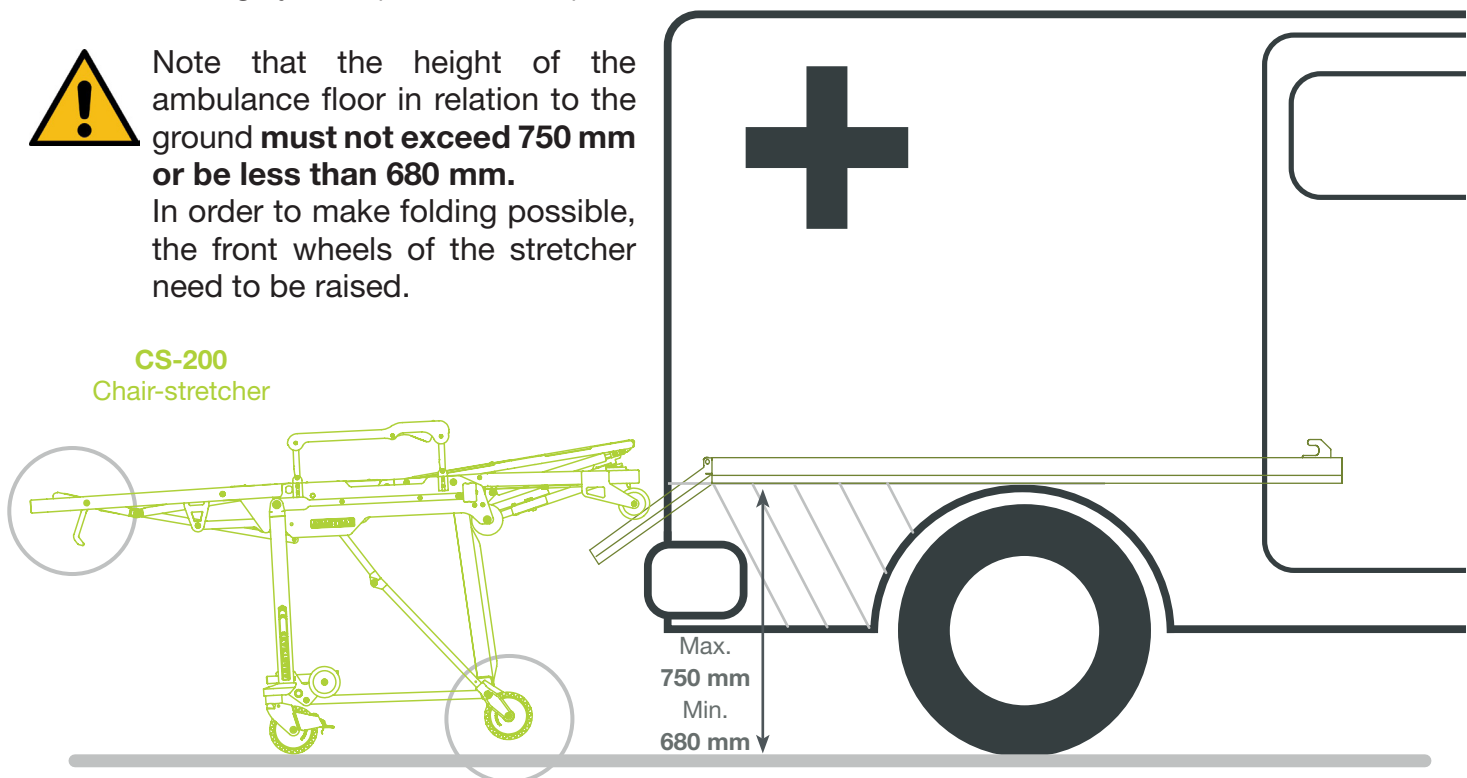
The proper use and functioning of the CS-200 stretcher depends on the correct installation of the fastening system (R-800 CS rail).

Note that the height of the ambulance floor in relation to the ground **must not exceed 750 mm or be less than 680 mm.**

In order to make folding possible, the front wheels of the stretcher need to be raised.





CS-200
Chair-stretcher

3.2 Fixing the rail on the ambulance

The R-800 CS rail must be fixed to the ambulance floor with the holes indicated in the diagram below.

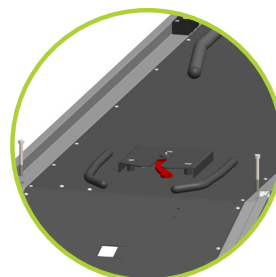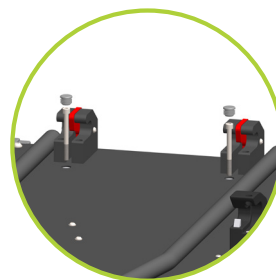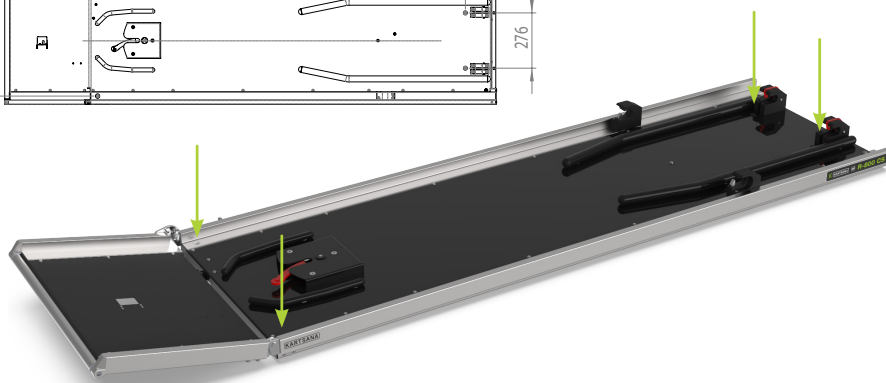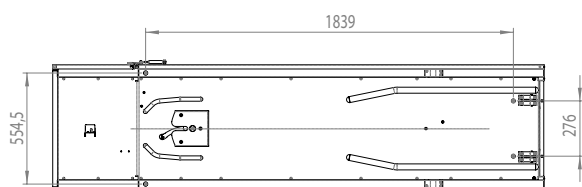4 Screws DIN 912 M8x90 mm

4. CARE AND MAINTENANCE OF THE PRODUCT

4.1. Contact information

If you have any questions regarding the use, maintenance or installation of a KARTSANA product, please contact our Technical Service Area (TSA).

Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient are located.

Telephone: +34 93 715 86 72

Email: sat@kartsana.com

Address: Narcís Monturiol, 34
08192 Sant Quirze del Vallès,
Barcelona (Spain)

4.2. Environmental conditions

Both for use and storage:

Recommended temperature: from -10°C to +60°C

Relative humidity: 5 – 95%

4.3. Cleaning

To clean the rail, use a cloth, rag or similar to wipe the surface to be cleaned with products that do not damage the surface of the materials. If the surface to be cleaned is already lubricated, re-grease it after cleaning has been completed.

Please use cleaning agents according to the manufacturer's instructions.

The manufacturer shall not be liable for any damage or anomaly caused by a cleaning product that may damage the surface of the rail materials.

For reasons of hygiene and preservation of the product components, the manufacturer recommends cleaning after each use.

4.4. Maintenance

4.4.1 Precautionary maintenance

Correct and regular maintenance guarantees the durability of the product.

Creating a maintenance plan, including regular reviews and establishing a responsible person to carry them out, is recommended.

The person in charge must adhere to the following **requirements**:

- Possess technical knowledge related to the product and the maintenance procedures described in these instructions.
- To have the support of qualified, trained and knowledgeable staff who are familiar with the operation of the product.
- Use original or manufacturer-approved components, spare parts and accessories.
- In accordance with the instructions of EU Regulation 2017/45 which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability, records of all product maintenance operations shall be kept.

Check for proper operation **before use**. Prior to service, the following should be checked:

- Functionality.
- Fastenings and hardware.
- Parts that move, such as wheels, belts and mattresses.
- Sensitivity of springs.

If the device does not appear to be suitable for correct and safe use, **it should be taken out of service** until the device is repaired or restored to full working order.

Do not modify the structure of the device for any reason, as this may cause serious harm to patients and/or operators.

Lubrication intervals of moving parts:

- 1 – 30 services per month: every 3 months.
- 30 – 50 services per month: every 2 months.
- More than 150 services per month: every month.

All maintenance other than lubrication, tightening of nuts and bolts and ordinary cleaning must be carried out by KARTSANA or an authorised service centre.

4.4.2. Service maintenance

The person entrusted with the product review shall ensure the following basic requirements:

- ◇ Adequate knowledge of the product, its technical/constructional characteristics, control points and final tests, packaging, preservation and handling.
- ◇ Knowledge of all product functions and any possible risks, malfunctions or failures of the product.
- ◇ Possession of all necessary tools to carry out any technical service or minor repairs.
- ◇ Possession (or ability to procure) original manufacturer's or manufacturer's authorised spare parts.
- ◇ Use or support of specialised technical personnel trained by the manufacturer to service the product in question.
- ◇ Keeping a record of all maintenance operations carried out on the product, in accordance with the instructions of EU Regulation 2017/45 which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability.

4.5. Transport and storage

Before transporting the device, ensure that it is properly packaged and secured to prevent damage during transportation.

Keep the original packaging for possible further transport. Any damage caused to the device during transport is not covered by the warranty.

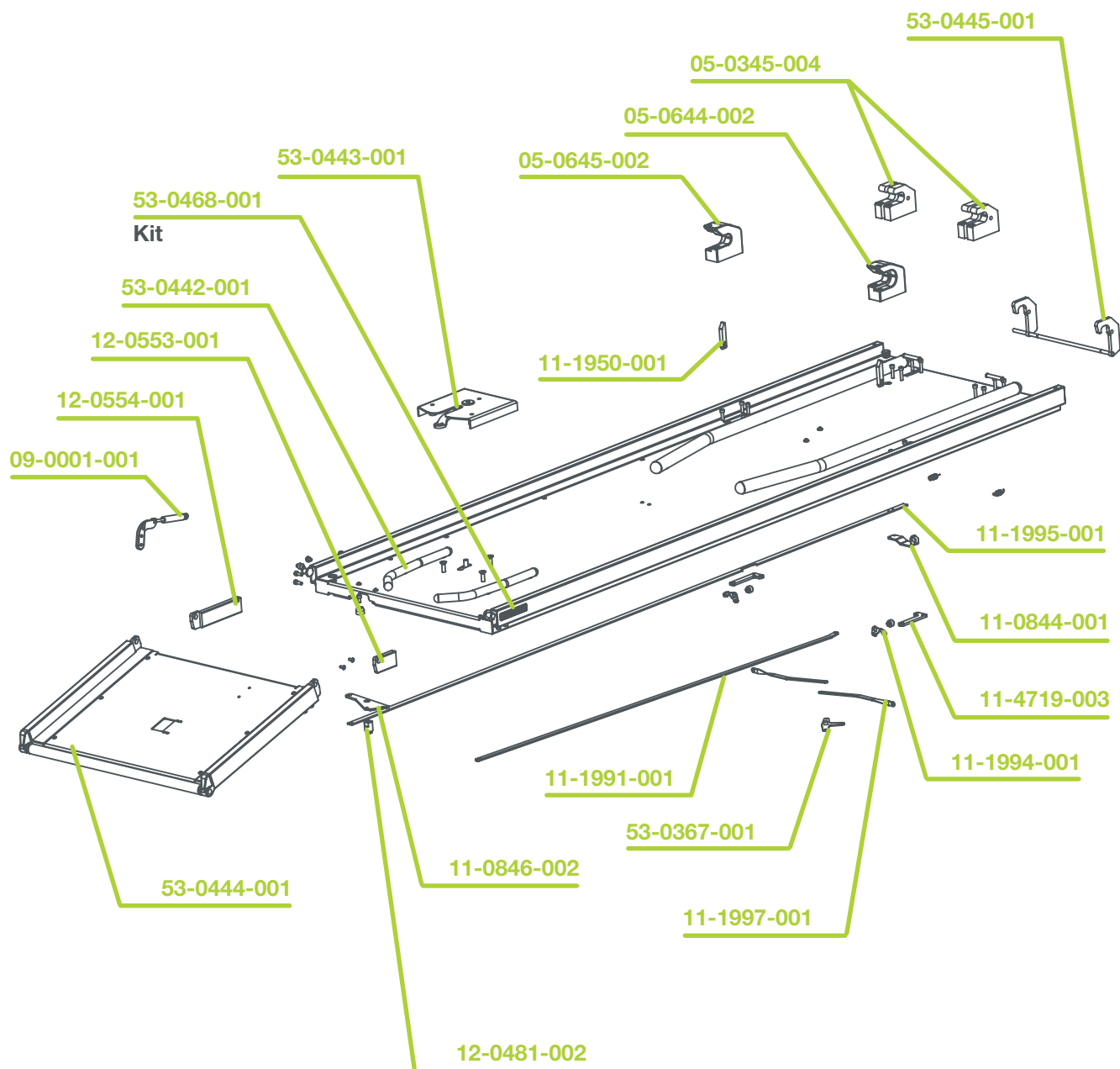
Repairs and replacements of damaged parts are the responsibility of the customer. The device should be stored in a dry place.

When storing, do not place any heavy objects on top of the product. It must not be used as a support for any kind of object.

4.6. Disposal

When the product is no longer in a usable condition, do not throw it away but develop a take-back strategy for recycling by an authorised dealer. The packaging shall be managed as reusable waste. Metals are fully recyclable. Plastic products should be treated as recycling material. Waste management should be as established in each country. If you have any questions, please ask your municipal administration.

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