
Serie SUPERBRAVO

R-2021 / R-2022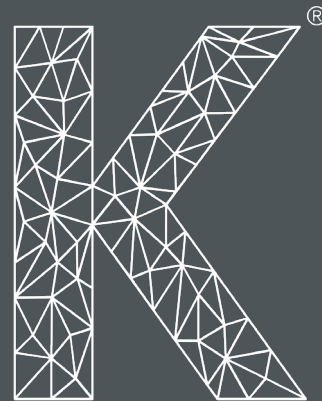

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

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Instruction manual

Electric rail SUPERBRAVO 1 and 2

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ESPAÑOL

ENGLISH

FRANÇAIS

SUPERBRAVO 1 / 2

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Modelo R-2021 / R-2022

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Electric rail

Model R-2021 / R-2022

Please read these maintenance instructions carefully before using the product and keep them for future reference.

Electric

R10

CONTENTS

1. INTRODUCTION	24
1.1 Preface	24
1.2 Liability and warranty	24
1.3 - Specifications	24
1.4 Precautions	25
1.5 Summary of safety precautions	25
2. INSTRUCTION MANUAL	27
2.1 Technical characteristics of the rail	27
2.2 Mounting the stretcher on the rail	28
2.3 Operation and handling of the stretcher	31
3. MOUNTING THE RAIL ON THE AMBULANCE	32
4. CARE AND MAINTENANCE	35
4.1 Contact information	35
4.2 Environmental conditions	35
4.3 Cleaning	35
4.4 Maintenance	36
4.4.1 Precautionary maintenance	36
4.4.2 Service maintenance	37
4.5 Transportation and storage	37
4.6 Disposal	37
5. SPARE PARTS	38
6. SYSTEM ERRORS	41

1. INTRODUCTION

1.1 Preface

The **SUPERBRAVO 1 and 2 (R-2021 / R-2022)** rails has been specifically designed for perfect attachment to the Kartsana SUPERBRAVA (TG-2022) model stretcher.

Note: Throughout this manual, **SUPERBRAVO** will be used to refer to both models. When some functionality is only available in one of the 2 models, this will be specified in the corresponding section.

The Superbravo rail has been tested in accordance with the UNE-EN 1865-5 standard. All information on treatment, disinfection and maintenance is based on our experience and current knowledge. We reserve the right to make changes in the technical specifications of the rail in order to improve the product.

1.2 Liability and warranty

The rail must be checked at the time of delivery to the relief organisation. All of the functions must be explained in detail. The relief organisation is responsible for training all employees in the correct use of the equipment.

The product warranty is 24 months from the date indicated on the purchase invoice. (See Warranty Conditions and Warranty Certificate enclosed with the product).

The Warranty will not cover breakdown due to incorrect installation, mishandling or misuse of the product. Repairs must be carried out by a service technician authorised by Kartsana or its authorised representative.

The manufacturer shall not be liable for any damage to the rail caused by the use of non-original Kartsana products. Changes or modifications made to the unit that have not been expressly approved by Kartsana may result in errors in the equipment's operation.

1.3 Specifications

	R-2021	R-2022
Maximum workload (unassisted):	400kg	400kg
Main regulations	UNE-EN-1865-5 EN-1789+A1	
Total length	2120 mm	2150 mm
Total width	618 mm	674 mm
Height	320 mm	372 mm

Recommended operators for loading/unloading an occupied stretcher	1 operator or 2 operators in the event that the stretcher is used by a very large person
Compatible stretchers	TG-2022 Superbrava Stretcher

1.4 Precautions

Throughout this manual, there are 3 types of symbols. As a **warning symbol**, we have ISO 7010-W001. The **prohibition** is indicated by ISO 7010-P001 and the **obligation** by ISO 7010-M001.

We use these signs to provide information on important safety measures for the correct use of the rail and to avoid possible accidents.

Warnings alert the reader to situations that, if not avoided, may lead to death or serious injury. Furthermore, prohibitions indicate actions that must not be carried out under any circumstance, and obligations indicate measures and/or actions that must be taken to ensure the proper use of the product and to maintain safe functionality.

1.5 Summary of safety precautions

Read and follow the warnings and cautions on these pages carefully. Maintenance and repairs may only be carried out by qualified personnel.

PRECAUTIONS

- Improper use of the rail may cause injury to the patient or the handler. Use the rail only as described in this manual.

- Do not modify the rail or any of its components. Modification of the device may result in an unpredictable operation that could cause injury to the patient or operator. Modification of the device voids the warranty.

- This rail must work with a compatible stretcher.

- The fastening system to the emergency vehicle compatible with our rail must be correctly installed for correct fastening of the stretcher. Otherwise injury to patient, operator or vehicle could occur.

- Request the correct installation of the fixing rail to the vehicle floor. The position of this rail will determine the correct functioning of the stretcher. After installation, practice changing the rail stretcher entry and exit positions until you are familiar with the product. Improper use can lead to injury.

- Do not allow untrained assistants to help you operate the rail. Inadequately trained technicians and assistants can cause injury to the patient and to themselves.

- Do not climb on the base of the rail as this may damage the product, unbalance the patient and/or cause injury.

- Incorrect fastening of the stretcher may result in injury. Avoid placing hands, fingers or feet on the moving parts of both the rail and the stretcher. To avoid injury, when raising and lowering the stretcher, be very careful when placing hands and feet near the moving parts (inlet tray).

- The SUPERBRAVO rail is designed to be compatible with the Kartsana SUPERBRAVO (TG-2022) stretcher.

- When cleaning, use appropriate personal protective equipment (protective goggles or respirators) to avoid the risk of inhaling infectious organisms.

- Some cleaning products are corrosive and can damage the product if used incorrectly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure that the rails are cleaned with clean water and thoroughly dried after cleaning. Failure to properly rinse and dry the units may leave a corrosive residue on the surface of the units, which may cause premature corrosion of critical components.

PRECAUTIONS

- Changes or modifications to the unit not expressly approved by Kartsana may void the user's authority to operate the system.
- The installation of the stretcher-compatible fixing rail must be carried out by a certified mechanic familiar with the ambulance structure. Consult the vehicle manufacturer before installing the rail and ensure that the installation does not damage or interfere with the vehicle's brake, oxygen or fuel hoses, fuel tank or electrical wiring.
- Before putting the stretcher or the rail into operation, remove all obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment (from the rail), make sure that the wheels of the stretcher are placed securely on the floor, otherwise the product may be damaged.
- Ensure that the belts do not become entangled in the rail when loading and unloading the stretcher.
- Do not use steam or ultrasonic cleaners to clean the unit.
- Failure to comply with these instructions may result in some or all of the warranties becoming void.
- A preventive maintenance programme should be established for all Kartsana equipment. Depending on the product's frequency of use, preventive maintenance may need to be carried out more frequently.
- For further information on maintenance, please refer to our technical department.
- Improper maintenance may cause injury or damage to the product. The rail should be maintained as described in this manual. Follow the maintenance procedures and use only Kartsana approved parts. The use of unapproved parts or procedures may result in an unpredictable operation and may cause injury and void the warranty.
- Failure to use authorised spare parts, lubricants, etc. may damage the rail and void the product warranty.

NOTES

- Loose elements or dirt accumulation on the floor of the ambulance compartment can interfere with the operation of the rail and the stretcher retaining system. Keep the floor of the patient compartment clean.
- This manual should be considered as part of the rail and as such should be kept with the product even if it is sold.
- Kartsana works constantly to improve the design and quality of its products. Therefore, although this manual contains the most current information available at the time of printing, there may be minor discrepancies between the rail and this manual. If you have any questions, please contact Kartsana's customer service or technical service area.
- Kartsana recommends that, prior to installation, a certified mechanic should plan the placement of the mounting rail on the inside of the emergency vehicle.

2. INSTRUCTION MANUAL

2.1. Technical characteristics of the SUPERBRAVO 1 and 2 (R-2021 / R-2022) rails

Only the SUPERBRAVA (TG-2022) stretcher can be mounted on the SUPERBRAVO rail.

R-2021

Weight: 112Kg

Rail open lenght: 3326 mm



R-2022

Weight: 143 Kg

Rail open length 3326 mm

(Dimensions and measurements in mm)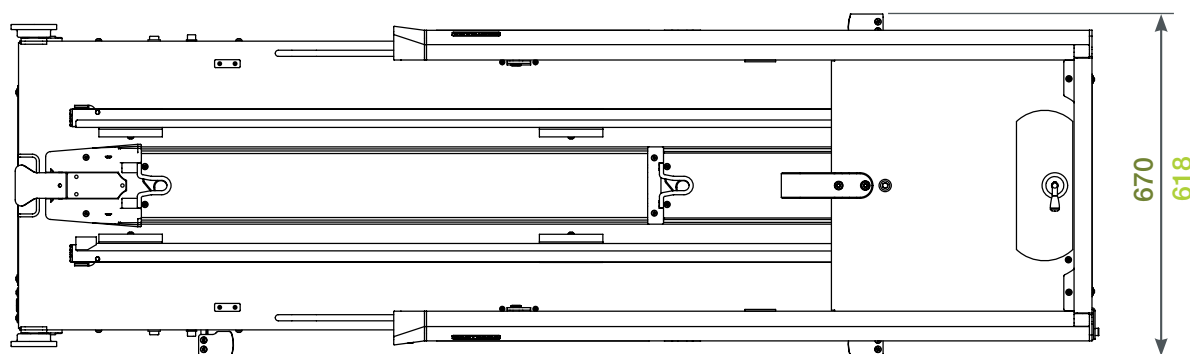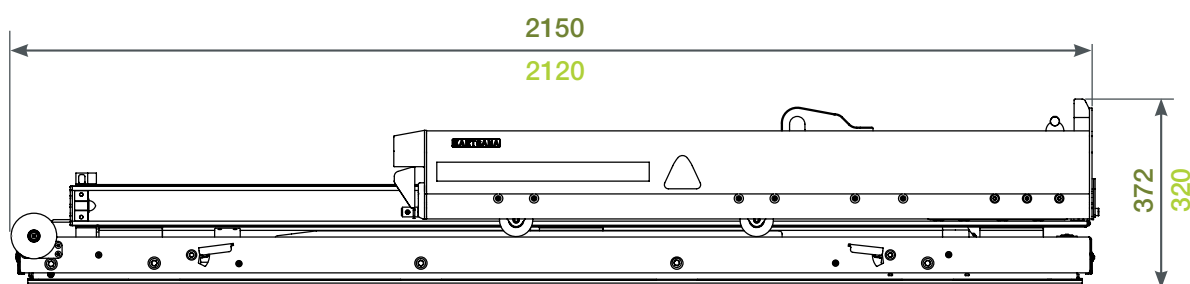

2.2. Mounting the Superbrava (TG-2022) stretcher on the Superbravo rail

The maximum range of the stretcher should only be adjusted **the first time** the stretcher is loaded onto the rail. If moving to another ambulance of a different height, **it will have to be readjusted**.

To set the maximum height of the stretcher's upward trajectory, the stretcher should be positioned in front of the deployed rail on the outside of the ambulance. Position the front wheels half a centimetre above the profile where the small load wheels slide. (see side picture).

- 1** The stretcher must be kept in this position when removed from the rail.
- 2** Then look for the adjustment cam on the top of the stretcher near the shock absorber attachment on the headrest.
- 3** Rotate the cam until the end-of-range light comes on.
- 4** Once in position, tighten the screw 8 Nm to fix the cam in its final position.

Thereafter, the stretcher **shall be raised** to the maximum level.

Check the loading manoeuvre and readjust if necessary.

On uneven terrain or changes in gradient, the stretcher can be raised to its maximum range by pressing the right red and yellow button at the same time.

To load the stretcher onto the Superbravo rail, it must be in the outer position of the ambulance. Pull the red lever to release the block and remove it. Once outside, the stretcher is directed towards the ambulance's protruding profiles and the small wheels of the stretcher are aligned with the rail.

Loading the stretcher on the rail

- 1 With the rail removed, face the stretcher, centring the front wheels with the rail castings.

- 2 Push the stretcher as far as the front wheels will go.

The front wheels must **always be locked** when loading the stretcher onto the rail. Failure to do so could damage important parts of the stretcher.

- 3 Press the left button to raise the legs. Keep pressing it down until it no longer rises.

(*) If the operation is successful, the front axle should remain inside the hooks, and the LED for the stretcher folding on the frame will light up.

- 4 Operate the right rear red handle on the rail to unlock the mechanism and push the stretcher inwards until the end of the range is reached.


Once this process has been completed, the stretcher must be fixed securely to the rail. A check can also be made to ensure that the release lever is in the idle position.

Unloading of the stretcher when on the rail

To unload the stretcher, the procedure is as follows:

- 1 Lift the red central lever of the rail or press and hold the right front red button.

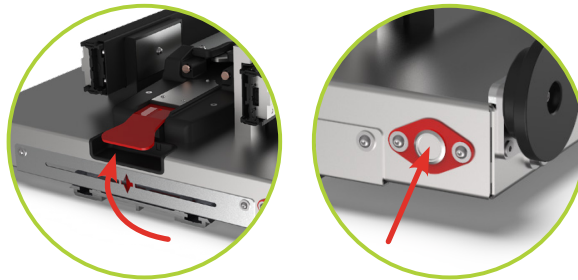- 2 Slide the rail and stretcher until they stop.

- 3 Unfold the legs by pressing the right button on the control unit.

- 4 Pull the stretcher outwards and store the rail.

2.3 Lateral displacement

This functionality is **only available in the R-2022 model**. The Superbravo 2 rail can be moved laterally, which allows the position of the rail-stretcher assembly to be modified with respect to its anchorage point.

To activate the **lateral movement**, simply press any of the side pedals along the rail profile.

Once pressed, the internal system is released and the rail can be moved laterally in any direction. When the pedal is no longer pressed, the system will anchor when it finds any of its **11 fixed positions**, returning to its rest position.

3. MOUNTING THE RAIL ON THE AMBULANCE


3.1 Vehicle configuration

These installation instructions are intended for the attachment of the Kartsana Superbrava (TG-2022) - Superbravo assembly.

The correct use and function of the Superbrava stretcher depends on the correct installation of the fixing system (Superbravo rail) in the ambulance.

The height of the ambulance floor in relation to the ground must not exceed 75 cm.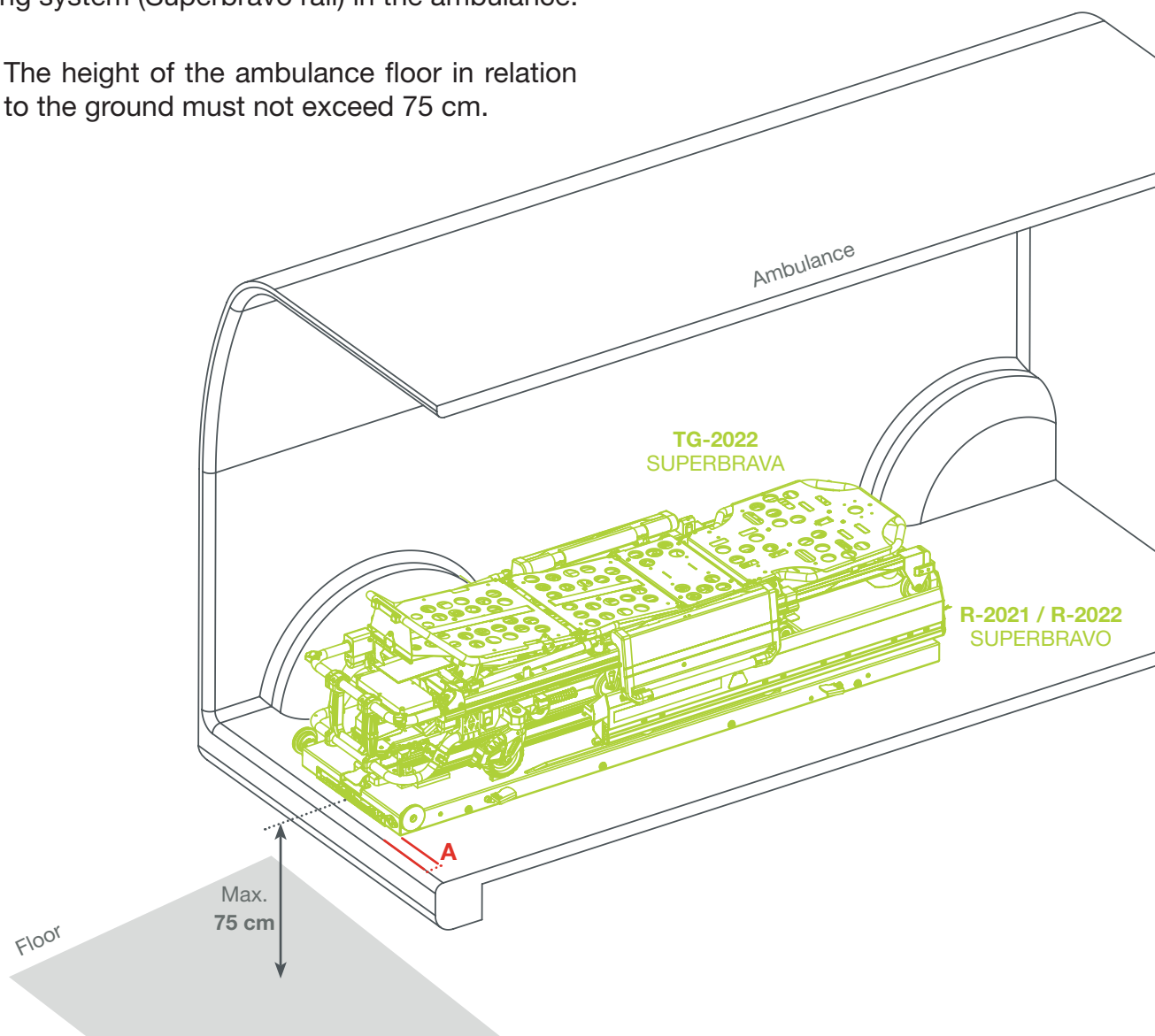

The fixed part of the rail should be fixed at a **distance of approx. 50 mm. (A)** from the rear door of the ambulance (from the rectangular base of the stainless steel).


Depending on the ambulance model and the type of rear **bumper**, the fixing dimensions may vary and/or be adapted.

The maximum distance between the edge of the bumper and the end of the floor attachment **must not exceed 37 cm**. This ensures that the stretcher can be deployed without coming into contact with the bumper.

3.2 Fixing the rail on the ambulance

The Superbravo 2 rail must be attached to the ambulance by means of the lower aluminium profile and 26 DIN 7991 M8-10.1 screws. Simply remove the 4 hexagonal screws from the ends and move the upper plate to release the profile.




The situation of the drill holes is as follows: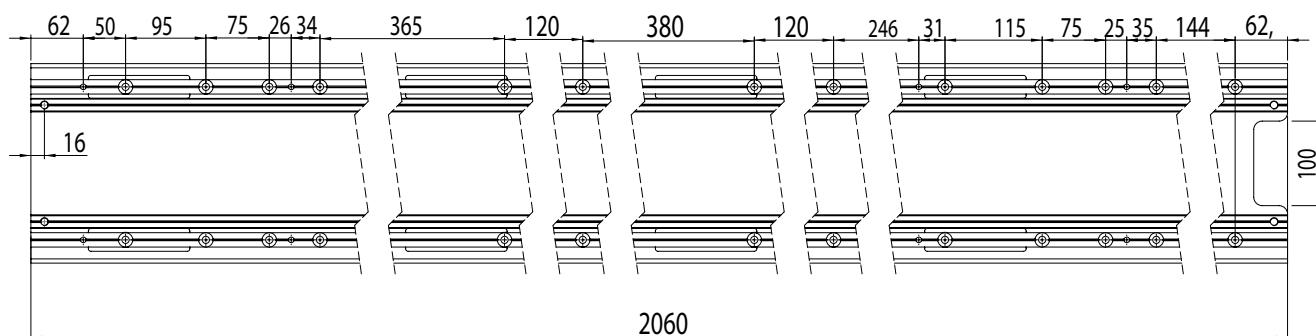

(*) If necessary, there is a template 97-0635-001 for drilling holes in the ambulance floor. You can talk to our technical service if you have any questions.

Once the plate is fixed, the rail must be installed following these points:

- 1** Place the 'T' fasteners (12-0749) in the corresponding 8 slots of the aluminium profile until they lock into position:
- 2** Push the rail until the mechanism stops. Once this position is reached, tighten the 4 hexagonal screws DIN 933 M10x20, previously removed, to 60Nm.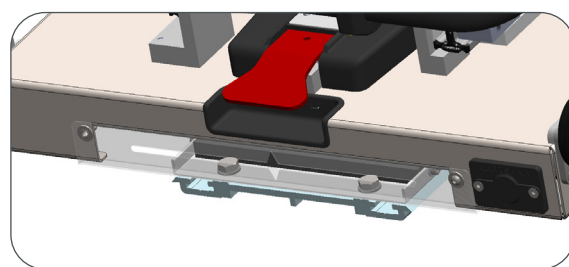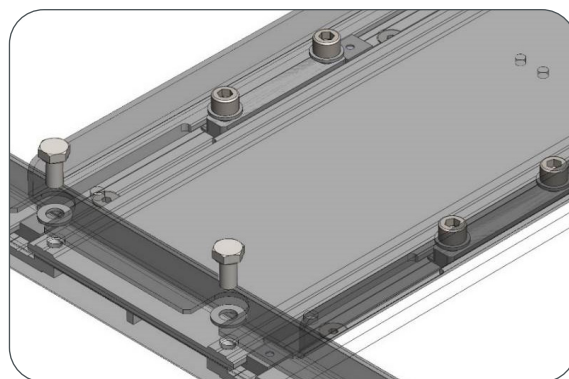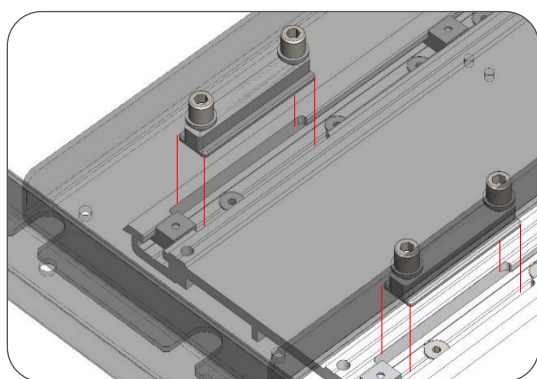

3.3 Electrical connection


The electronics are ready to support a 12V tension.

The **Superbravo rail** must be connected to the vehicle's electrical system with an electrical cable of at least 4 mm² cross-section (pay special attention to the + or - poles). The purpose of this connection is for the **Superbrava stretcher** to charge the batteries when it is inside the ambulance.

It is recommended that this cable be protected on the outside with a plastic sheath to avoid electrical problems (chafing, short circuits, etc.).

The **electrical system has to be secured** by means of a 20A fuse. (*Built-in behind the central rear cover*).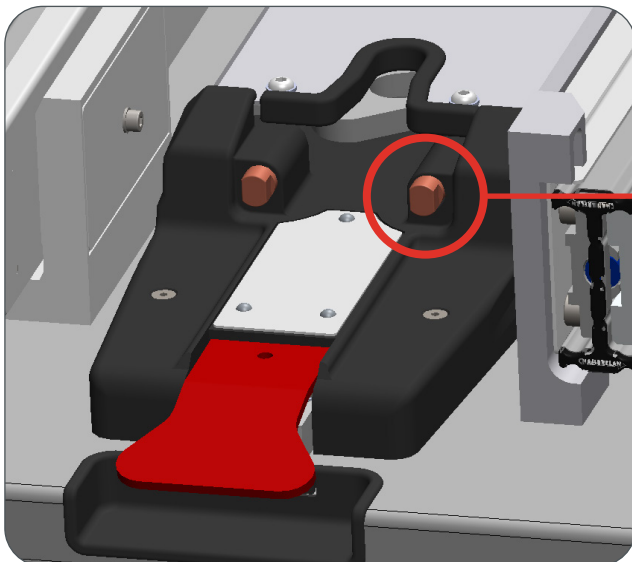

It must be ensured that the **positive pole** is as indicated in the figure, otherwise the stretcher will not be loaded into the vehicle.

4. CARE AND MAINTENANCE OF THE PRODUCT

4.1. Contact information

If you have any questions regarding the use, maintenance or installation of a KARTSANA product, please contact our Technical Service Area (SAT).

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient are established.

Telephone: +34 93 715 86 72
Email: sat@kartsana.com
Address: Narcís Monturiol, 34
08192 Sant Quirze del Vallès,
Barcelona (Spain)

4.2. Environmental conditions

Both for use and storage:

Recommended temperature: from -10°C to +60°C

Relative humidity: 5 – 95%

4.3. Cleaning

To clean the rail, use a cloth, rag or similar to wipe the surface to be cleaned with products that do not damage the surface of the materials. If the surface to be cleaned is already lubricated, re-grease it after cleaning has been completed.

Please use cleaning agents according to the manufacturer's instructions.

The manufacturer shall not be liable for any damage or anomaly caused by a cleaning product that may damage the surface of the stretcher materials.

For reasons of hygiene and preservation of the product components, the manufacturer recommends cleaning after each use.

4.4. Maintenance

4.4.1 Precautionary maintenance

Correct and regular maintenance guarantees the durability of the product.

Creating a maintenance plan, including regular reviews and establishing a responsible person to carry them out, is recommended.

The person in charge must adhere to the following **requirements**:

- Possess technical knowledge related to the product and the maintenance procedures described in these instructions.
- To have the support of qualified, trained and knowledgeable staff who are familiar with the operation of the product.
- Use original or manufacturer-approved components, spare parts and accessories.
- In accordance with the instructions of the European Council Directive 93/42/EEC which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability, records of all product maintenance operations shall be kept.

Check for proper operation **before use**. Prior to service, the following should be checked:

- Functionality.
- Fastenings and hardware.
- Parts that move, such as wheels, belts and mattresses.
- Sensitivity of springs.
- Electrical functions: Full range of motion, lighting, etc.

If the device does not appear to be suitable for correct and safe use, **it should be taken out of service** until the device is repaired or restored to full working order.

Do not modify the structure of the device for any reason, as this may cause serious harm to patients and/or operators.

Lubrication intervals of moving parts:

- 1 – 30 services per month: every 3 months.
- 30 – 50 services per month: every 2 months.
- More than 150 services per month: every month.

All maintenance other than lubrication, tightening of nuts and bolts and ordinary cleaning must be carried out by KARTSANA or an authorised service centre.

4.4.2. Service maintenance

The person entrusted with the product review shall ensure the following basic requirements:

-  Adequate knowledge of the product, its technical/constructional characteristics, control points and final tests, packaging, preservation and handling.
-  Knowledge of all product functions and any possible risks, malfunctions or failures of the product.
-  Possession of all necessary tools to carry out any technical service or minor repairs.
-  Possession (or ability to procure) original manufacturer's or manufacturer's authorised spare parts.
-  Use or support of specialised technical personnel trained by the manufacturer to service the product in question.
-  Keeping a record of all maintenance operations carried out on the product, in accordance with the instructions of the European Council Directive 93/42/EEC which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability.

4.5. Transport and storage

Before transporting the device, ensure that it is properly packaged and secured to prevent damage during transportation.

Keep the original packaging for possible further transport. Any damage caused to the device during transport is not covered by the warranty.



Repairs and replacements of damaged parts are the responsibility of the customer. The device should be stored in a dry place.

When storing, do not place any heavy objects on top of the product. It must not be used as a support for any kind of object.

4.6. Disposal

When the product is no longer in a usable condition, do not throw it away but develop a take-back strategy for recycling by an authorised dealer. The packaging shall be managed as reusable waste. Metals are fully recyclable. Plastic products should be treated as recycling material. Waste management should be as established in each country. If you have any questions, please ask your municipal administration.

5. SPARE PARTS

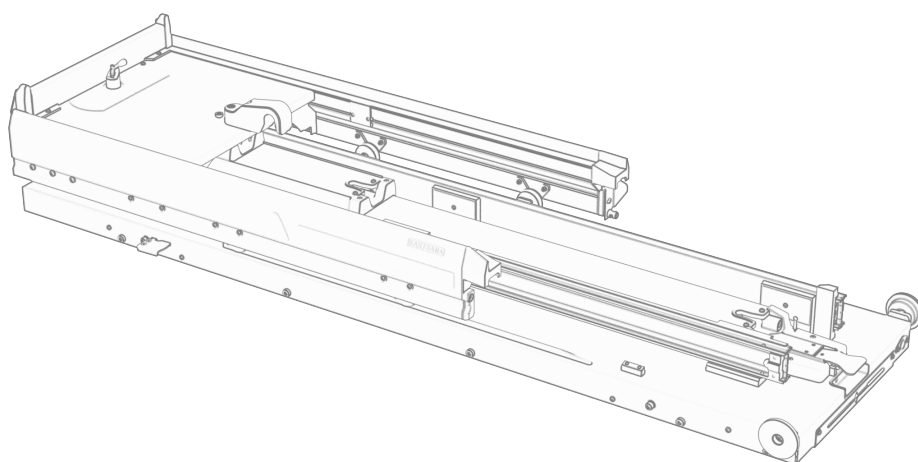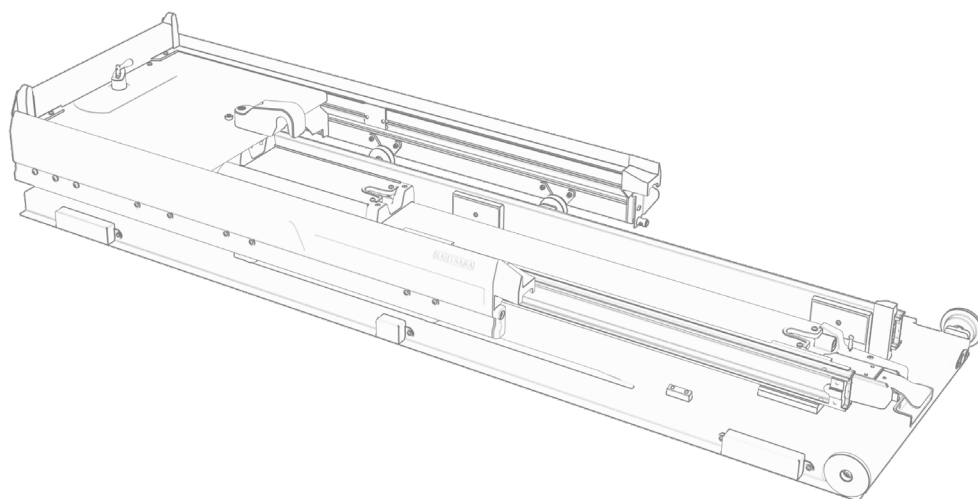To consult the spare parts for this product, you can visit the **following link**:

https://www.kartsana.com/global/Spare_parts_R2022.pdf

If you have any questions, please contact our technical service.

6. SYSTEM ERRORS

Like the stretcher, the rail has possible errors related to various causes. Below are the common errors of the rail system:

TROUBLESHOOTING			
CODE	DESCRIPTION OF THE ERROR	POSSIBLE CAUSE OF THE ERROR	ERROR SOLUTION
1	The stretcher does not receive voltage from the ambulance.	The stretcher-rail plug is not connecting correctly	Check that the two contacts of the stretcher plug and the rail are touching each other and that there is no foreign body between them.
		Ambulance battery is discharged or damaged.	Charge the battery or replace it with a new one, respectively.
		Power cables incorrectly connected either on the ambulance, rail or stretcher.	Check connection of the cables. Recommended order of testing: ambulance, rail and stretcher.
		One of the copper contacts of the stretcher plug has not come out of its housing completely in the plastic part.	Check that the plug contacts move smoothly in and out of the socket.
2	Failure in charging process.	Contact technical support	Contact technical support.

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