
Serie R-800

R-800G / R-800L

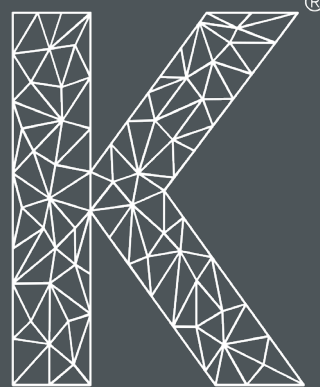KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

Rail mecánico R-800G / R-800L

Instruction manual

Mechanical rail R-800G / R-800L

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Ancrage mécanique R-800G / R-800L

ESPAÑOL

ENGLISH

FRANÇAIS

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
BARCELONA (SPAIN)
TEL. +34 93 715-49-24
info@kartsana.com
www.kartsana.com

Serie R-800

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Modelos R-800G / R-800L

MANUAL DE INSTRUCCIONES

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INSTRUCTIONS MANUAL

R-800-G / R-800-L Models

Read these maintenance instructions before using the product and keep them for future reference.

ÍNDEX

1.1.- FOREWORD	5
1.2.- RESPONSIBILITY AND WARRANTY	5
1.3.- SPECIFICATIONS	5
1.4.- ATTENTION	6
1.5.- SUMMARY OF SAFETY PRECAUTIONS	6
2.- INSTRUCTIONS MANUAL	8
2.1.- DIAGRAM SHOWING RAIL FASTENING TO THE FLOOR	8
2.2.- TECHNICAL CHARACTERISTICS	10
2.2.1.- MODEL R-800-G	10
2.2.2.- MODEL R-800-L	10
2.3.- RAIL OPERATION AND USAGE	11
2.3.1.- FASTENING SYSTEMS:	11
2.3.2.- STRETCHER LOCK ON THE RAIL	11
2.3.3.- UNLOCKING OF THE STRETCHER ON THE RAIL.	15
2.3.4.- WIDTH ADJUSTMENT OF FRONTAL WHEELS	15
3.- DISINFECTION	17
4.- MAINTENANCE	17
5.- EXPLODED VIEW	18
5.1.- MODEL R-800-G	ERROR! NO S'HA DEFINIT EL MARCADOR.
5.2.- MODEL R-800-L	ERROR! NO S'HA DEFINIT EL MARCADOR.

1.- INTRODUCTION

1.1- Foreword

Rails series R-800 have been tested according to standards UNE-EN 1865-5 and UNE-EN 1789.

All information regarding treatment, disinfection and maintenance is indicated, taking into consideration our current experience and know-how.

We reserve the right to change the specifications of the rail to improve the product.

1.2.- Responsibility and warranty

The rail must be checked at the time of its delivery to the rescue organisation. All its functions must be explained in detail. The assistance organisation must take charge of teaching all its employees how to use it correctly.

The warranty is valid for two years following the date indicated on the purchase invoice.

The warranty does not cover defects resulting from incorrect installation, misuse or improper use of the stretcher. All repairs must be made by a technical service authorised by Kartsana or its respective representative.

The manufacture will not be liable for any anomaly caused to the stretcher because of using products other than those provided by Kartsana.

Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.

1.3.- Specifications

	R-800-G	R-800-L
Maximum unassisted load:	220Kg	300Kg
Standards	UNE-EN-1865-5 EN-1789+A1	UNE-EN-1865-5 EN-1789+A1
Total length max/min	2226mm/1978mm	2351/2055mm
Total Width	603mm	622mm
Loading Height	Depends on the used stretcher	Depends on the used stretcher
Recommended operatives required to load / unload an occupied stretcher	1 operative, or 2 if the stretcher were occupied by a larger sized patient	1 operative, or 2 if the stretcher were occupied by a larger sized patient
Compatible stretchers	TG-241 (series) / TG-880S C4	TG-880 (Jupiter series)

1.4.- Attention

Through this sign, information is furnished on important safety measures for the correct use of the stretcher, to prevent accidents. The warnings alert the reader about a situation which, if not avoided, could result in death or serious injury. The cautions alert the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur because of use or misuse.

1.5.- Summary of safety precautions

Read carefully and strictly follow the warnings and cautions listed on these pages. Service only by qualified personnel.

WARNINGS:

- Improper usage of the stretcher can cause injury to the patient or operator. Operate the stretcher only as described in this manual.
- Do not modify the rail or any components of the rail. Modifying the product can cause unpredictable operation resulting in injury to the patient or operator. Modifying the product also voids its warranty.
- This rail must operate with a compatible stretcher.
- Have the vehicle safety rail compatible installed by a certified mechanic, following instructions in this manual. Improper rail installation can cause injury to the patient or operator and/or damage to the stretcher
- Practice changing height positions and loading the stretcher until operation of the product is fully understood. Improper use can cause injury.
- Do not allow untrained assistants to assist in the operation of the stretcher. Untrained technicians/assistants can cause injury to the patient or themselves.
- Do not ride on the base of the rail. Damage to the product could occur, resulting in injury to the patient or operator.
- Grasping the stretcher improperly can cause injury. Keep hands, fingers and feet away from moving parts. To avoid injury, use extreme caution when placing your hands and feet near the base tubes while raising and lowering the stretcher.
- Always use all restraint straps to secure the patient on the stretcher. An unrestrained patient may fall from the stretcher and be injured.
- The R-800-G rail is designed to be compatible with the TG-241 series and TG-880S C4 Kartsana stretchers, and the R-800-L with TG-880 stretchers, and is operator responsibility that these products work together.
- Two operators must be present when the stretcher is occupied.
- Operators must be able to lift the total weight of the patient, stretcher and any items on the stretcher.
- When cleaning, use any appropriate personal safety equipment (goggles, respirator, etc.) to avoid the risk of inhaling contagion.
- Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure the stretchers are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive residue on the surface of the stretchers, possibly causing premature corrosion of critical components.
- Failure to properly clean or dispose of contaminated mattress or other stretcher components will increase the risk of bloodborne pathogens and may cause injury to the patient or operator.

CAUTIONS:

- Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.
- Installation of the safety hook should be done by a certified mechanic familiar with ambulance vehicle construction. Consult the vehicle manufacturer before installing the safety hook and be sure that the installation of the safety hook does not damage or interfere with the brake lines, oxygen lines, fuel lines, fuel tank or electrical wiring of the vehicle.
- Before operating the stretcher, clear any obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment, ensure that the caster wheels are safely set on the ground or damage to the product may occur.
- Ensure that the restraints are not entangled in the base frame when raising and lowering the stretcher.
- Do not steam clean or ultrasonically clean the unit.
- Maximum water temperature should not exceed 80°C.
- Failure to comply with these instructions may invalidate any/all warranties.
- For additional maintenance information, see the preventative maintenance information.
- Improper maintenance can cause injury or damage to the product. Maintain the stretcher as described in this manual. Use only Kartsana approved parts and maintenance procedures. Using unapproved parts and procedures could cause unpredictable operation and/or injury and will void the product warranty
- Failure to use authorized parts, lubricants, etc. could cause damage to the stretcher and will void the warranty of the product.

NOTES:

- Loose items or debris on the patient compartment floor can interfere with the operation of the safety hook and stretcher fastener. Keep the patient compartment floor clear
- This manual should be considered a permanent part of the rail and should remain with the product even if the stretcher is subsequently sold.
- Kartsana continually seeks advancements in product design and quality. Therefore, while this manual contains the most current product information available at the time of printing, there may be minor discrepancies between your stretcher and this manual
- Kartsana recommends that, prior to installation, the certified mechanic plan the placement of the safety hook in the rear of the vehicle.


2.- INSTRUCTIONS MANUAL

The R-800 and the R-800-G anchoring rails have been designed to securely hold the homologated Kartsana stretchers. The R-800-L only for TG-880 Jupiter series.

Make sure that no damage is caused to the areas containing the stretcher mechanisms, to prevent them from malfunctioning.


2.1.- Diagram showing rail fastening to the floor

The R-800 G rail should be fixed to the floor of the ambulance using the holes indicated in the above diagram.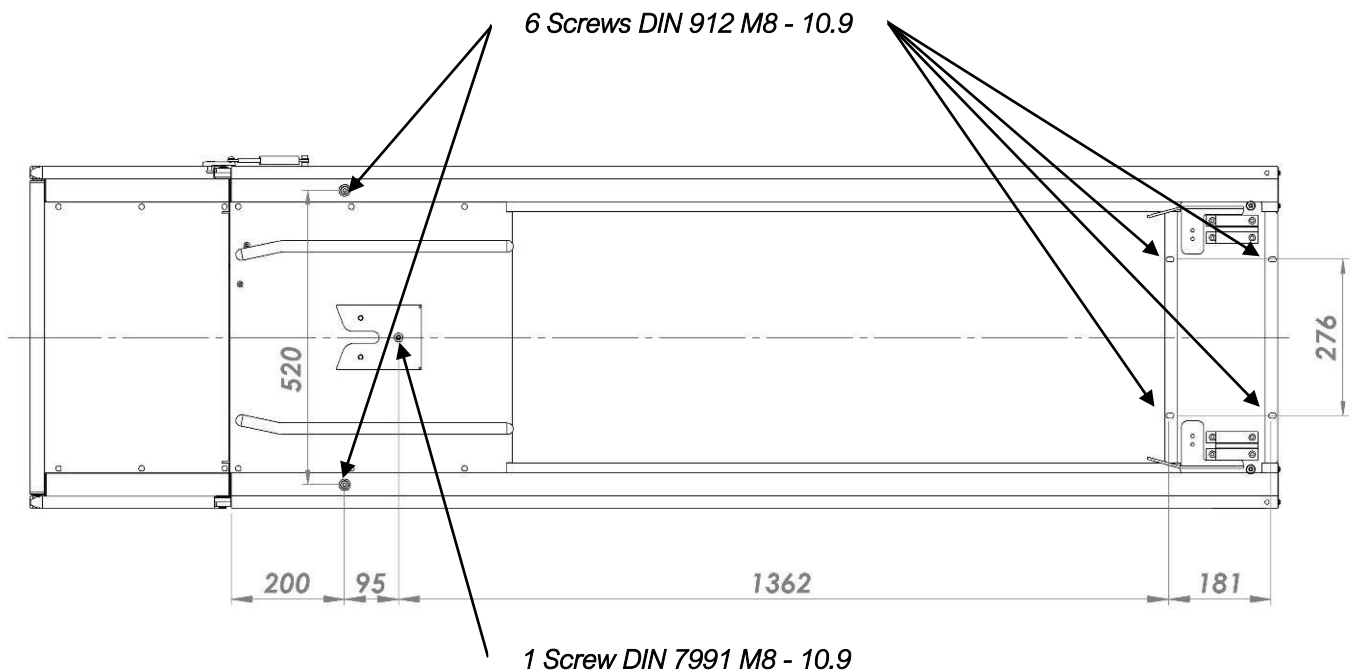

7 FLOOR FASTENING HOLES FOR M8 DIN 912 AND DIN 7991 SCREWS.

The R-800-L rail should be fixed to the floor of the ambulance using the holes indicated in the above diagram.
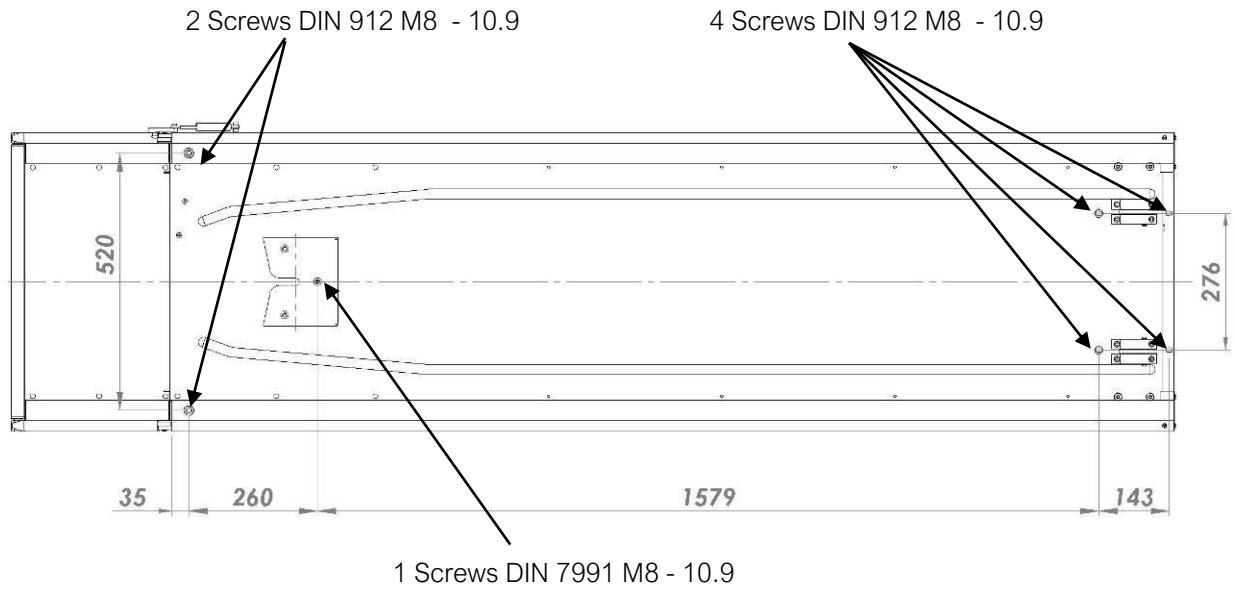


7 FLOOR FASTENING HOLES FOR M8 DIN 912 AND DIN 7991 SCREWS.

2.2.- Technical characteristics

2.2.1.- Model R-800-G

Approximate rail weight 28.5 kg.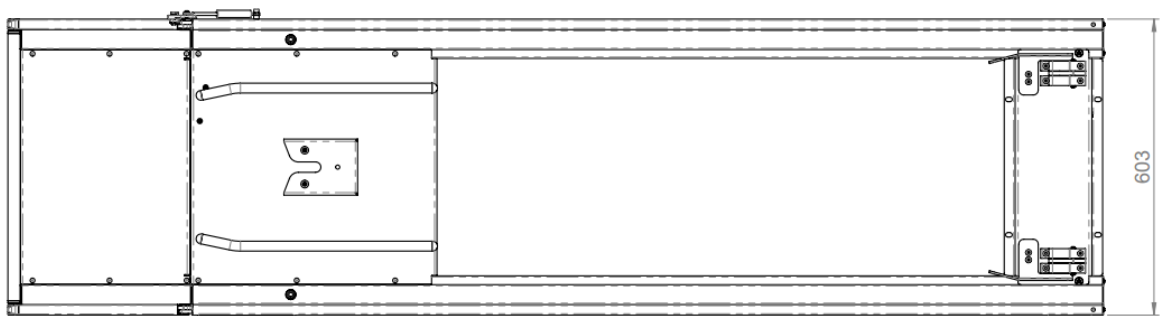
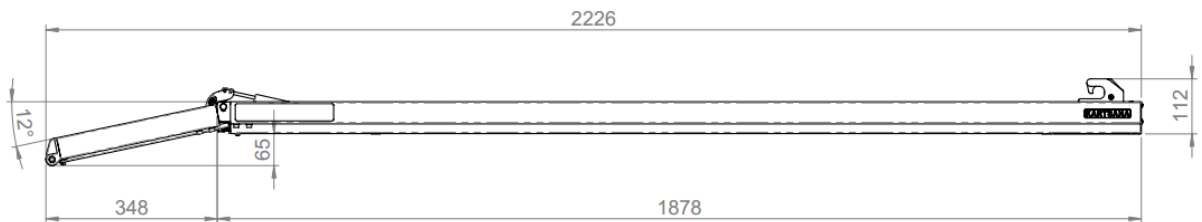


Measurements (in mm) based on the drawing below.

2.2.2.- Model R-800-L

Approximate rail weight 30 kg.

Approximate rail measurements (in mm) based on the drawing below.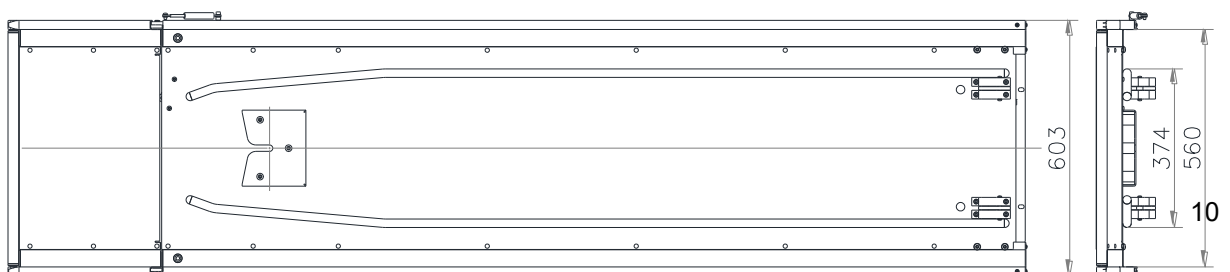
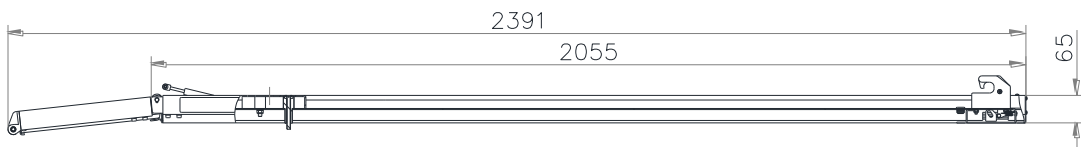


2.3.- Rail operation and usage

To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

2.3.1.- Fastening systems:

The Kartsana R-800 rail is delivered fitted with the following rear attachment system:

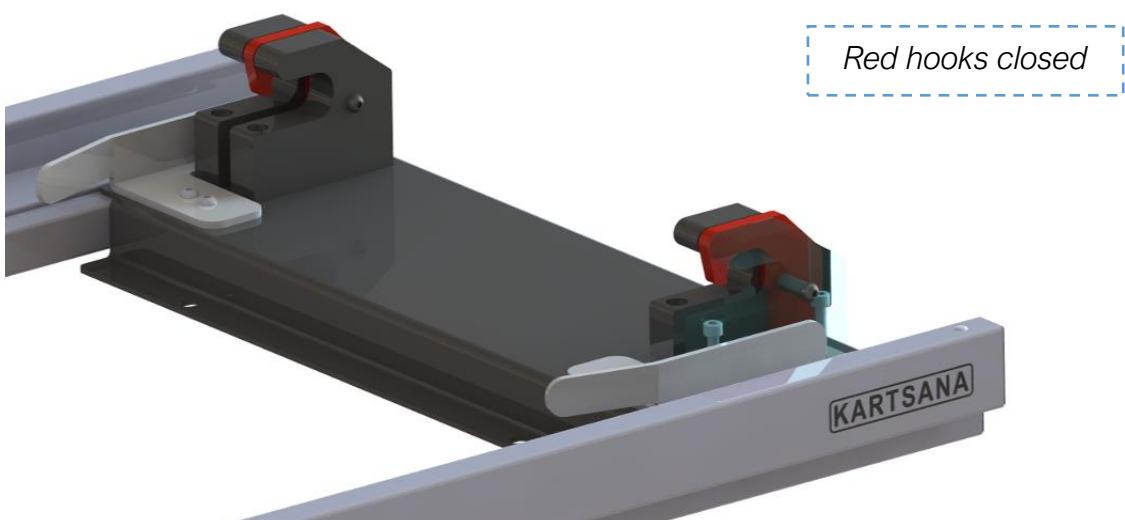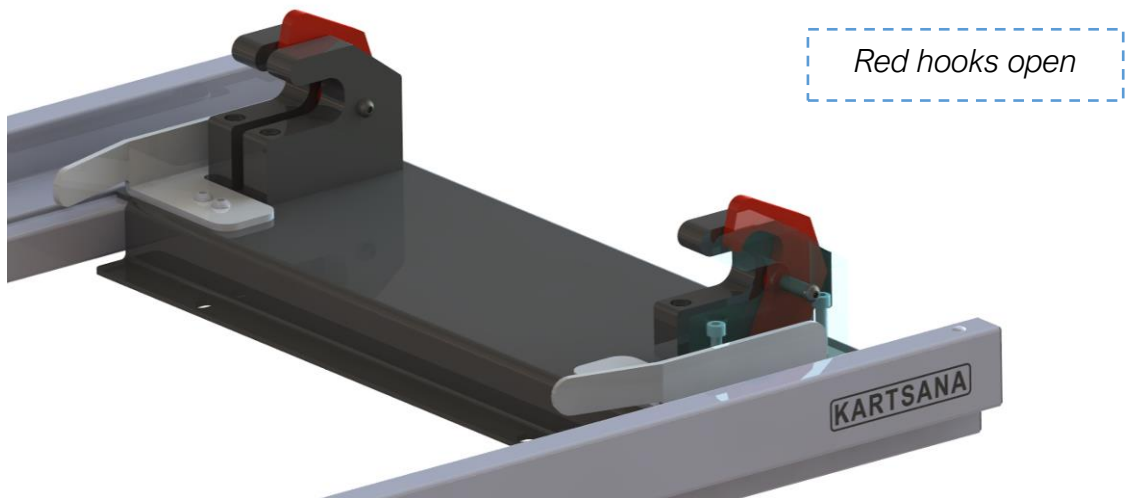
2.3.2.- Stretcher lock on the rail

To fit the stretcher over the rail, follow the steps below:

- Place the rail tray in the horizontal position (open). This arrangement means that the red fastening hooks will be open.


- Position the guide wheels over the rail so that they slide over the aluminium profiles, the wheels are continuously guided to prevent side movements.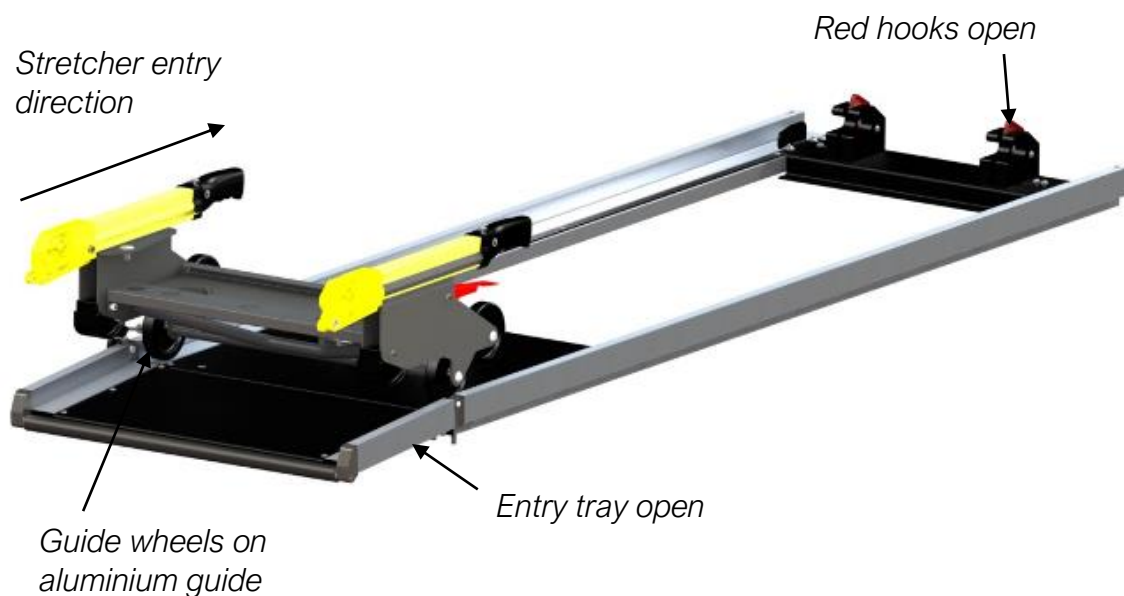


- Unlock the front stretcher legs and insert the stretcher on the rail until the rear leg also stops on the edge of the rail base.
- Unlock the rear leg, insert the stretcher until the guide wheel shaft stops in the two front hooks (the red part of the hooks moves, and the black part is fixed)

- When the wheels arrive at the end of their travel, the rear pin will enter the stainless-steel structure, this prevents vertical movement of the stretcher.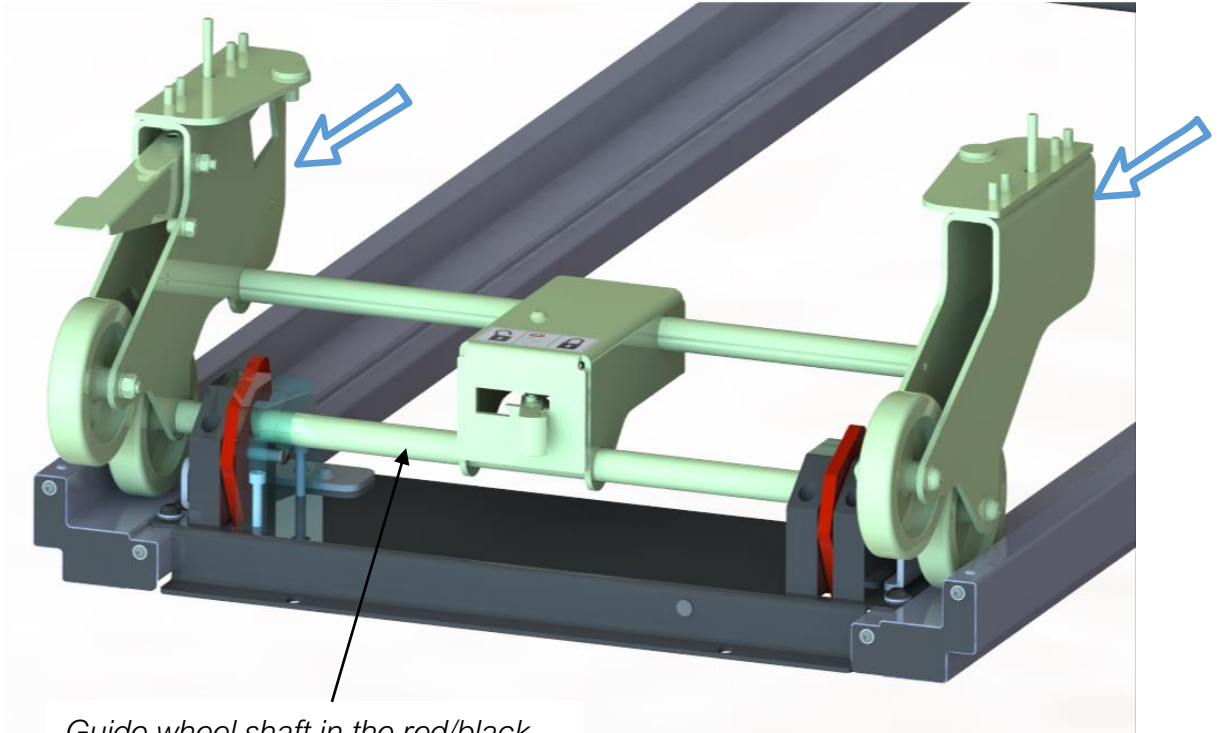


Guide wheel shaft in the red/black hooks (still open).


Rear pin in the stainless-steel plate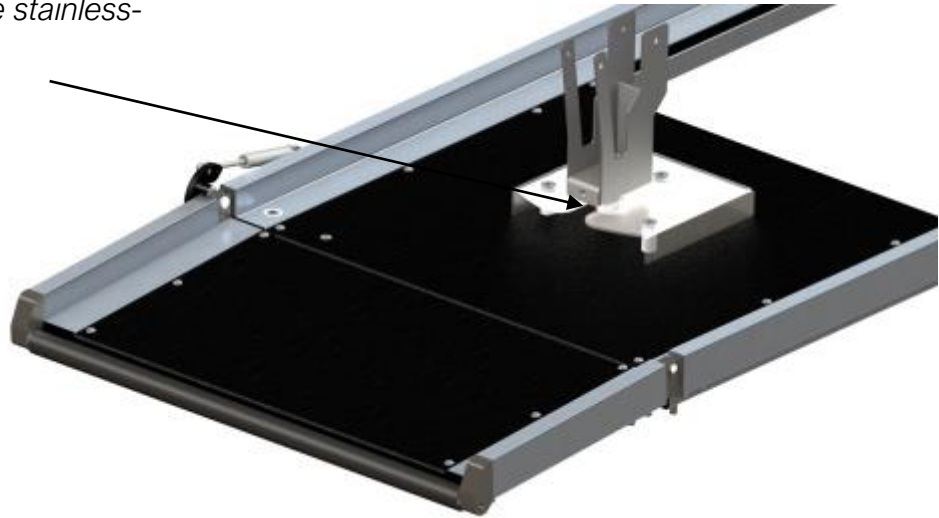

- Finally, to close the red hooks and fasten the rail at the front, lift the rail entry tray.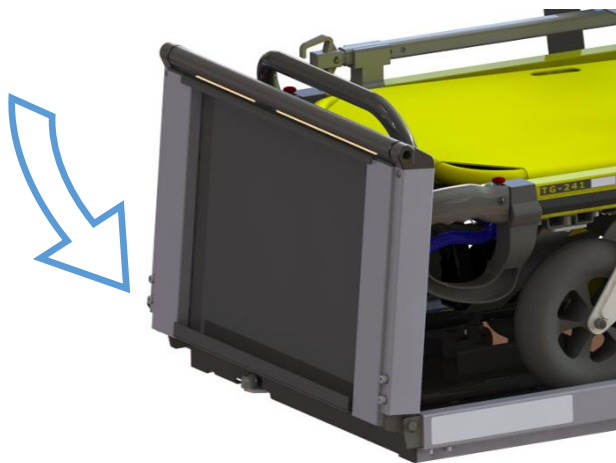

1° Tray lifted

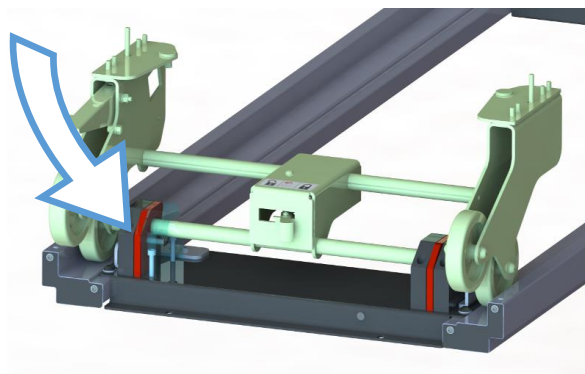2° Red hooks closed and stretcher fastened.

When unlocking the stretcher's legs, hold the stretcher firmly when inserting it in the rail to prevent it from falling.

2.3.3.- Unlocking of the stretcher on the rail.

When the rail needs to be unlocked to remove the stretcher, just open the entry tray and place it in its horizontal position.

- Once the tray is open, the stretcher must be pulled out from the rail to remove the stretcher from it.
- Ensure the rear and front legs are unlocked respectively, before the edge of the rail base reaches the height of each one of these, to prevent the stretcher dropping to the floor with the legs folded.

2.3.4.- Width adjustment of frontal wheels

Rail R-800 provides two lateral platens to adapt the interior width of the stretcher attack wheels to get the minimum lateral looseness between the stretcher and the rail R-800.

These platens are located next to the final fixation hooks, and each one is fixed with two DIN 7380 M6X16 screws.


For the displacement of the platens, you must perform the following points:

Introduce the stretcher to fix into the rail R-800 to the end and ensure it is placed perfectly centred and straight.

Loosen the two screws of each platen with a 4mm Allen key until these can move. (not loosen too much to avoid the drop of the double rectangular nut that we can find inside)

- Move the platens until the required position, that it will be a millimetre away from the attack wheels of the stretcher.

- Tighten the screws of the platen until they remind immovable in the basis plate of rail R-800.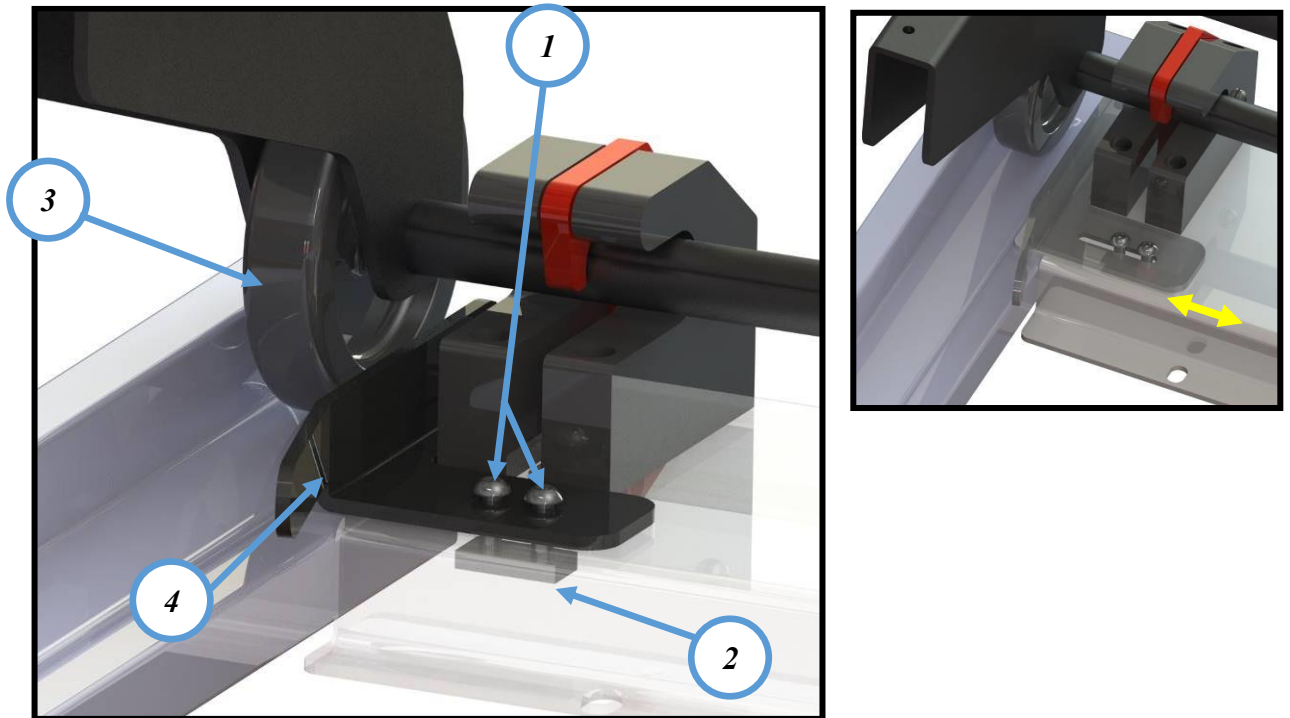

1. Screws DIN 7380 M6X16
2. Double rectangular nut M6
3. Stretcher attack wheel
4. Left lateral adaptation platen

3.- DISINFECTION

To disinfect the rail, use products on the surface to disinfect that will not damage the rail's surface using a sponge cloth rag or similar item. If the surface to be disinfected is greased, re-apply grease once the disinfection has been completed.

All disinfectants must be used in accordance with the manufacturer's instructions.

The manufacturer will not be held responsible for any anomaly or damage caused by using a cleaning product that could damage the surface of the rail materials.

To ensure the hygiene and proper conservation of the product components, the manufacturer recommends that they be disinfected after each use.


4.- MAINTENANCE

Check and oil the moving parts and areas subject to friction every two months.

It will also be necessary to check the parts used for immobilization purposes. We

recommend servicing by a specialist from our company every six months during which a general inspection will be carried out and any parts which may have deteriorated or worn out through use can be replaced.


5.- SPARE PARTS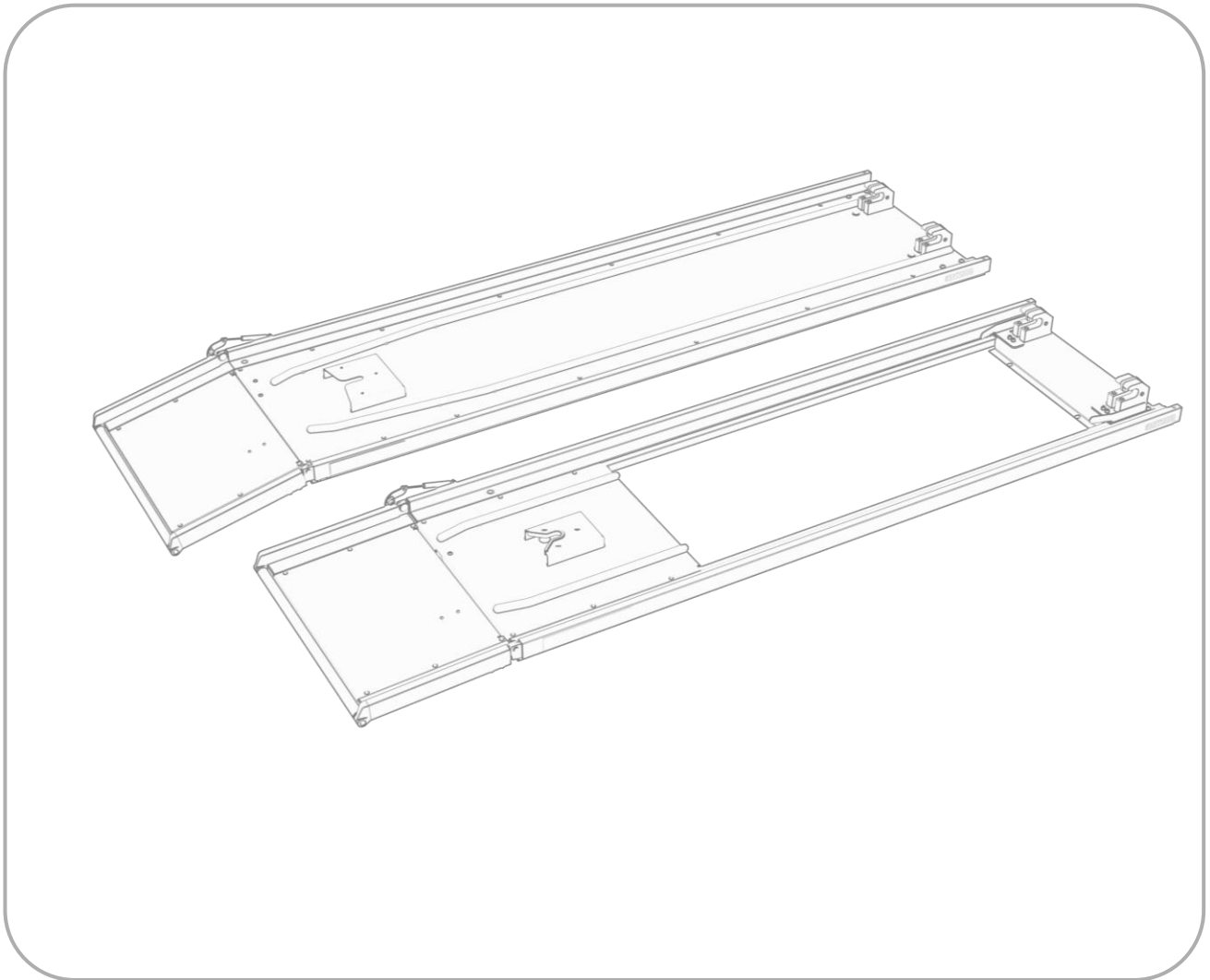

For spare parts for this product, please visit the following link:

https://www.kartsana.com/global/Spare_parts_R-800.pdf

If you have any questions or queries, please contact our technical service.

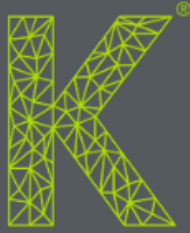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