Serie

CS-200

CAMILLA SILLA

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

Camilla-silla mecánica CS-200

Instruction manual

Mechanical chair-stretcher CS-200

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Chaise-brancard mécanique CS-200

ESPAÑOL

ENGLISH

FRANÇAIS

CS-200

Camilla-silla mecánica

Modelo CS-200

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Mechanical chair-stretcher

Model CS-200

Please read these maintenance instructions carefully before using the product and keep them for future reference.

CONTENTS

1. INTRODUCTION	22
1.1 Preface	22
1.2 Liability and warranty	22
1.3 - Specifications	22
1.4 Precautions	23
1.5 Summary of safety precautions	23
2. INSTRUCTION MANUAL	25
2.1 Technical characteristics of the stretcher	25
2.2 Operation and handling of the stretcher	27
2.2.1. Steps to follow for the proper use of the chair-stretcher	27
2.2.2. Safety railings	29
2.2.3. Adjustable head	30
2.2.4. Drip tube	31
2.2.5. Safety belts	32
3. CARE AND MAINTENANCE	33
3.1 Contact information	33
3.2 Environmental conditions	33
3.3 Cleaning	33
3.4. Maintenance	34
3.4.1 Precautionary maintenance	34
3.4.2. Service maintenance	35
3.5 Transportation and storage	35
3.6 Disposal	35
4. SPARE PARTS	36

1. INTRODUCTION

1.1 Preface

The **CS-200 chair-stretcher** has been specifically designed for patient transport. It is a unique solution, fusing the versatility of a stretcher with the mobility of a chair.

It has been designed to save space in hospitals and to be able to convert the stretcher into a chair when the patient is in the emergency room.

All information on treatment, disinfection and maintenance is based on our experience and current knowledge. We reserve the right to make variations in the technical specifications of the stretcher in order to improve the product.

1.2 Liability and warranty

The stretcher should be checked when handed over to the relief organisation. All of the functions should be explained in detail. The relief organisation is responsible for training all employees in the correct use of the equipment.

The product has a warranty period of 24 months from the date indicated on the purchase invoice. (see *Warranty Conditions* and *Warranty Certificate* enclosed with the product).

The Warranty will not cover breakdown due to incorrect installation, mishandling or misuse of the product. Repairs must be carried out by a service technician authorised by Kartsana or its authorised representative.

The manufacturer shall not be liable for any damage to the stretcher caused by the use of non-original Kartsana products.

Changes or modifications made to the unit that have not been expressly approved by Kartsana may result in errors in the equipment's operation.

1.3 Specifications

CS-200	Stretcher position	Chair position
Maximum workload (unassisted)	300 Kg.	
Diameter of rotating wheels	150 mm	
Total length	1983 mm	1350 mm
Total width	608 mm	
Max height	1028 mm	1404 mm
Loading height	665 mm	-
Weight	41,4 Kg.	

Recommended operators for loading/unloading an occupied stretcher	1 operator or 2 operators in the event that the stretcher is used by a very large person
Compatible restraining systems	R-800 CS Rail

*Stretcher not suitable for bariatric or paediatric transport. Maximum weight is recommended not to exceed 300 Kg.

1.4 Precautions

Throughout this manual, there are 3 types of symbols. As a **warning symbol**, we have ISO 7010-W001. The **prohibition** is indicated by ISO 7010-P001 and the **obligation** by ISO 7010-M001.

Using these symbols, we provide information on important safety measures for the correct use of the stretcher and to avoid possible accidents.

Warnings alert the reader to situations that, if not avoided, may lead to death or serious injury. Furthermore, prohibitions indicate actions that must not be carried out under any circumstance, and obligations indicate measures and/or actions that must be taken to ensure the proper use of the product and to maintain safe functionality.

1.5 Summary of safety precautions

Read and follow the warnings and cautions on these pages carefully. Maintenance and repairs may only be carried out by qualified personnel.

- Improper use of the stretcher may cause injury to the patient or the handler. Use the stretcher only as described in this manual.
- Do not modify any of its components. Modification of the device may result in an unpredictable operation that could cause injury to the patient or operator. Modification of the device voids the warranty.
- Practice changing the rail stretcher entry and exit positions until you are familiar with the product. Improper use can lead to injury.
- Do not allow untrained assistants to help you operate the rail. Inadequately trained technicians and assistants can cause injury to the patient and to themselves.
- Incorrect fastening of the stretcher may result in injury. Avoid putting hands, fingers or feet on any moving parts of the stretcher. To avoid injury, when raising and lowering the stretcher, be very careful when placing hands and feet near the moving parts (inlet tray).
- Always use all the restraining belts to secure the patient on the stretcher. If not properly restrained, the patient could fall off the stretcher and injure him or herself.
- If the stretcher is occupied by a person of large size, we recommend two operators to handle the stretcher and secure the patient.
- Operators must be able to lift the full weight of the patient, stretcher and additional stretcher elements.
- When cleaning, use appropriate personal protective equipment (protective goggles or respirators) to avoid the risk of inhaling infectious organisms.

- Some cleaning products are corrosive and can damage the product if used incorrectly. Measures should be taken to ensure that the stretchers and rails are cleaned with clean water and thoroughly dried after cleaning. Failure to properly rinse and dry the units may leave a corrosive residue on the surface of the units, which may cause premature corrosion of critical components.

- Failure to properly clean or improperly dispose of a contaminated mattress or other components increases the risk of exposure to blood-borne pathogens and could result in injury to the patient or operator.

PRECAUTIONS

- Changes or modifications to the unit not expressly approved by Kartsana may void the user's authority to operate the system.

- Before putting the stretcher or the rail into operation, remove all obstacles that may interfere and cause injury to the operator or patient.

- When unloading the stretcher from the patient compartment (from the rail), make sure that the wheels of the stretcher are placed securely on the floor, otherwise the product may be damaged.

- Ensure that the belts do not become entangled in the rail when loading and unloading the stretcher.

- Do not use steam or ultrasonic cleaners to clean the unit.

- The maximum temperature of the water must not exceed 80°C.

- Failure to comply with these instructions may result in some or all of the warranties becoming void.

- A preventive maintenance programme should be established for all Kartsana equipment. Depending on the product's frequency of use, preventive maintenance may need to be carried out more frequently. See section 3.4 of this document.

- For further information on maintenance, please refer to our technical department.

- Improper maintenance may cause injury or damage to the product. The stretcher should be maintained as described in this manual. Follow the maintenance procedures and use only Kartsana approved parts. The use of unapproved parts or procedures may result in an unpredictable operation and may cause injury and void the warranty.

- Failure to use authorised spare parts, lubricants, etc. may damage the stretcher and void the product warranty.

NOTES

- Loose elements or dirt accumulation on the floor of the patient compartment of the ambulance can interfere with the operation of the rail and the stretcher. Keep the floor of the patient compartment clean.

- This manual should be considered as part of the product and as such should be kept with the product even if it is sold.

- Kartsana works constantly to improve the design and quality of its products. Therefore, although this manual contains the most current information available at the time of printing, there may be minor discrepancies between the stretcher and this manual. If you have any questions, please contact Kartsana's customer service or technical service area.

- Kartsana recommends that, prior to installation, a certified mechanic should plan the placement of the mounting rail on the inside of the emergency vehicle.

2. INSTRUCTION MANUAL

2.1. Technical characteristics of the chair-stretcher (CS-200)

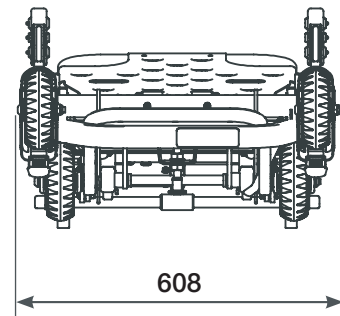
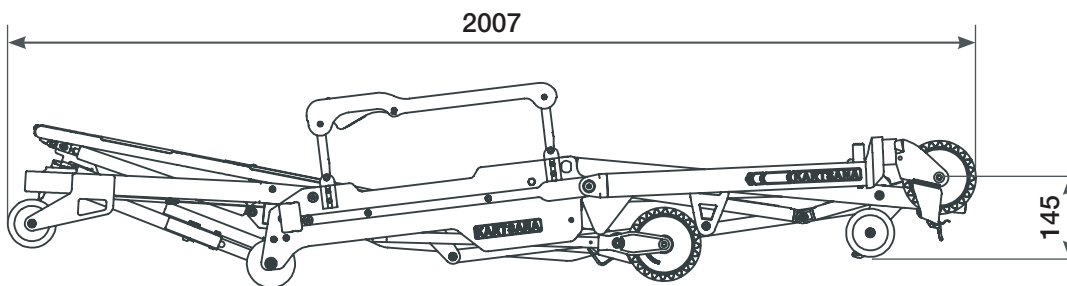
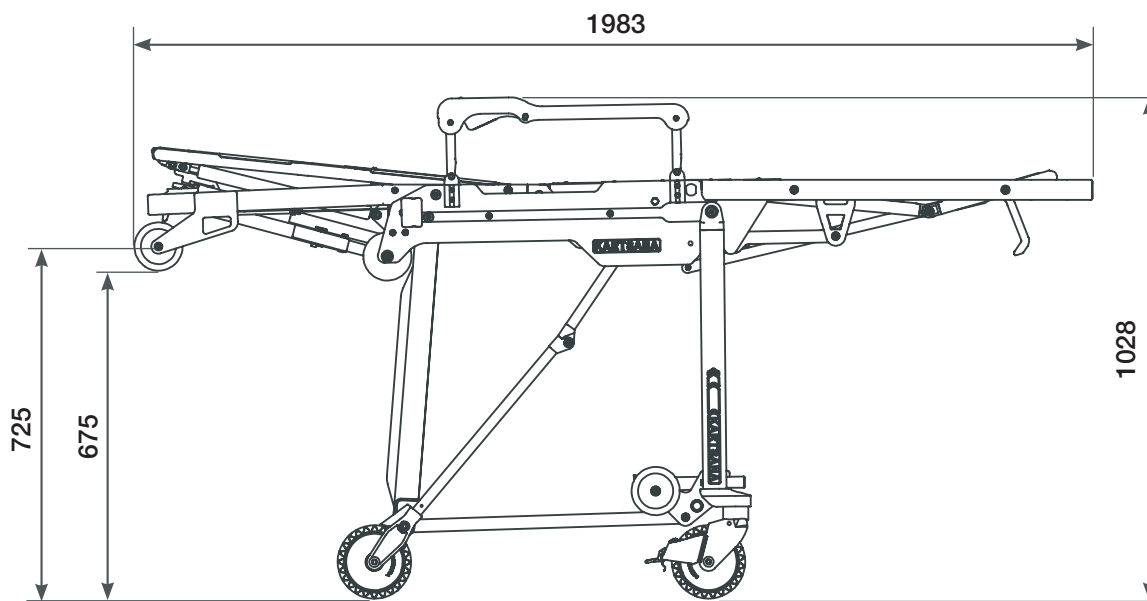
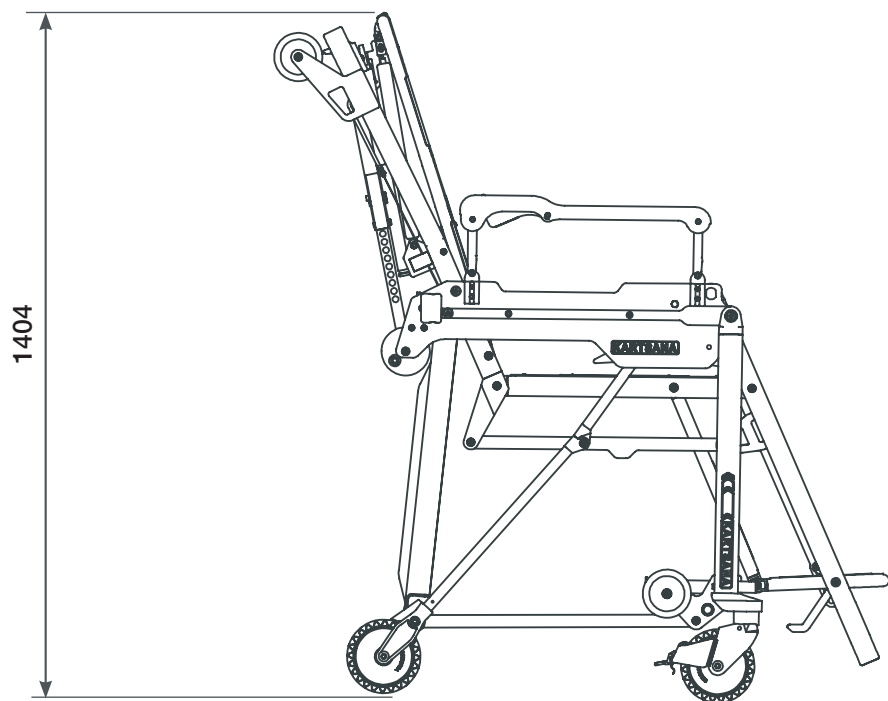
To minimise the risk of injury to the patient and attendants in the event of an accident, it is advisable to avoid sharp edges and protruding surfaces inside the ambulance, paying special attention to the areas near the stretcher.

Do not damage the areas where the stretcher mechanisms are located in order to avoid malfunctioning of the stretcher mechanisms.

Stretcher weight: 41.4 Kg

Dimensions and measurements in mm

2.2 Operation and handling of the chair-stretcher

To avoid possible injury to limbs and other body parts, do not place them in the path of the moving parts of the stretcher.

Note: The front part of the stretcher corresponds to the head part of the stretcher.

2.2.1. Steps to follow for the proper use of the chair-stretcher:

1

Brakes

To facilitate the patient's access to the chair-stretcher, the brakes on the wheels should be engaged to prevent any movement that could hinder the patient's access.

2

Belts

The next step is to secure the patient once he/she is in the chair.

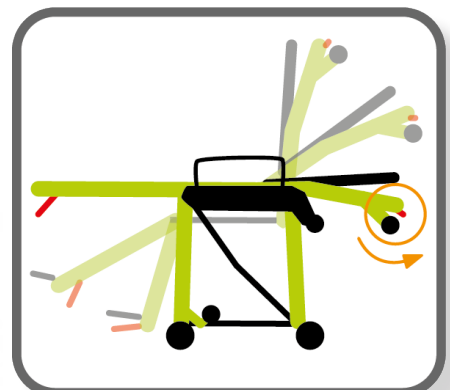
Belts are available from the torso and waist to the feet.

3

Conversion to a stretcher

Once the patient is secured, pull the red trigger at the back of the headrest, converting the chair into a stretcher.


Note that the height of the ambulance floor in relation to the ground **must not exceed 750 mm or be less than 680 mm.**

In order to make folding possible, the front wheels of the stretcher need to be raised.

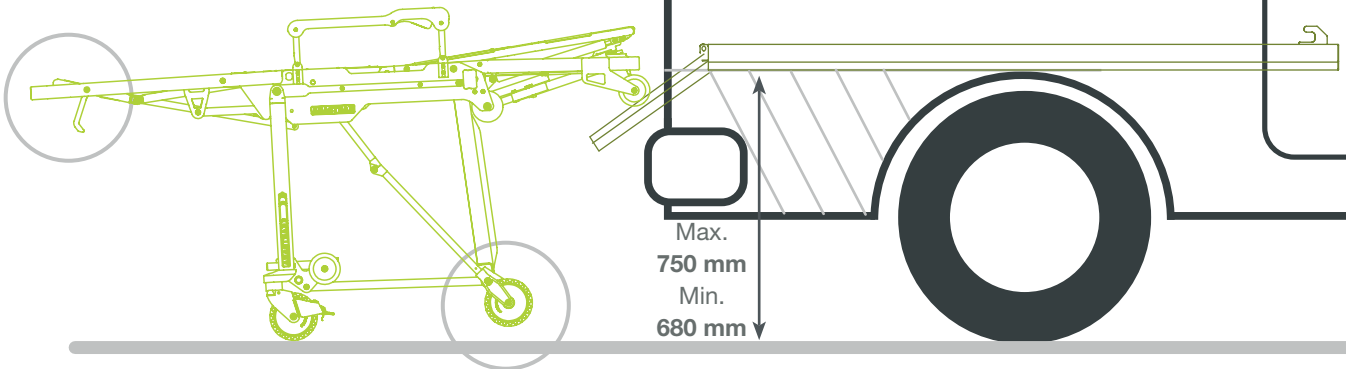CS-200
Chair-stretcher

4

Front lift

In order for the chair-stretcher to function properly on the rail, the front wheel must always be raised so that the legs can fold and unfold completely.

5

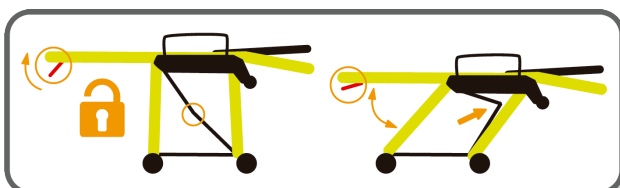
Folding of the legs

In order to fold the legs and fit on the rail, the red trigger on the underside of the leg must be pressed. Using the hips, help to push and load the stretcher into the ambulance.

Keep the stretcher in a horizontal position when loading and unloading to allow the legs to fully extend.

2.2.2. Safety railings

The safety rail has a red handle that locks the rail.

To fold down the railing in the longitudinal direction, just press this handle as shown in the figure below. To re-lock it in the safety position, it must be moved manually to this position and it locks automatically.

2.2.3. Adjustable head

When the patient is in a horizontal position, adjust the headrest to the most suitable position for the patient's situation.

Operate the lever located under the headrest and move it to the desired position.

Then release the lever so that the headrest is locked in the chosen position.

2.2.4. Drip tube

The stretcher is equipped with one removable round IV pole with a diameter of Ø25. This accessory allows the user to position the IV bag at two heights: 481 mm and 855 mm.

Dimensions:
467 mm x 60 mm
(extended is **390 mm**
longer)

Weight:
0.68 kg

Maximum permissible
load: **6 Kg**

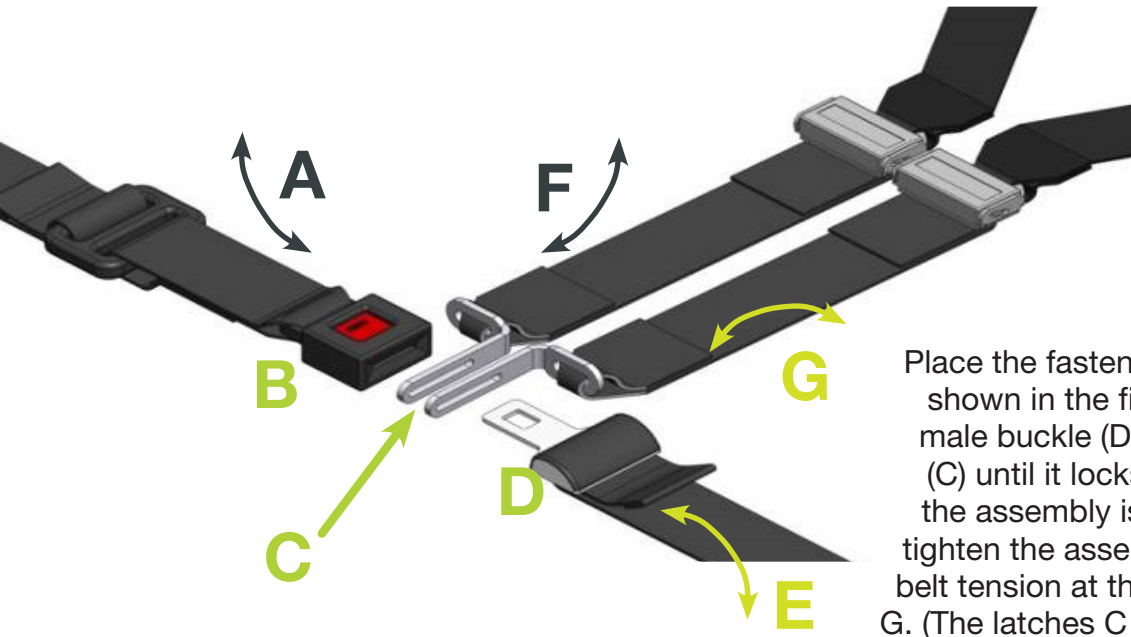
When the IV pole is not in use, the stretcher is equipped with a storage compartment at the bottom of the headrest. Using a lever, it can be easily pulled out and removed.

The stretcher allows the IV pole to be placed on both sides of the product. This is the fitting where it is screwed in.


2.2.5. Safety belts

Head belts:

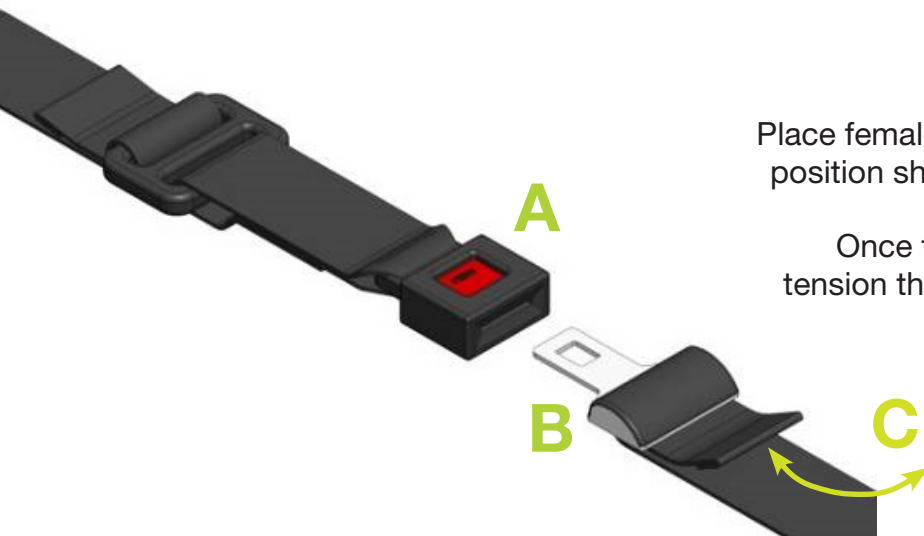Place the fasteners (C) in the position shown in the figure above. Pass the male buckle (D) between the latches (C) until it locks into buckle B. Once the assembly is secured, centre and tighten the assembly by adjusting the belt tension at the ends A, E, F and/or G. (The latches C do not appear on the foot belt).

Lap and leg belts:

Place female buckle A and male buckle B in the position shown in the side figure and insert the latter into the groove of the former. Once the assembly is secured, centre and tension the assembly by adjusting the tension of the belt using end C.

It is recommended that the patient is always secured with safety belts during the time he or she is on the stretcher.

Belt knot

3. CARE AND MAINTENANCE OF THE PRODUCT

3.1 Contact information

If you have any questions regarding the use, maintenance or installation of a KARTSANA product, please contact our Technical Service Area (TSA). Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient are located.

Telephone: +34 93 715 86 72
Email: sat@kartsana.com
Address: Narcís Monturiol, 34
08192 Sant Quirze del Vallès,
Barcelona (Spain)

3.2 Environmental conditions

Both for use and storage:

Recommended temperature: from -10°C to +60°C

Relative humidity: 5 – 95%

3.3 Cleaning

To clean the stretcher, use a cloth, rag or similar to wipe the surface to be cleaned with products that do not damage the surface of the materials. If the surface to be cleaned is already lubricated, re-grease it after cleaning has been completed.

Please use cleaning agents according to the manufacturer's instructions.

The manufacturer shall not be liable for any damage or anomaly caused by a cleaning product that may damage the surface of the stretcher materials.

For reasons of hygiene and preservation of the product components, the manufacturer recommends cleaning after each use.

3.4. Maintenance

3.4.1 Precautionary maintenance

Correct and regular maintenance guarantees the durability of the product.

Creating a maintenance plan, including regular reviews and establishing a responsible person to carry them out, is recommended.

The person in charge must adhere to the following **requirements**:

- Possess technical knowledge related to the product and the maintenance procedures described in these instructions.
- To have the support of qualified, trained and knowledgeable staff who are familiar with the operation of the product.
- Use original or manufacturer-approved components, spare parts and accessories.
- In accordance with the instructions of EU Regulation 2017/45 which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability, records of all product maintenance operations shall be kept.

Check for proper operation **before use**. Prior to service, the following should be checked:

- Functionality.
- Fastenings and hardware.
- Parts that move, such as wheels, belts and mattresses.
- Sensitivity of springs.

If the device does not appear to be suitable for correct and safe use, **it should be taken out of service** until the device is repaired or restored to full working order.

Do not modify the structure of the device for any reason, as this may cause serious harm to patients and/or operators.

Lubrication intervals of moving parts:

- 1 – 30 services per month: every 3 months.
- 30 – 50 services per month: every 2 months.
- More than 150 services per month: every month.

All maintenance other than lubrication, tightening of nuts and bolts and ordinary cleaning must be carried out by KARTSANA or an authorised service centre.

3.4.2. Service maintenance

The person entrusted with the product review shall ensure the following basic requirements:

- ◇ Adequate knowledge of the product, its technical/constructional characteristics, control points and final tests, packaging, preservation and handling.
- ◇ Knowledge of all product functions and any possible risks, malfunctions or failures of the product.
- ◇ Possession of all necessary tools to carry out any technical service or minor repairs.
- ◇ Possession (or ability to procure) original manufacturer's or manufacturer's authorised spare parts.
- ◇ Use or support of specialised technical personnel trained by the manufacturer to service the product in question.
- ◇ Keeping a record of all maintenance operations carried out on the product, in accordance with the instructions of EU Regulation 2017/45 which establishes the obligation of the purchaser to provide on request the above mentioned after-sales care record for the purpose of product traceability.

3.5 Transportation and storage

Before transporting the device, ensure that it is properly packaged and secured to prevent damage during transportation.

Keep the original packaging for possible further transport. Any damage caused to the device during transport is not covered by the warranty.

Repairs and replacements of damaged parts are the responsibility of the customer. The device should be stored in a dry place.

When storing, do not place any heavy objects on top of the product. It must not be used as a support for any kind of object.

3.6 Disposal

When the product is no longer in a usable condition, do not throw it away but develop a take-back strategy for recycling by an authorised dealer. The packaging shall be managed as reusable waste. Metals are fully recyclable. Plastic products should be treated as recycling material. Waste management should be as established in each country. If you have any questions, please ask your municipal administration.

4. SPARE PARTS

To consult the spare parts for this product, you can visit the **following link**:

https://www.kartsana.com/global/Spare_parts_CS200.pdf

If you have any questions, please contact our technical service.

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