# Serie BRAVO

R-1002C

**KARTSANA**

Life-moving innovation

C/ Narcís Monturiol, 34  
08192 Sant Quirze del Vallès  
Barcelona (España)  
+34 93 715 86 72  
info@kartsana.com  
www.kartsana.com

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Model R-1002C

INSTRUCTIONS MANUAL

Read these maintenance instructions  
Before using the product and keep them  
for future references.

**KARTSANA**

Life-moving innovation

OPERATION AND USAGE OF BRAVO  
2C RAIL WITH DRAWER R-1002C

## CONTENTS

---

1. INTRODUCTION	4
1.1. FOREWORD	4
1.2. RESPONSIBILITY AND WARRANTY	4
1.3. SPECIFICATIONS	4
1.4. ATTENTION	5
1.5. SUMMARY OF SAFETY PRECAUTIONS	5
2. INSTRUCTIONS MANUAL	7
2.1. TECHNICAL CHARACTERISTICS	8
2.2. RAIL OPERATION AND USAGE	8
2.2.1. RED CHARGING TRIGGER	8
2.2.2. UNLOADING LEVER.	10
2.2.3. SIDE MOVEMENT (LOWER PEDAL)	11
2.2.4. FASTENING FOR SCOOP STRETCHER / SPINAL BOARD	12
3. ASSEMBLY OF THE RAIL IN THE AMBULANCE	13
3.1. SETTING THE RAIL IN THE AMBULANCE	13
3.2. ELECTRICAL CONNECTION	15
4. INTERMEDIATE POSITION ( <i>OPTIONAL</i> )	16
5. DISINFECTION	17
6. MAINTENANCE	17
7. BRAVO 2C EXPLODED VIEW	18

**KARTSANA**

Life-moving innovation

OPERATION AND USAGE OF BRAVO  
2C RAIL WITH DRAWER R-1002C

## 1. INTRODUCTION

---

### 1.1. Foreword

The BRAVO 2C rail has been solely designed for perfectly holding the KARTSANA “BRAVA” stretcher model.

The Bravo 2C rail has been tested in conformity with the regulation UNE-EN 1789.

All the information regarding treatment, disinfection and maintenance is indicated, taking into consideration our current experience and know-how.

We reserve the right to change the specifications of the BRAVO 2C rail to improve the products.

### 1.2. Responsibility and warranty

The stretcher must be checked at the time of its delivery to the rescue organisation. All its functions must be explained in detail. The assistance organisation must take charge of teaching all its employees how to use it correctly.

The product has a 24-months warranty from the delivery date to the end user.

The warranty does not cover defects resulting from incorrect installation, misuse, or improper use of the stretcher. All repairs must be made by a technical service authorised by Kartsana or its respective representative.

The manufacture will not be liable for any anomaly caused to the stretcher due to using any products other than original Kartsana products.

### 1.3. Specifications

SPECIFICATION	BRAVO 2C
Maximum unassisted load:	380 Kg
Standards	UNE-EN-1865-5 EN-1789+A1
Total length	2088,5 mm
Total Width	632 mm
Height	451 mm
Recommended operatives required to load / unload an occupied stretcher	1 operative, or 2 if the stretcher were occupied by a larger sized patient.
Compatible stretchers	Power Brava

## 1.4. Attention

Throughout this manual, you will find 3 types of ISO 20417 signs. For **warning**, we have ISO 7010-W001. **Prohibition** is indicated by ISO 7010-P001 and **obligation** by ISO 7010-M001.

With these signs we inform about important safety measures for the correct use of the stretcher and to avoid possible accidents.

Warnings alert the reader to situations which, if not avoided, could lead to death or serious injury. On the other hand, the prohibitions indicate actions that must not be carried out under any circumstances and the obligations, those measures and/or actions that must be carried out to use the product properly and maintain safe functionality.

## 1.5. Summary of safety precautions

Read carefully and strictly follow the warnings and cautions listed on these pages. Service only by qualified personnel.

### WARNINGS:

- Improper usage of the stretcher can cause injury to the patient or operator. Operate the stretcher only as described in this manual.
- Do not modify the rail or any components of the rail. Modifying the product can cause unpredictable operation resulting in injury to the patient or operator. Modifying the product also voids its warranty.
- This rail must operate with a compatible stretcher.
- Have the vehicle safety rail compatible installed by a certified mechanic, following instructions in this manual. Improper rail installation can cause injury to the patient or operator and/or damage to the stretcher
- Practice changing height positions and loading the stretcher until operation of the product is fully understood. Improper use can cause injury.
- Do not allow untrained assistants to assist in the operation of the stretcher. Untrained technicians/assistants can cause injury to the patient or themselves.
- Do not ride on the base of the rail. Damage to the product could occur, resulting in injury to the patient or operator.
- Grasping the stretcher improperly can cause injury. Keep hands, fingers, and feet away from moving parts. To avoid injury, use extreme caution when placing your hands and feet near the base tubes while raising and lowering the stretcher.
- Always use all restraint straps to secure the patient on the stretcher. An unrestrained patient may fall from the stretcher and be injured.
- The Bravo 2C rail is designed to be compatible with the Power Brava Kartsana stretchers and is operator responsibility that these products work together.
- Two operators must be present when the stretcher is occupied.
- Operators must be able to lift the total weight of the patient, stretcher, and any items on the stretcher.
- When cleaning, use any appropriate personal safety equipment (goggles, respirator, etc.) to avoid the risk of inhaling contagion.
  - Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure the stretchers are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive

# KARTSANA

Life-moving innovation

## OPERATION AND USAGE OF BRAVO 2C RAIL WITH DRAWER R-1002C

residue on the surface of the stretchers, possibly causing premature corrosion of critical components.

- Failure to properly clean or dispose of contaminated mattress or other stretcher components will increase the risk of bloodborne pathogens and may cause injury to the patient or operator.

### CAUTIONS:

- Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.
- Installation of the safety hook should be done by a certified mechanic familiar with ambulance vehicle construction. Consult the vehicle manufacturer before installing the safety hook and be sure that the installation of the safety hook does not damage or interfere with the brake lines, oxygen lines, fuel lines, fuel tank or electrical wiring of the vehicle.
- Before operating the stretcher, clear any obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment, ensure that the caster wheels are safely set on the ground or damage to the product may occur.
- Ensure that the restraints are not entangled in the base frame when raising and lowering the stretcher.
- Do not steam clean or ultrasonically clean the unit.
- Maximum water temperature should not exceed 80°C.
- Failure to comply with these instructions may invalidate any/all warranties.
- For additional maintenance information, see the preventative maintenance information.
- Improper maintenance can cause injury or damage to the product. Maintain the stretcher as described in this manual. Use only Kartsana approved parts and maintenance procedures. Using unapproved parts and procedures could cause unpredictable operation and/or injury and will void the product warranty
- Failure to use authorized parts, lubricants, etc. could cause damage to the stretcher and will void the warranty of the product.

### NOTES:

- Loose items or debris on the patient compartment floor can interfere with the operation of the safety hook and stretcher fastener. Keep the patient compartment floor clear
- This manual should be considered a permanent part of the rail and should remain with the product even if the stretcher is subsequently sold.
- Kartsana continually seeks advancements in product design and quality. Therefore, while this manual contains the most current product information available at the time of printing, there may be minor discrepancies between your stretcher and this manual
- Kartsana recommends that, prior to installation, the certified mechanic plan the placement of the safety hook in the rear of the vehicle.

## 2. INSTRUCTIONS MANUAL

---

Only the BRAVA stretcher may be put on the BRAVO 2C rail. (BASED ON THE UNE-EN 1789 STANDARD).

To reduce the risk of injuries to the patient and accompanying persons in the event of an accident, it is advisable to avoid sharp edges and projecting parts inside the ambulance, paying special attention to the areas nearest the BRAVO 2C rail.

Make sure that no damage is caused to the areas containing the stretcher mechanisms, to prevent them from malfunctioning.

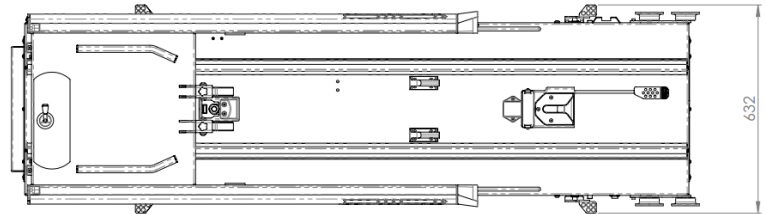
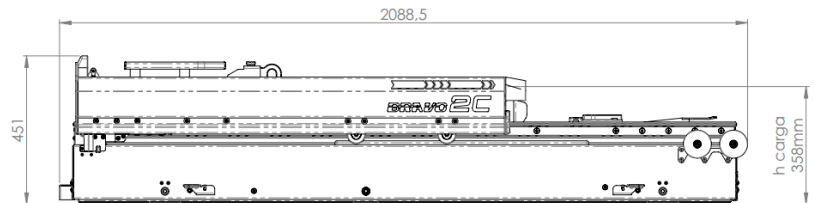
### 2.1. Technical characteristics

Approximate rail weight 120 Kg

Maximum load permitted: 300 Kg

Approximate measurements of the stretcher (in mm.) based on the drawing below.


### 2.2. Rail operation and usage

To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

#### 2.2.1. Red charging trigger → Stretcher lock on the rail

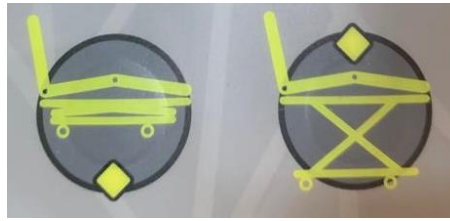1. Place the stretcher in front of the rail.
2. Place the small wheels of the stretcher on the rail cross section. This should be in the exterior position. Position the stretcher onto the rail so that the small wheels of the stretcher are guided by the rail, until the limit is reached at the edge of the cross section.
3. Press the left button to raise the stretcher legs until the end of its run. (If this action has been carried out correctly the front axis should be within the inner part of the hooks.


*Leg folding and unfolding buttons*

4. Press the red trigger “B” at the front right of the rail to unlock the mechanism and to push the stretcher towards the inside until the limit has been reached.

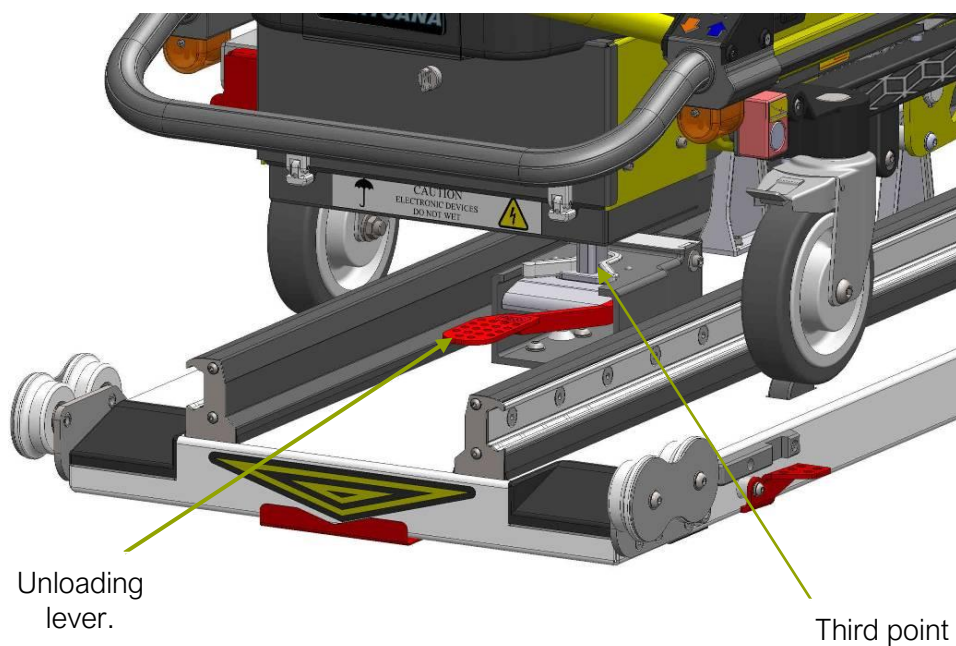To be sure that the whole stretcher rail is correctly placed make sure that the unload lever is in the horizontal position. This is the 3rd point in the inside of its compartment.

**IMPORTANT:** do not disconnect the supporting battery from the ambulance when it isn't doing any service, to allow the correct charge of the stretcher batteries between services.


### 2.2.2. Unloading lever

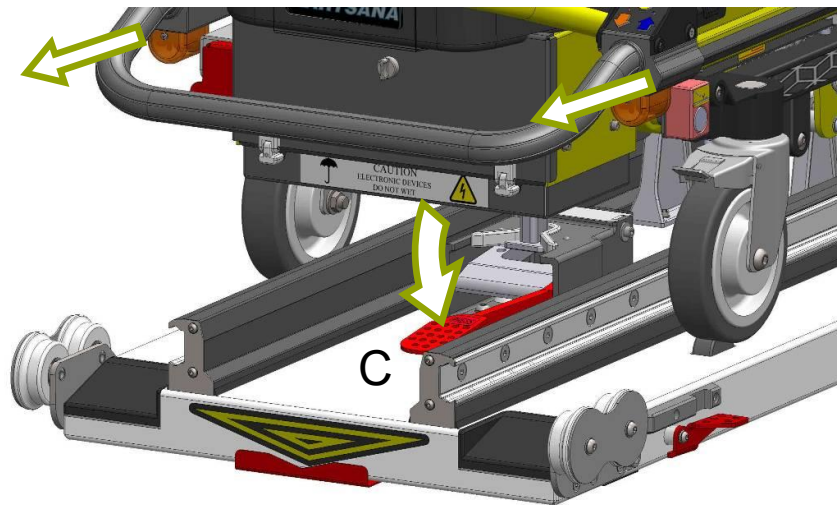Push down the unloading lever "C" located between the guides at the front of the rail as shown in the figure below. While maintaining the lever in this position, pull the stretcher towards the outside of the ambulance until the mechanical limit has been reached. Once the rail is fully extended, we will be able to open out the stretcher legs by pressing the right button (leg unfolding). Once the electric stretcher has been raised above the rail level, we can withdraw it.

To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

Note: The rear part of the rail corresponds to the part where the rail start lever is located.


**IMPORTANT:**


### 2.2.3. Side movement (lower pedal)

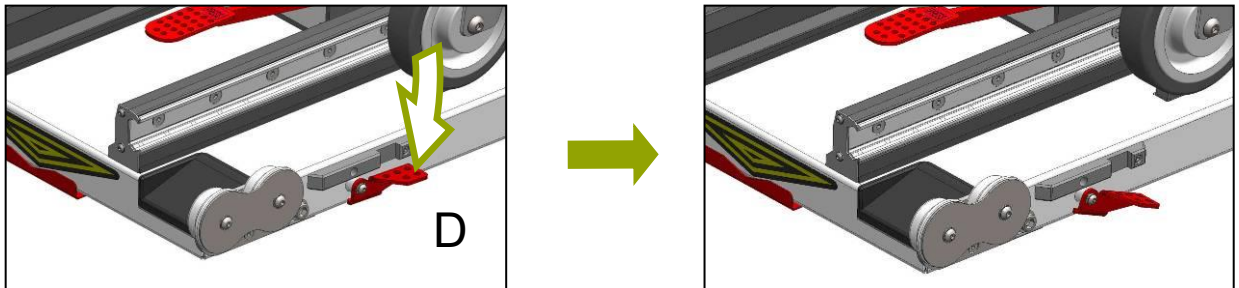Press one of the four red “D” levers and move the upper trolley sideways until the desired position is reached.

Then, release the red lever so that it locks in the chosen position.


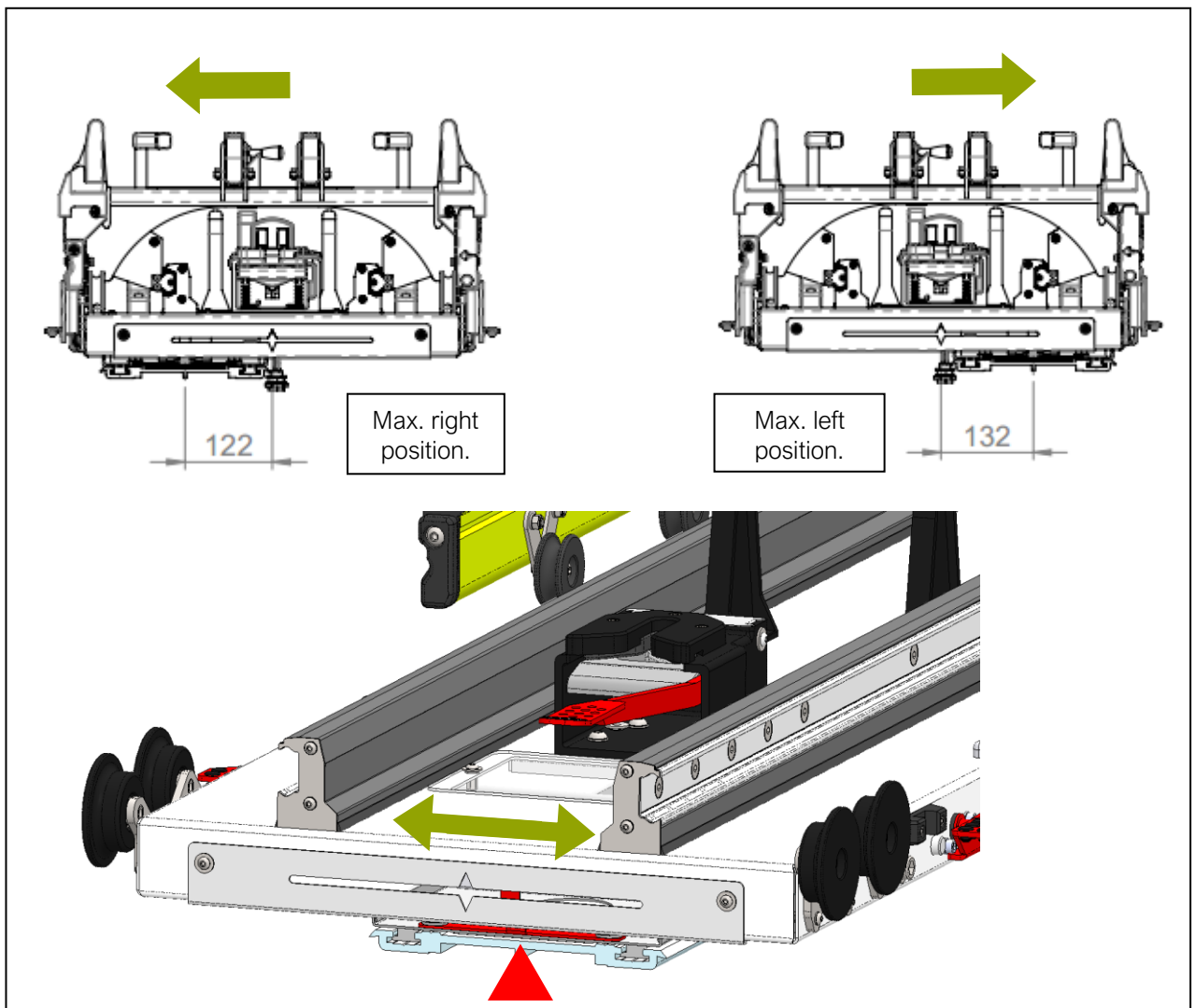
The maximum sideways movement possible is 254 mm, 122 mm to the left side and 132 mm to the right side if the base plates are secured in the centre of the rail. The red part indicates the position of the base related to the centre of the rail.


### 2.2.4. Fastening for scoop stretcher / spinal board

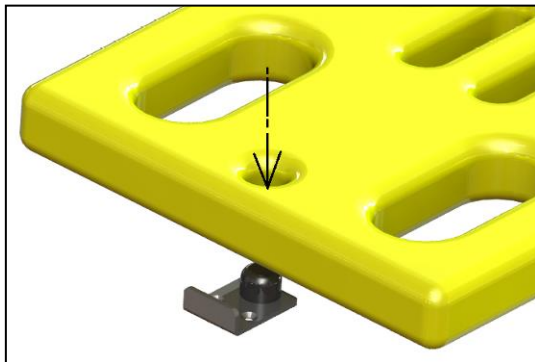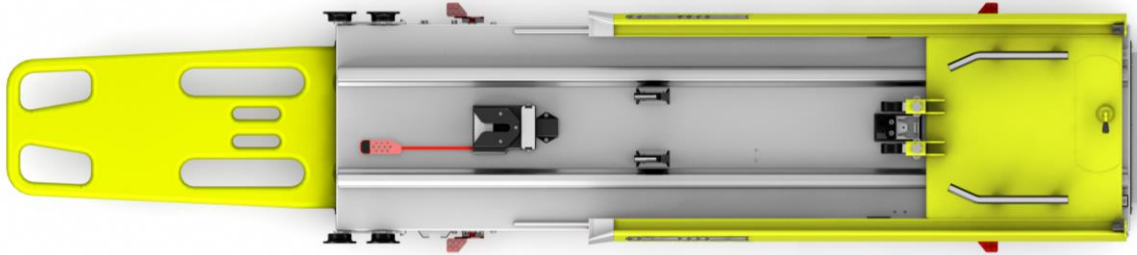The rail **Bravo 2C** has an accommodation for a scoop stretcher or a spinal board inside.


#### Spinal board:

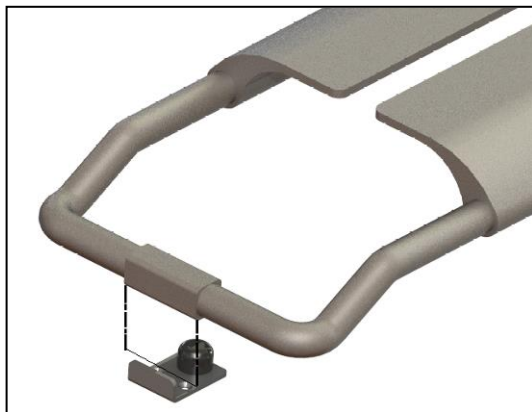You must put the spinal board under/inside the rail and put inside the round part you can find in the front part.

#### Scoop stretcher:

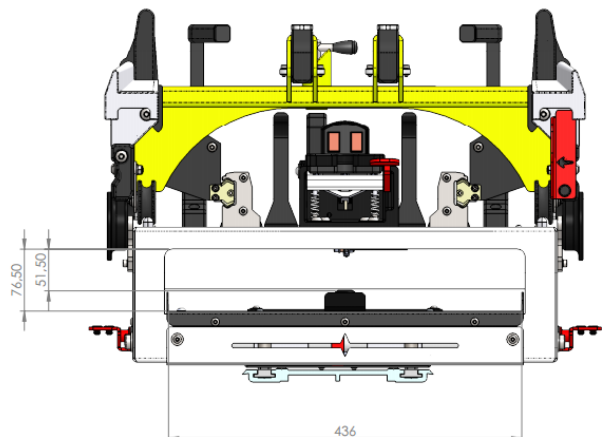To place the stretcher blades, we must fit the edge of the scoop stretcher itself in the part fixed on the rail.


### 3. ASSEMBLY OF THE RAIL IN THE AMBULANCE


#### 3.1. Setting the rail in the ambulance

The Bravo 2C rail must be secured to the ambulance using the aluminum profile 03-1165 and 22 screws DIN 7991 M8 – 10.1.

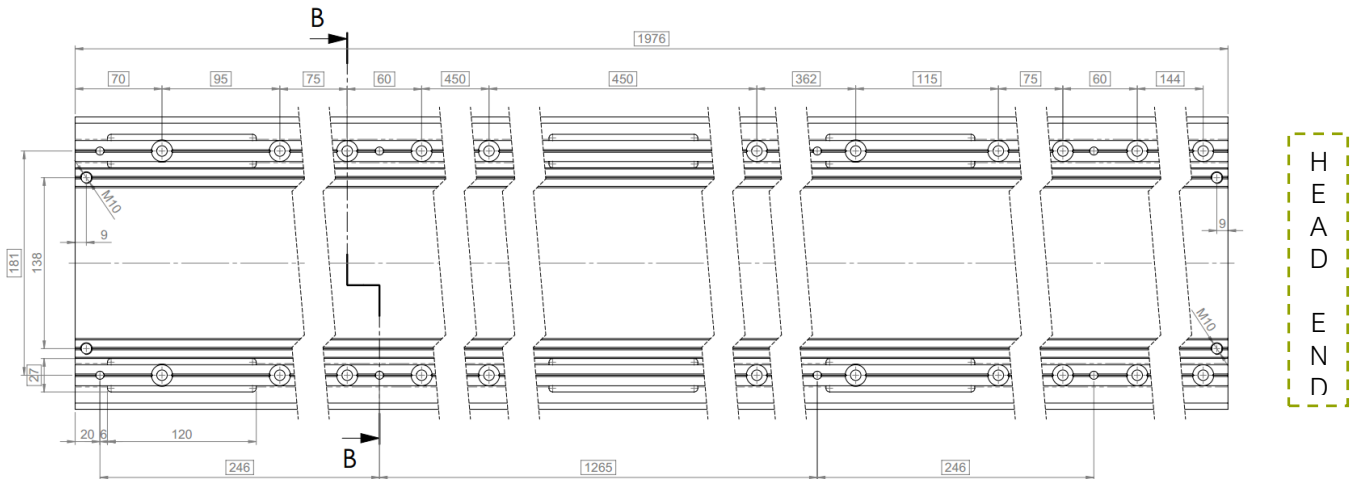The drilling situation is as follows:



FLOOR FLATNESS TOLERANCE


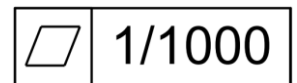
Once the floor profile is installed, the rail must be placed as shown below:

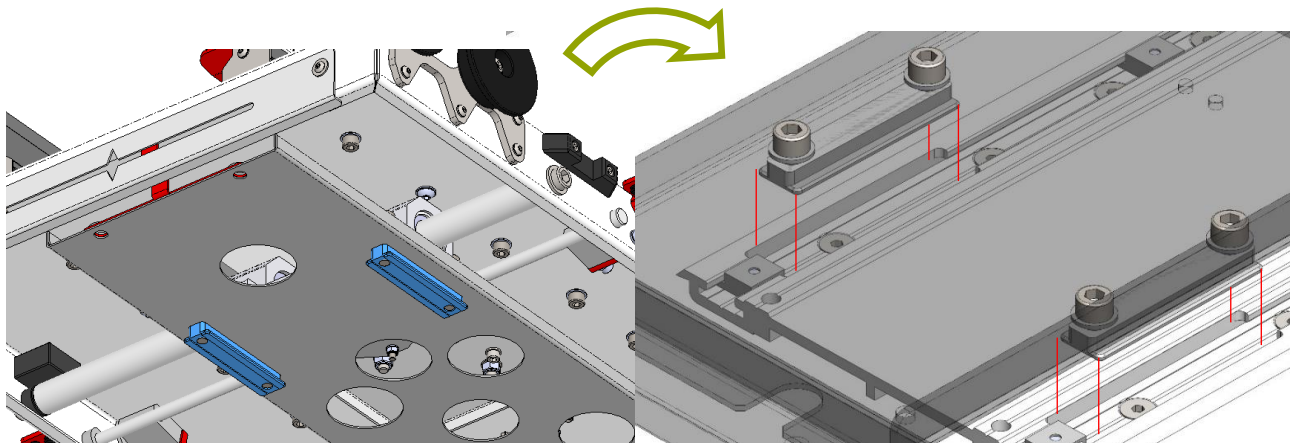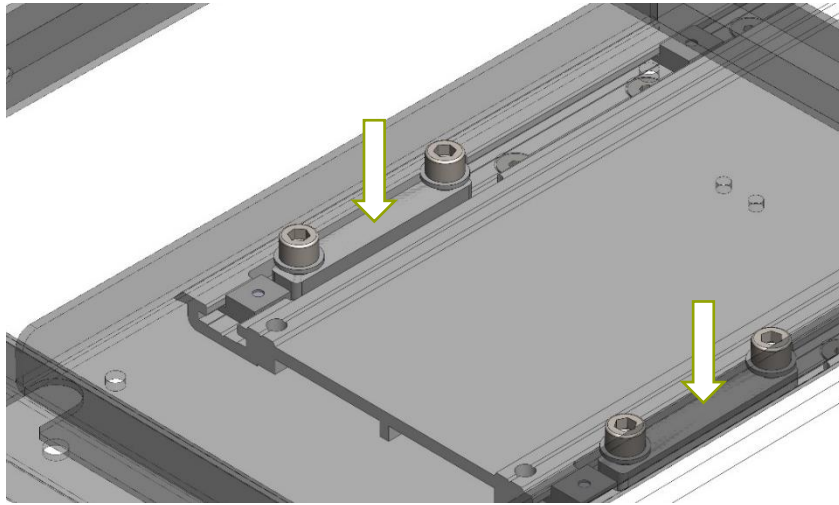1. Place the “T” fixation parts (in red) in the corresponding 6 slots of the aluminium profile.

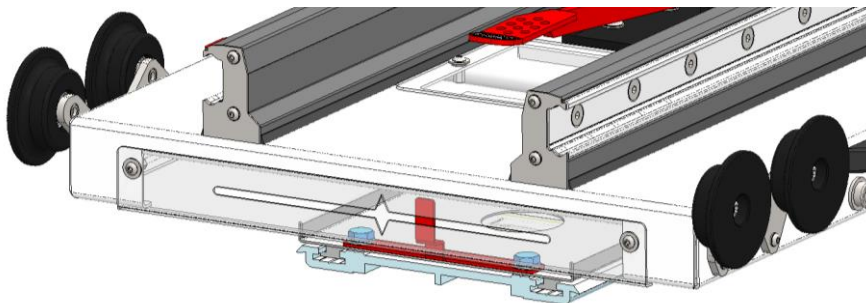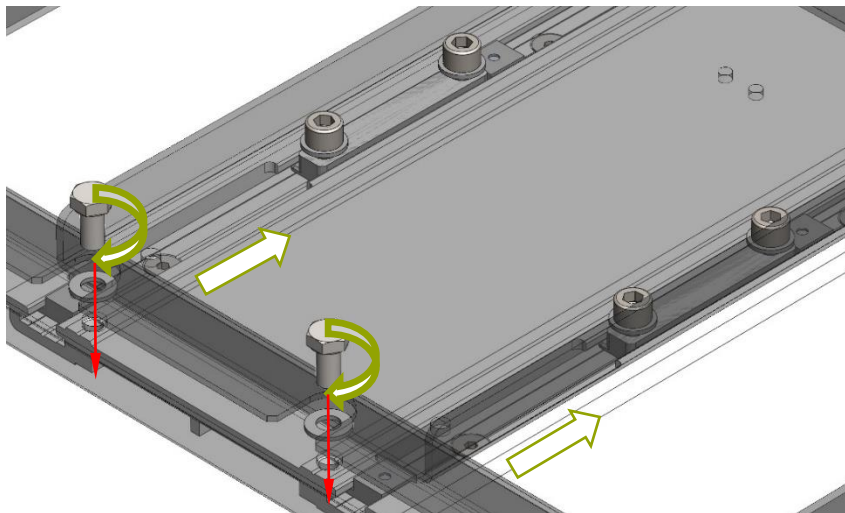2. Push the rail until the mechanical stop and then tighten the 4 hexagonal DIN 933 M10x20 screws at the front (x2) and rear (x2) zone.

### 3.2. Electrical connection

The electronics is prepared to admit a current of **12 V**.

The BRAVO rail must be connected to the electricity network of the vehicle using section 4 mm<sup>2</sup> electric cable as a minimum (paying special attention to the poles + o -). The aim of this connection is that the Brava stretcher charges the stretcher batteries when it is inside the ambulance with the rail closed and the third position lever in horizontal position.

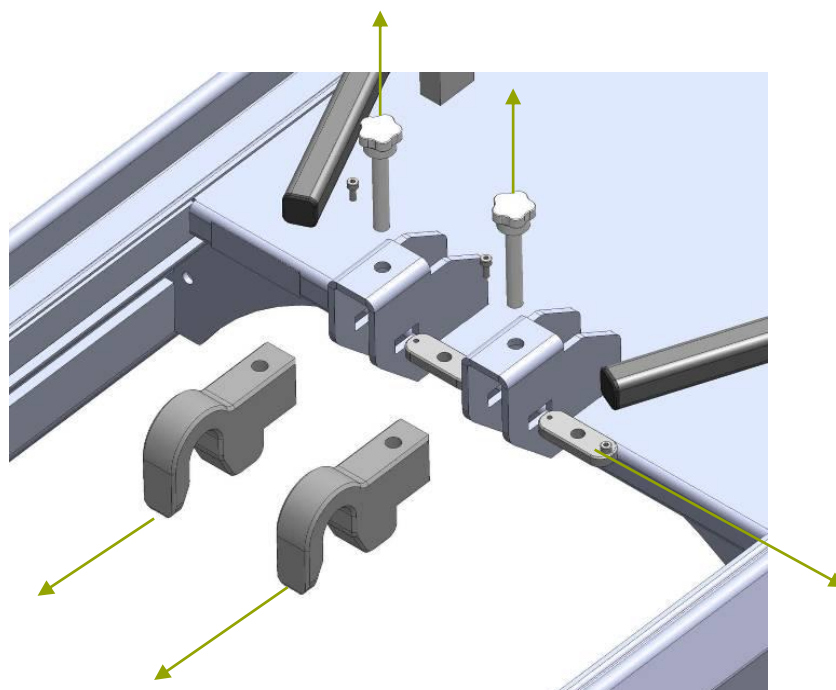
It's highly recommended to protect externally the wires with a plastic cover to prevent electrical problems (scratches, short-circuits...)

The electrical system must be secured with a 20A fuse.

The Bravo rail is provided with a fast connector, the female is connected to the Bravo rail and the male must be connected to the ambulance battery.


In case the electric stretcher breaks down, remove the rail hooks according to the figure below, take off the stretcher and replace the hooks in their original position.


### 4. INTERMEDIATE POSITION (OPTIONAL)

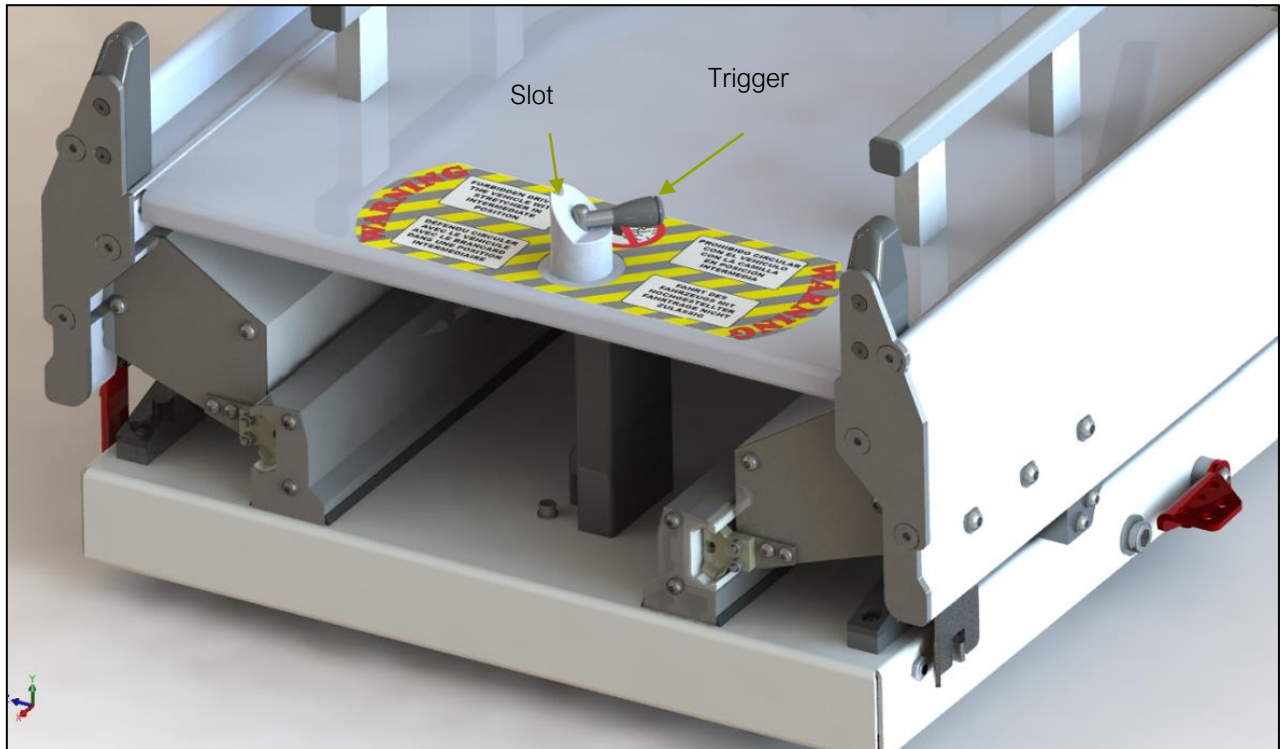If you want to rise the stretcher inside the ambulance, you must remove the trigger of the slot as you can see below and move the mobile part of the rail until the vertical trigger stays inside the black piece of plastic. When you want to return at locked position of the stretcher with the rail or if you want to take out the stretcher, you must lift the trigger into the slot.

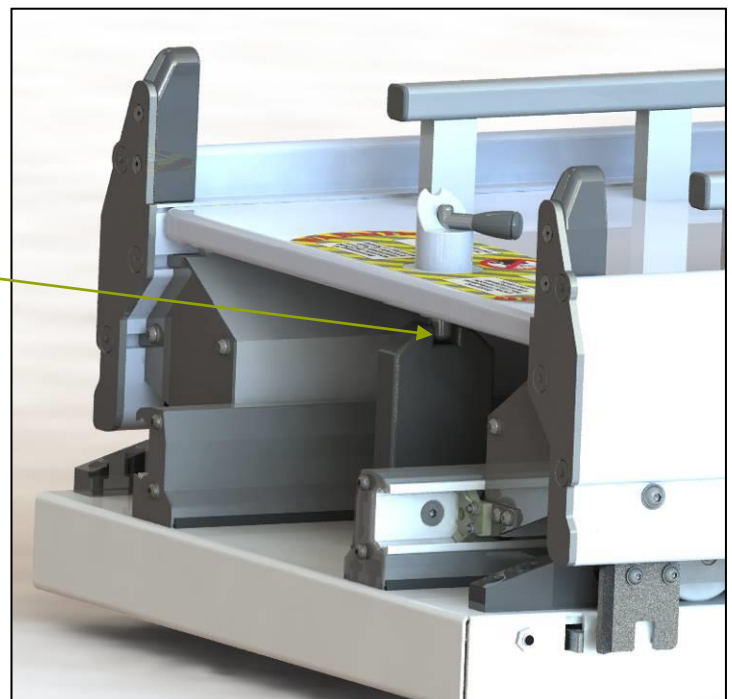Vertical trigger  
inside the black  
piece

# KARTSANA

Life-moving innovation

## OPERATION AND USAGE OF BRAVO 2C RAIL WITH DRAWER R-1002C

IMPORTANT:

## 5. DISINFECTION

---

When disinfecting the stretcher, use products that will not damage the surface of the materials and wipe with a cloth or something similar. If the surface to be disinfected is greased, re-greased it after disinfecting it.

All disinfectants must be used in accordance with the manufacturer's instructions.

The manufacturer will not be held responsible for any anomaly or damage caused by using a cleaning product that could damage the surface of the rail materials.

To ensure the hygiene and proper conservation of the product components, the manufacturer recommends that they be disinfected after each use.

## 6. MAINTENANCE

---

Every six months check and lubricate the interior of the side guides as well as the locking system. We recommend that maintenance be performed by a specialist from our company every six months. During this inspection, a general check will be made to replace any parts that could be damaged or worn due to use.

The rail should be regularly greased (more often when lower temperatures) with oil or lubricant to protect it against corrosion and external agents (such as salt and sanitizer) and to provide a proper guide way system movement. For example, you can use WD-40 without silicone.

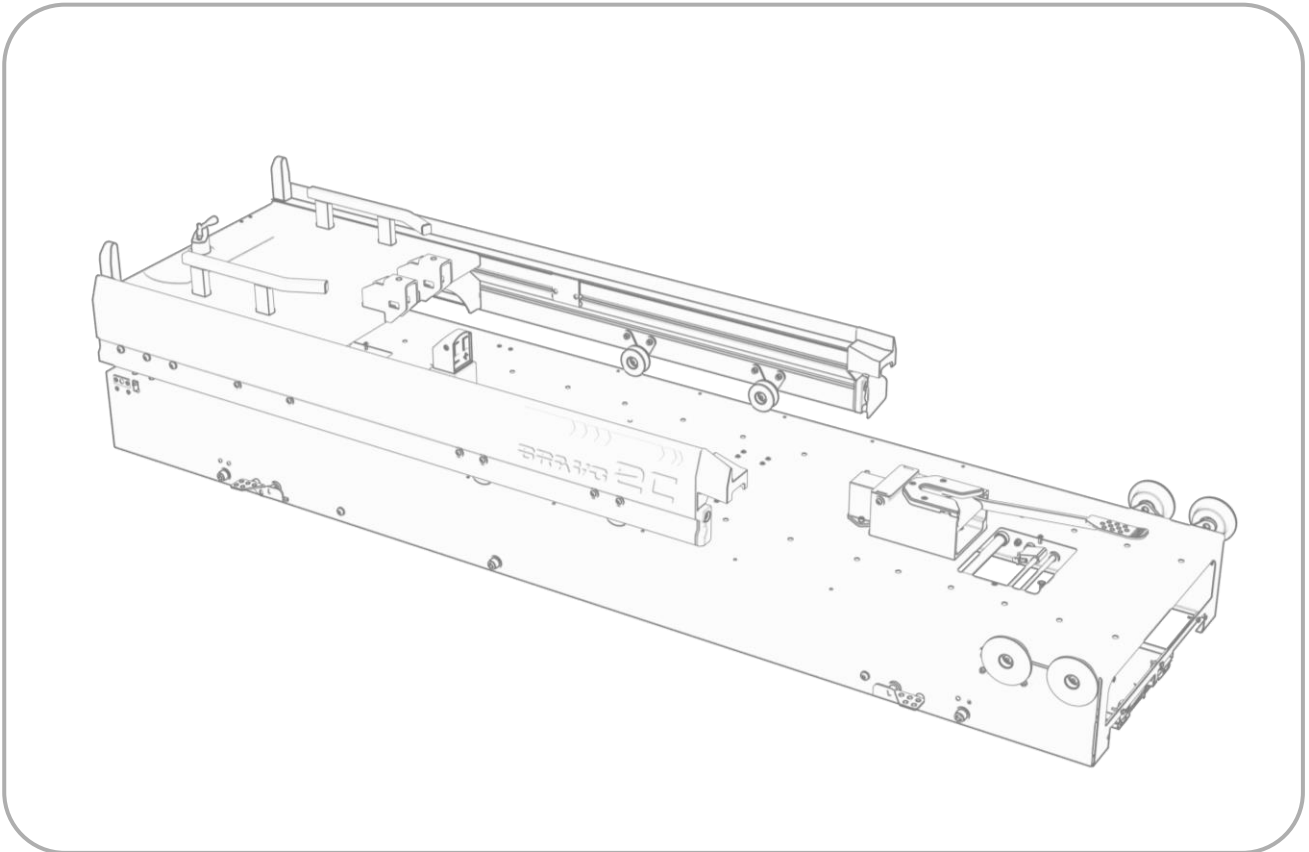
**KARTSANA**

Life-moving innovation

OPERATION AND USAGE OF BRAVO  
2C RAIL WITH DRAWER R-1002C

## 7. SPARE PARTS

---

For spare parts for this product, please visit the following link:

[https://www.kartsana.com/global/Spare\\_parts\\_R-1001\\_R1002.pdf](https://www.kartsana.com/global/Spare_parts_R-1001_R1002.pdf)

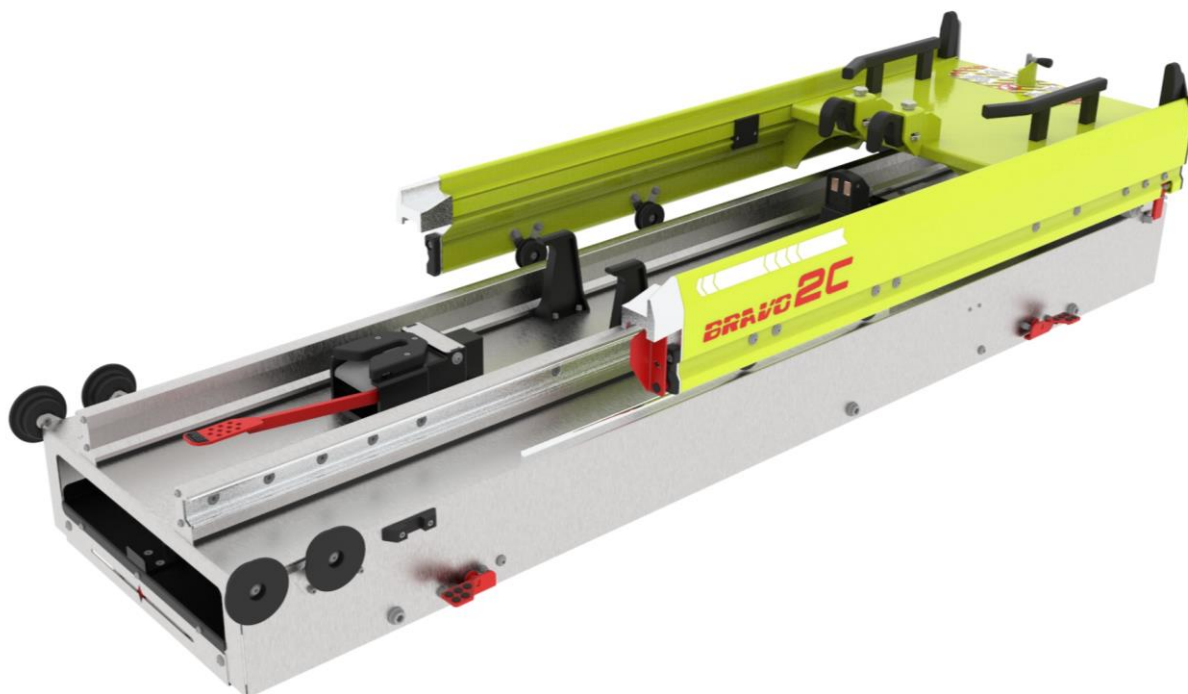
If you have any questions or queries, please contact our technical service.

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