Serie

R-419

R-419 / R-419L

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
+34 93 715 86 72
info@kartsana.com
www.kartsana.com

Manual de instrucciones

Rail mecánico R-419 / R-419 L

Instruction manual

Mechanical rail R-419 / R-419 L

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Ancrage mécanique R-419 / R-419 L

ESPAÑOL

ENGLISH

FRANÇAIS

KARTSANA

Life-moving innovation

C/ Narcís Monturiol, 34
08192 Sant Quirze del Vallès
Barcelona (España)
TEL. +34 93 715-86-72
info@kartsana.com
www.kartsana.com

Serie Railes R419

Modelos: R-419 / R-419L

MANUAL DE INSTRUCCIONES

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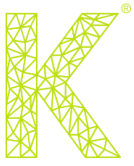
R-419 / R-419-L Models

INSTRUCTIONS MANUAL

Read these maintenance instructions
before using the product and keep them for
future reference.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

CONTENTS

1.- INTRODUCTION	2
1.1.- FOREWORD	2
1.2.- RESPONSIBILITY AND WARRANTY	2
1.3.- SPECIFICATIONS	2
1.4.- ATTENTION	3
1.5.- SUMMARY OF SAFETY PRECAUTIONS	3
2.- INSTRUCTIONS MANUAL	5
2.1.- DIAGRAM SHOWING RAIL FASTENING TO THE FLOOR	5
2.2.- TECHNICAL CHARACTERISTICS R-419	5
2.3.- RAIL OPERATION AND USAGE.	7
2.3.1.- FASTENING SYSTEMS:	7
2.3.2.- STRETCHER LOOK ON THE RAIL.	8
2.3.3.- UNLOCKING THE STRETCHER ON THE RAIL.	11
2.3.4.- FASTENING THE SPINAL BOARD OR SCOOP STRETCHER UNDER THE RAIL.	12
2.3.5.- WIDTH ADJUSTMENT OF FRONTAL WHEELS	13
3.- DESINFECTION	15
4.- MAINTENANCE	15
5.- EXPLODED VIEW	ERROR! NO S'HA DEFINIT EL MARCADOR.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

1.- INTRODUCTION

1.1.- Foreword

Rail R-419 has been tested according to standards UNE-EN 1865-5 and UNE- EN 1789.

All information regarding treatment, disinfection and maintenance is indicated, taking into consideration our current experience and know-how.

We reserve the right to change the specifications of the rail to improve the product.

1.2.- Responsibility and warranty

The rail must be checked at the time of its delivery to the rescue organisation. All its functions must be explained in detail. The assistance organisation must take charge of teaching all its employees how to use it correctly.

The product has a 24-months warranty from the delivery date to the end user.

The warranty does not cover defects resulting from incorrect installation, misuse or improper use of the stretcher. All repairs must be made by a technical service authorised by Kartsana or its respective representative.

The manufacturer will not be liable for any anomaly caused to the stretcher because of using products other than those provided by Kartsana.

Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.

1.3.- Specifications

	R-419	R-419-L
Maximum unassisted load	220Kg	300Kg
Standards	UNE-EN-1865-5 EN-1789+A1	UNE-EN-1865-5 EN-1789+A1
Total Length max/min	2290mm/1957mm	2473mm/2078mm
Total Width	622mm	622mm
Loading height	Depends on the used stretcher	Depends on the used stretcher
Recommended operatives required to load / unload an occupied stretcher	1 operative, or 2 if the stretcher were occupied by a larger sized patient.	1 operative, or 2 if the stretcher were occupied by a larger sized patient.
Compatible stretchers	TG-241 (serie) / TG-242 / TG-243 (P) / TG-244 (P) / TG-247 / TG-249	TG-880 (series)

KARTSANA

Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

1.4.- Attention

Through this sign, information is furnished on important safety measures for the correct use of the stretcher, to prevent accidents. The warnings alert the reader about a situation which, if not avoided, could result in death or serious injury. The cautions alert the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur because of use or misuse.

1.5.- Summary of safety precautions

Read carefully and strictly follow the warnings and cautions listed on these pages. Service only by qualified personnel.

WARNINGS:

- Improper usage of the stretcher can cause injury to the patient or operator. Operate the stretcher only as described in this manual.
- Do not modify the rail or any components of the rail. Modifying the product can cause unpredictable operation resulting in injury to the patient or operator. Modifying the product also voids its warranty.
- This rail must operate with a compatible stretcher.
- Have the vehicle safety rail compatible installed by a certified mechanic, following instructions in this manual. Improper rail installation can cause injury to the patient or operator and/or damage to the stretcher
- Practice changing height positions and loading the stretcher until operation of the product is fully understood. Improper use can cause injury.
- Do not allow untrained assistants to assist in the operation of the stretcher. Untrained technicians/assistants can cause injury to the patient or themselves.
- Do not ride on the base of the rail. Damage to the product could occur, resulting in injury to the patient or operator.
- Grasping the stretcher improperly can cause injury. Keep hands, fingers and feet away from moving parts. To avoid injury, use extreme caution when placing your hands and feet near the base tubes while raising and lowering the stretcher.
- Always use all restraint straps to secure the patient on the stretcher. An unrestrained patient may fall from the stretcher and be injured.
- The R-419 rail is designed to be compatible with the TG-241 series and TG-242/243 Kartsana stretchers and is operator responsibility that these products work together.
- Two operators must be present when the stretcher is occupied.
- Operators must be able to lift the total weight of the patient, stretcher and any items on the stretcher.
- When cleaning, use any appropriate personal safety equipment (goggles, respirator, etc.) to avoid the risk of inhaling contagion.
- Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. If the products described above are used to clean Kartsana equipment, measures must be taken to ensure the stretchers are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive residue on the surface of the stretchers, possibly causing premature corrosion of critical components.
- Failure to properly clean or dispose of contaminated mattress or other stretcher components will increase the risk of bloodborne pathogens and may cause injury to the patient or operator.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

CAUTIONS:

- Changes or modifications to the unit not expressly approved by Kartsana could void the user's authority to operate the system.
- Installation of the safety hook should be done by a certified mechanic familiar with ambulance vehicle construction. Consult the vehicle manufacturer before installing the safety hook and be sure that the installation of the safety hook does not damage or interfere with the brake lines, oxygen lines, fuel lines, fuel tank or electrical wiring of the vehicle.
- Before operating the stretcher, clear any obstacles that may interfere and cause injury to the operator or patient.
- When unloading the stretcher from the patient compartment, ensure that the caster wheels are safely set on the ground or damage to the product may occur.
- Ensure that the restraints are not entangled in the base frame when raising and lowering the stretcher.
- Do not steam clean or ultrasonically clean the unit.
- Maximum water temperature should not exceed 80°C.
- Failure to comply with these instructions may invalidate any/all warranties.
- For additional maintenance information, see the preventative maintenance information.
- Improper maintenance can cause injury or damage to the product. Maintain the stretcher as described in this manual. Use only Kartsana approved parts and maintenance procedures. Using unapproved parts and procedures could cause unpredictable operation and/or injury and will void the product warranty
- Failure to use authorized parts, lubricants, etc. could cause damage to the stretcher and will void the warranty of the product.

NOTES:

- Loose items or debris on the patient compartment floor can interfere with the operation of the safety hook and stretcher fastener. Keep the patient compartment floor clear
- This manual should be considered a permanent part of the rail and should remain with the product even if the stretcher is subsequently sold.
- Kartsana continually seeks advancements in product design and quality. Therefore, while this manual contains the most current product information available at the time of printing, there may be minor discrepancies between your stretcher and this manual
- Kartsana recommends that, prior to installation, the certified mechanic plan the placement of the safety hook in the rear of the vehicle.

KARTSANA

Life-moving innovation

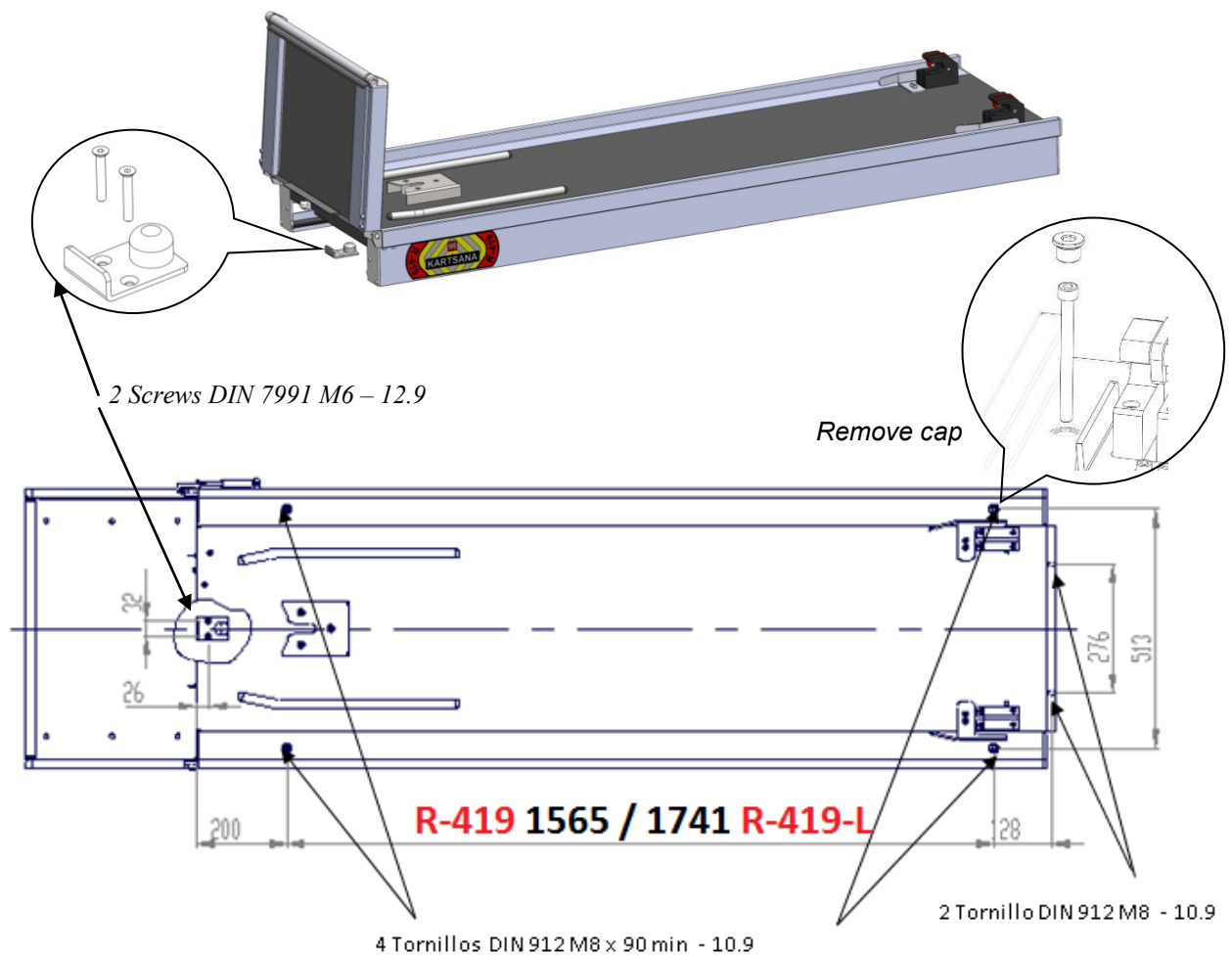
OPERATION AND HANDLING RAÍL R-419 / R-419-L

2.- INSTRUCTIONS MANUAL

The anchor rail R-419 has been designed for properly fastening of homologated Kartsana series TG-241 y TG-242-249 and the R-419-L has been designed for fastening series TG-880.

2.1.- Diagram showing rail fastening to the floor

The R-419 rail should be fixed to the floor of the ambulance using the holes indicated in the above diagram.

8 FLOOR FASTENING HOLES FOR M8 AND M6 DIN 912, AND DIN 7991.

2.2.- Technical characteristics R-419

Approximate rail weight: 46 Kg. (419) 50Kg (419-L)

Rev.7
06.24

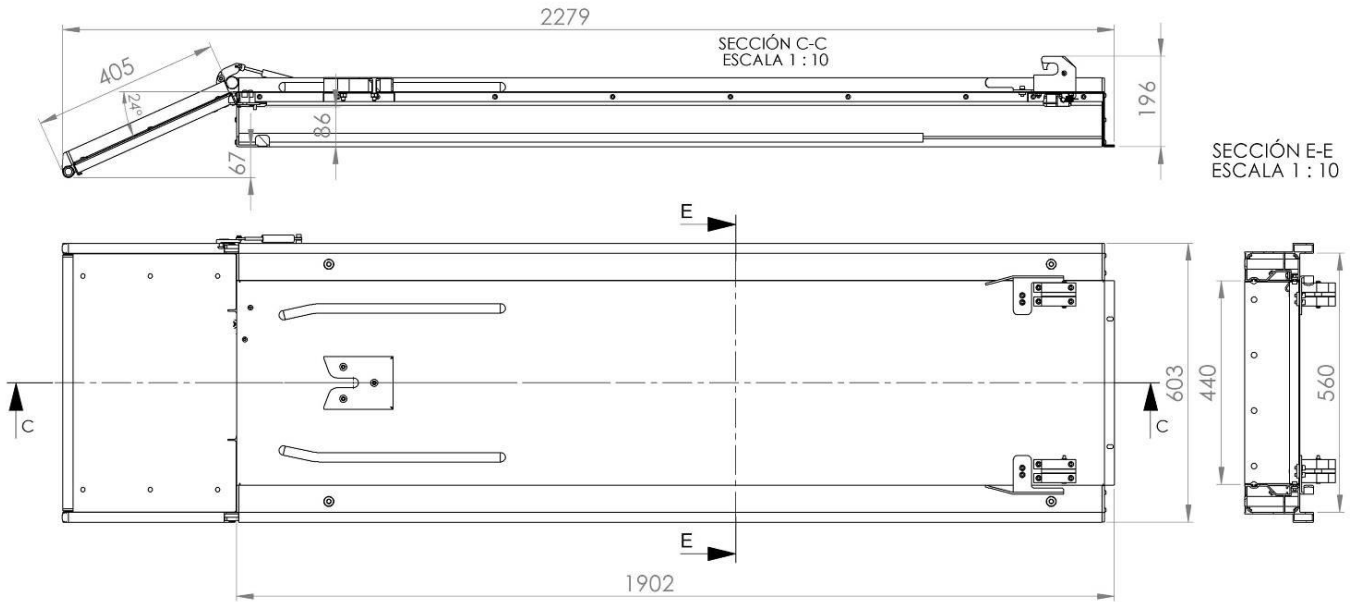
KARTSANA

Life-moving innovation


OPERATION AND HANDLING RAÍL R-419 / R-419-L

Approximate rail measurements (en mm.) based on the drawing below.

R-419

R-419-L

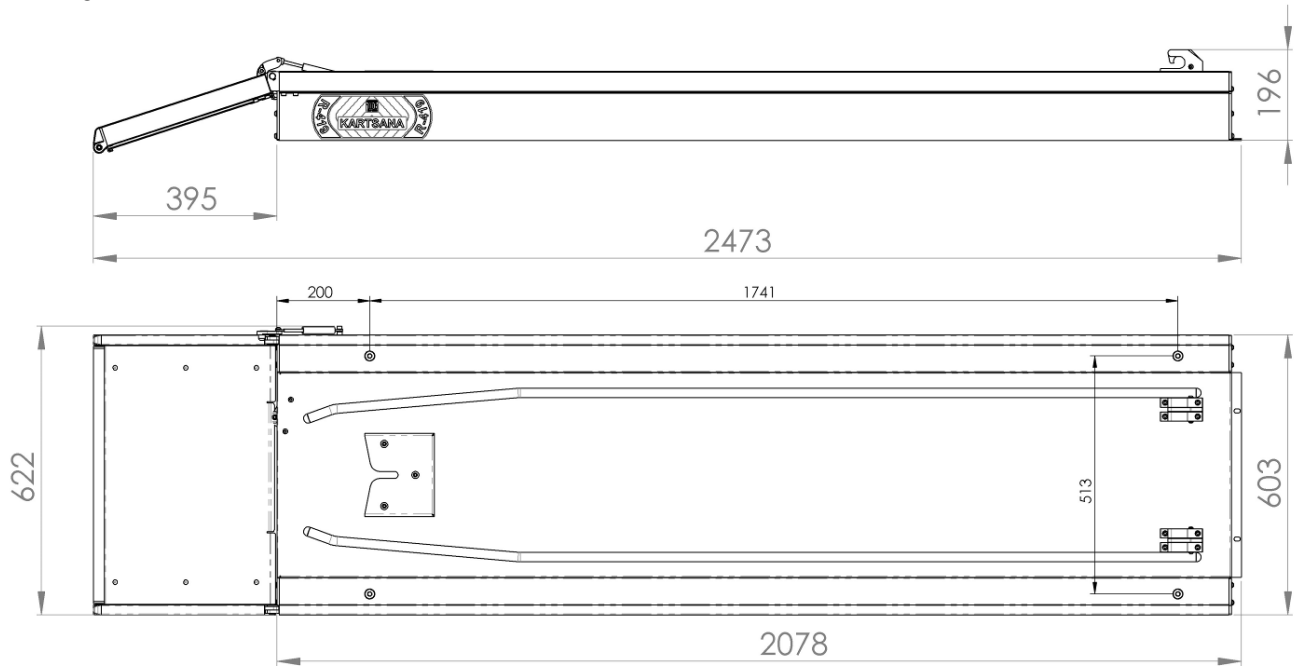KARTSANA

Life-moving innovation

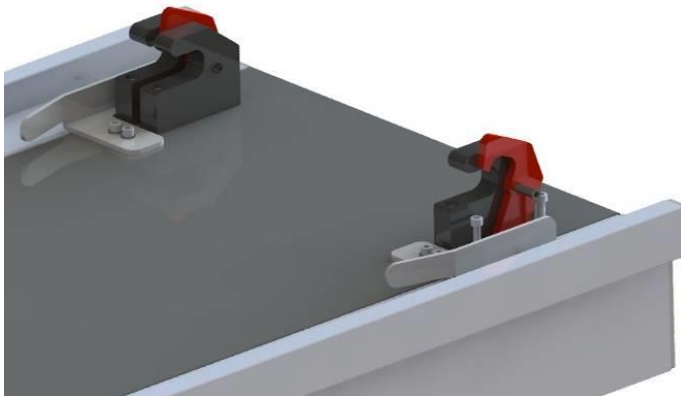
OPERATION AND HANDLING
RAÍL R-419 / R-419-L

2.3.- Rail operation and usage.


- To prevent injuries to limbs and other body parts, make sure they are not within the range of the rail moveable parts.

2.3.1.- Fastening systems:

- The Kartsana R-419 rail is delivered fitted with the following rear attachment system:

Red hooks opened

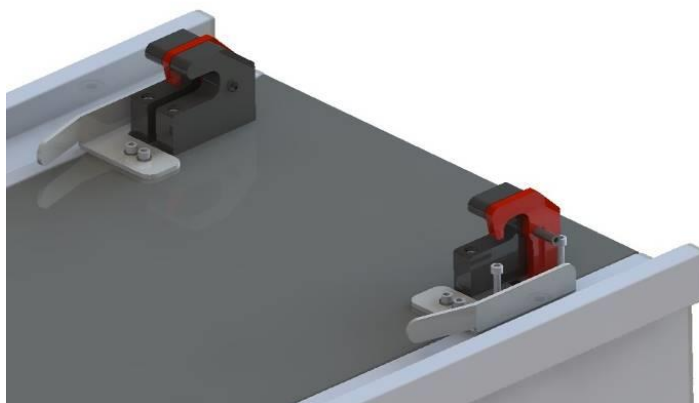Red hooks closed

KARTSANA

Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

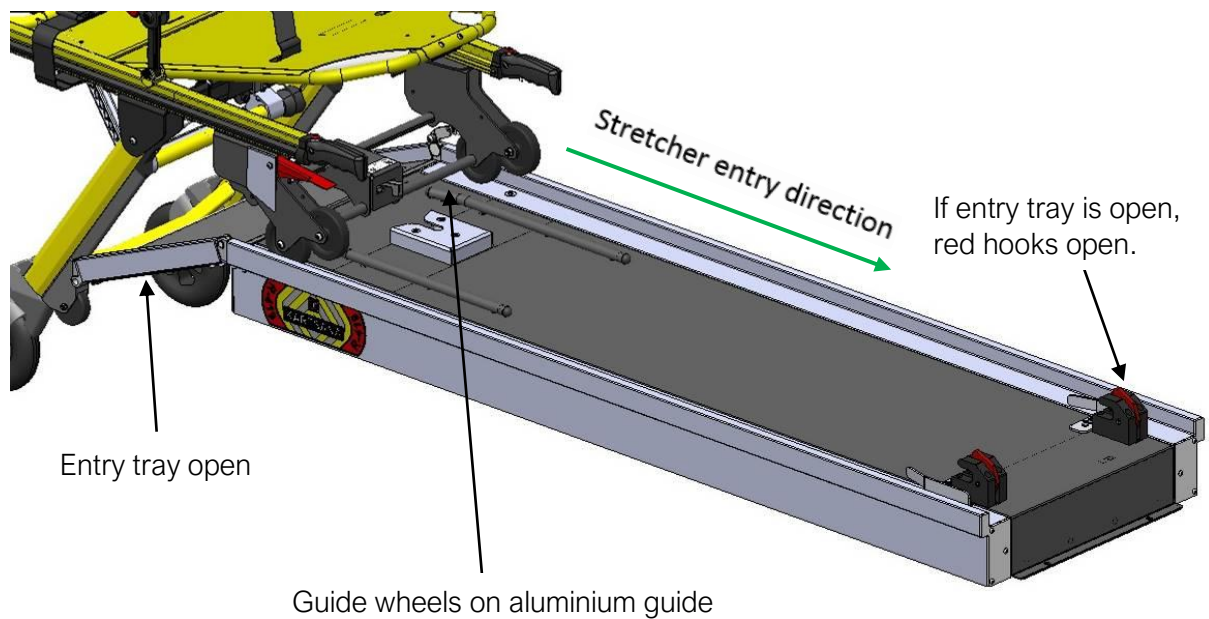
2.3.2.- Stretcher look on the rail.

- To fit the stretcher over the rail, follow the steps below:

Entry tray open

- Place the rail tray in the horizontal position (open). This arrangement means that the red fastening hooks will be open.
- Position the guide wheels over the rail so that they slide over the aluminium profiles, the wheels are continuously guided to prevent side movements.



KARTSANA

Life-moving innovation

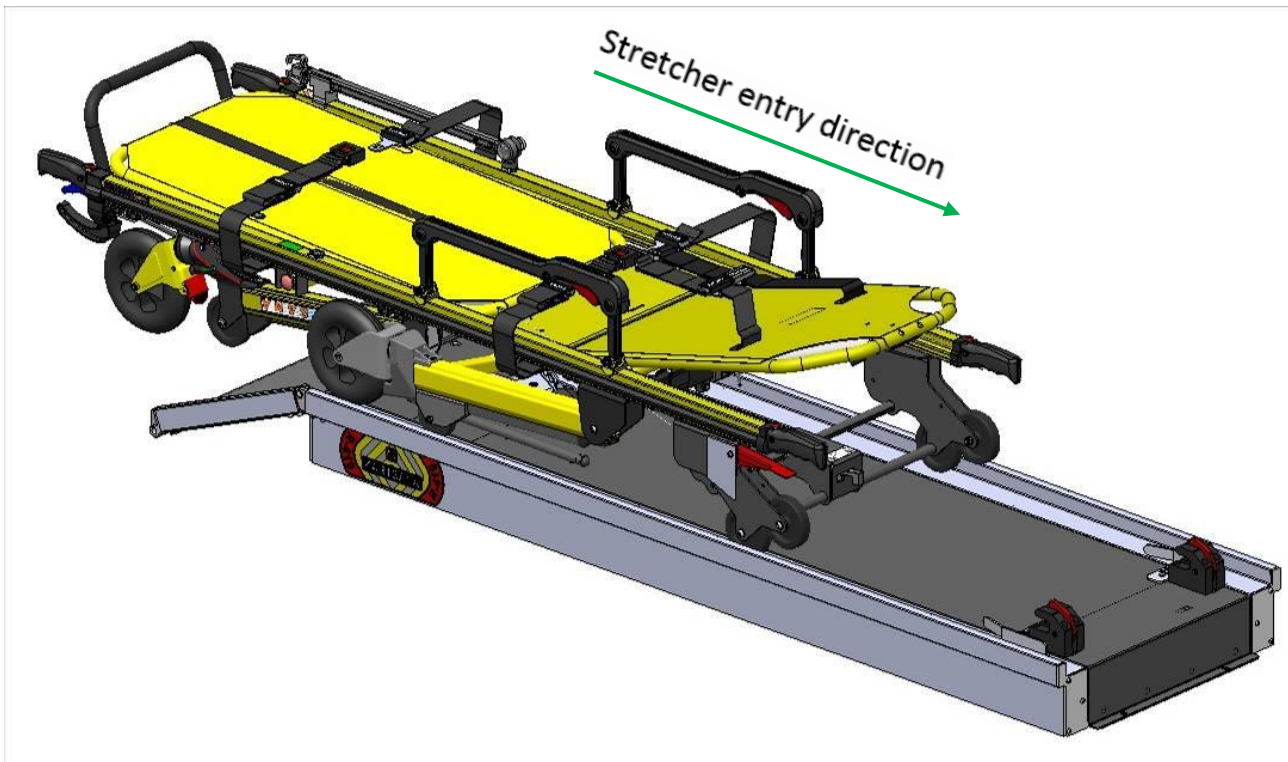OPERATION AND HANDLING RAÍL R-419 / R-419-L

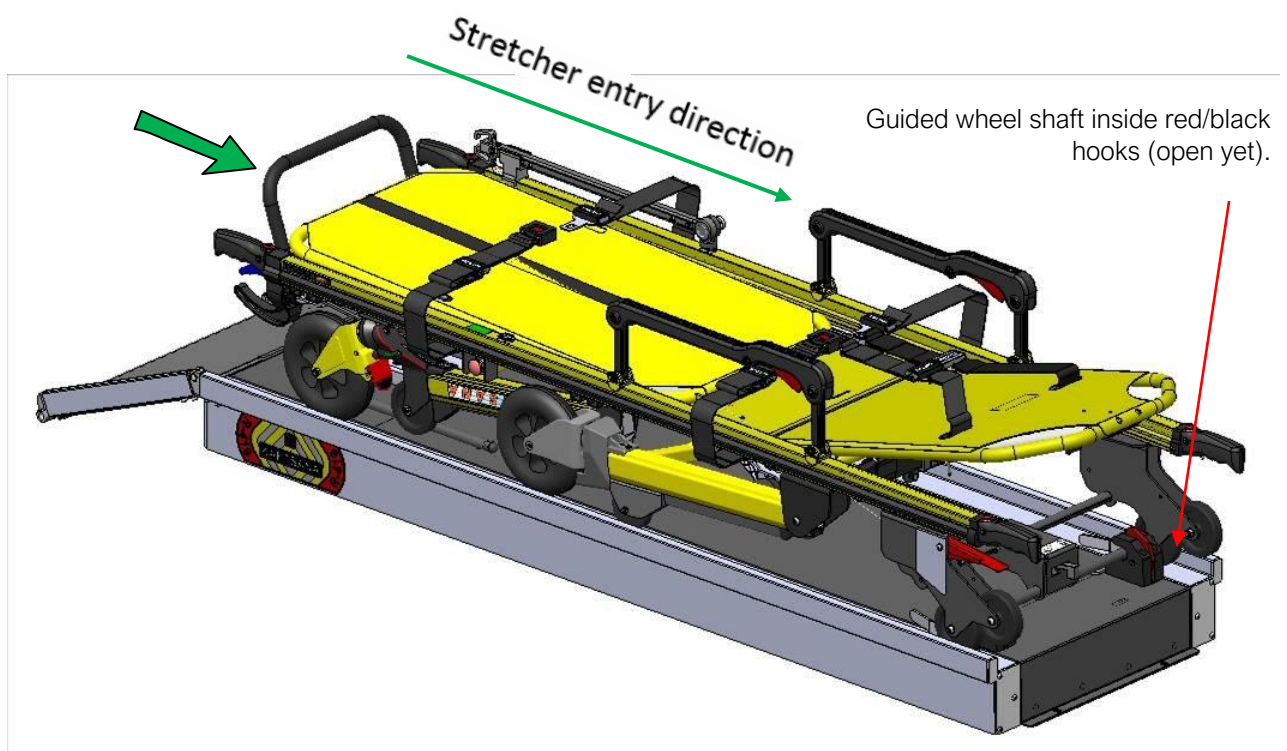- Unlock the front stretcher legs and insert the stretcher on the rail until the rear leg also stops on the edge of the rail base.
- Unlock the rear leg, insert the stretcher until the guide wheel shaft stops in the two front hooks (the red part of the hooks moves, and the black part is fixed)


KARTSANA

Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

- When the wheels arrive at the end of their travel, the rear pin will enter in the stainless steel structure, this prevents vertical movement of the stretcher.

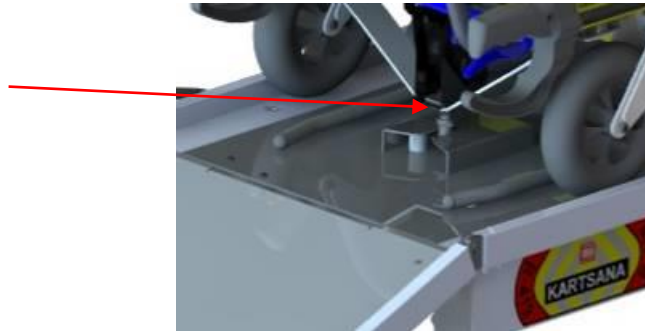
Rear pin inside stainless steel structure

- Finally, to close the red hooks and fasten the rail at the front, lift the movable tray.

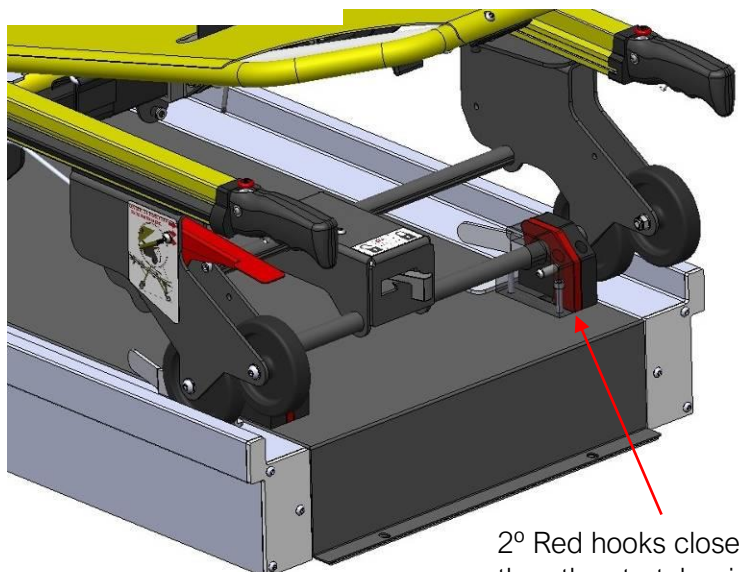1° Raise the tray.


2° Red hooks close, then the stretcher is secured.

KARTSANA

Life-moving innovation

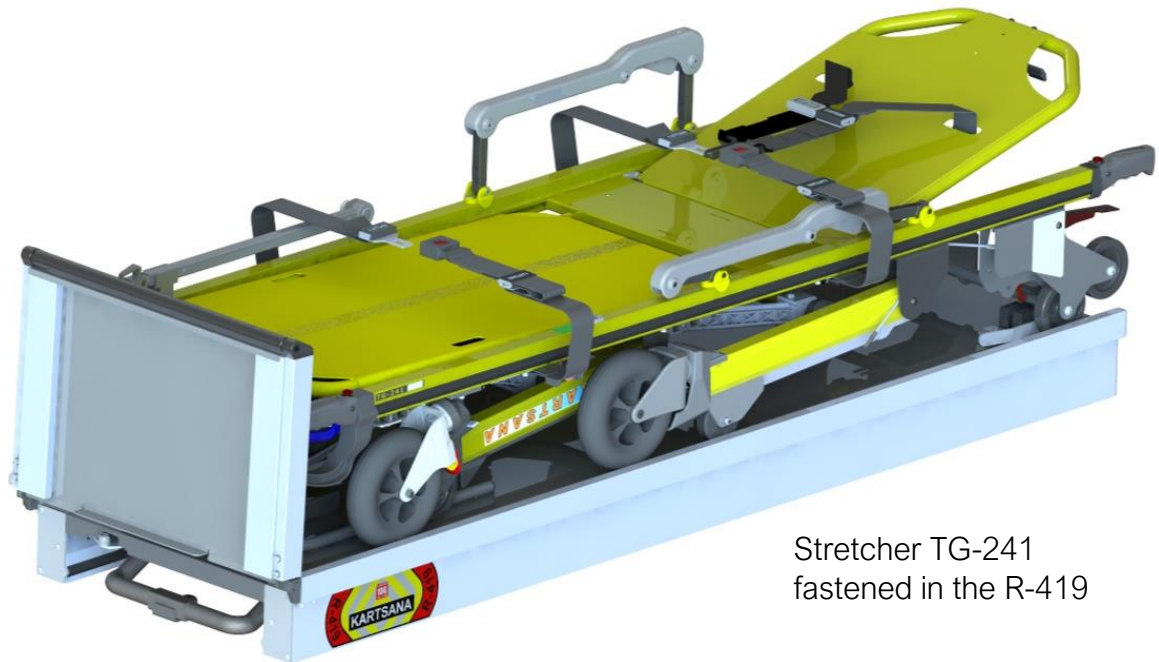OPERATION AND HANDLING RAÍL R-419 / R-419-L


Stretcher TG-241
fastened in the R-419

When unlocking the stretcher's legs, hold the stretcher firmly when inserting it in the rail to prevent it from falling.

2.3.3.- Unlocking the stretcher on the rail.

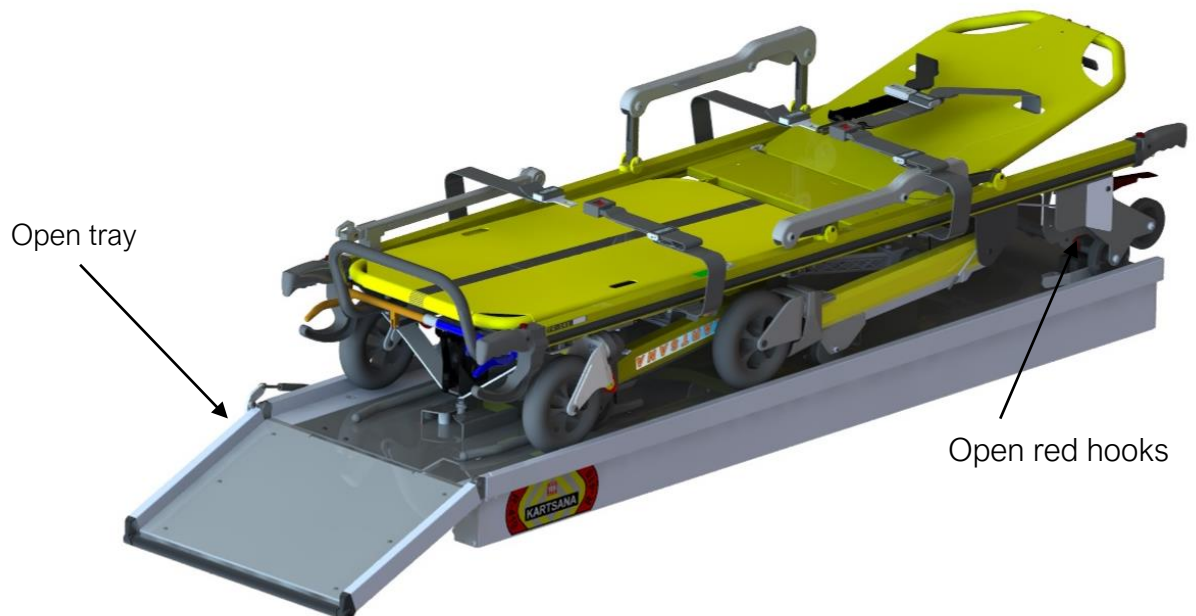When the rail needs to be unlocked to remove the stretcher, just open the entry tray and place it in its horizontal position.

Open tray

Open red hooks

KARTSANA

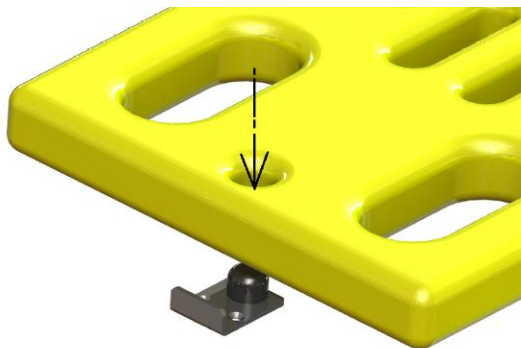
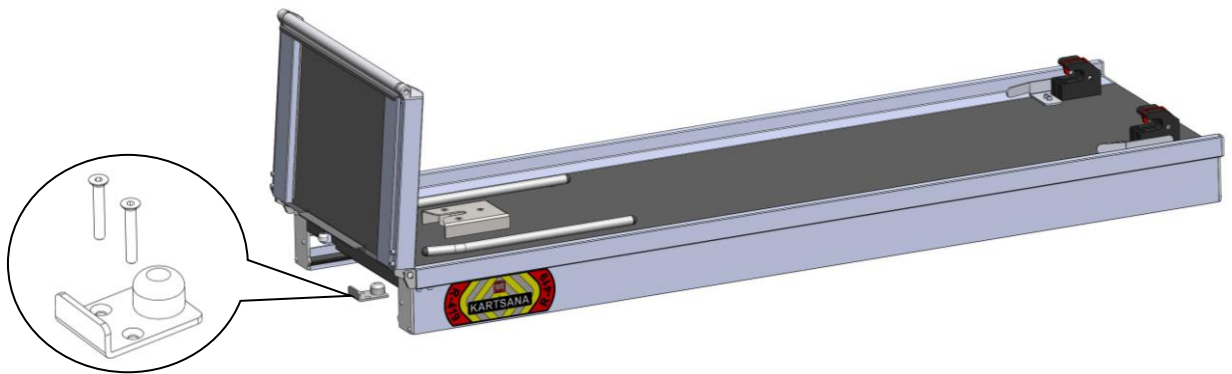
Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

- Once the tray is open, the stretcher must be pulled out from the rail to remove it.
- Ensure the rear and front legs are unlocked respectively, before the edge of the rail base reaches the height of each one of these, to prevent the stretcher dropping to the floor with the legs folded.

2.3.4.- Fastening the spinal board or scoop stretcher under the rail.

To positioning and fixing the spinal board/scoop inside R-419 rail, we will need 2 screws (DIN 7991 M6 quality 8.8 or higher), and the parts showed below:

Spinal board:

You must put the spinal board under/inside the rail and put inside the round part you can find in the front part fixed on the floor of the ambulance, where we've mounted previously (see information in point 2.1 of this manual)

Scoop stretcher:

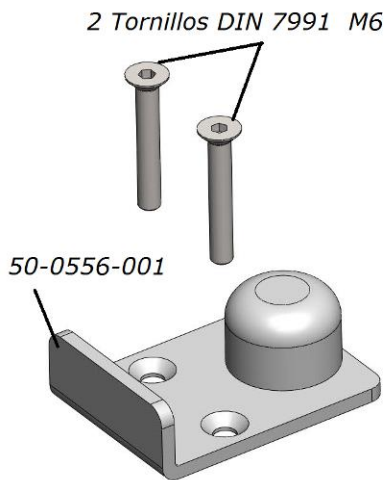
To place the stretcher blades, we must fit the edge the scoop stretcher itself in the part fixed on the floor of the ambulance, where we've mounted previously (see information in point 2.1 of this manual)

KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

Examples how positioning the scoop stretcher and the spinal board:

2.3.5.- Width adjustment of frontal wheels

Rail R-419 provides two lateral platens to adapt the interior width of the stretcher attack wheels to get the minimum lateral looseness between the stretcher and the rail R-419.

These plates are located next to the final fixation hooks, and each one is fixed with two DIN 7380 M6X16 screws.

KARTSANA

Life-moving innovation

OPERATION AND HANDLING RAÍL R-419 / R-419-L

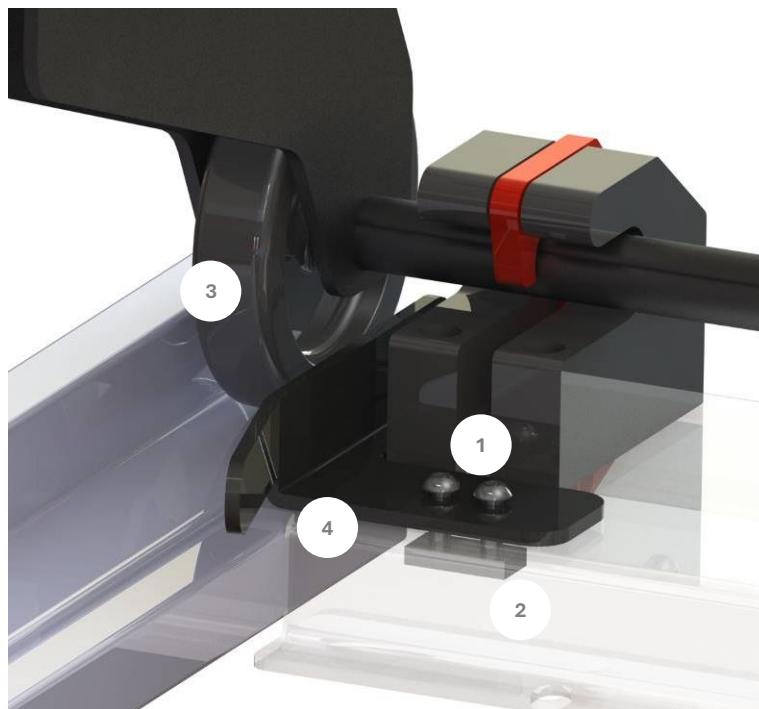
For the displacement of the platens, you must perform the following points:

Introduce the stretcher to fix into the rail R-419 to the end and ensure it is placed perfectly centred and straight.

Loosen the two screws of each platen with a 4mm Allen key until these can move. (not loosen too much to avoid the drop of the double rectangular nut that we can find inside)

Move the platens to the required position, that it will be a millimetre away from the attack wheels of the stretcher.

Tighten the screws of the platen until they remain immovable in the basis plate of rail R-419.

1- Screws DIN 7380 M6X16

2- Double rectangular nut M6

3- Stretcher attack wheel

4- Left lateral adaptation platen

KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

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To disinfect the rail, use products on the surface to disinfect that will not damage the rail's surface using a sponge cloth, rag or similar item. If the surface to be disinfected is greased, re-apply grease once the disinfection has been completed.

All disinfectants must be used in accordance with the manufacturer's instructions.

The manufacturer will not be held responsible for any anomaly or damage caused by using a cleaning product that could damage the surface of the rail materials.

To ensure the hygiene and proper conservation of the product components, the manufacturer recommends that they be disinfected after each use.

4.- MAINTENANCE

Check and grease moving parts and friction areas every two months.

The locking parts must also be checked. We recommend that maintenance be performed by a specialist from our company every six months. During this inspection, a general check will be carried out to replace any parts that may be damaged or worn from use.

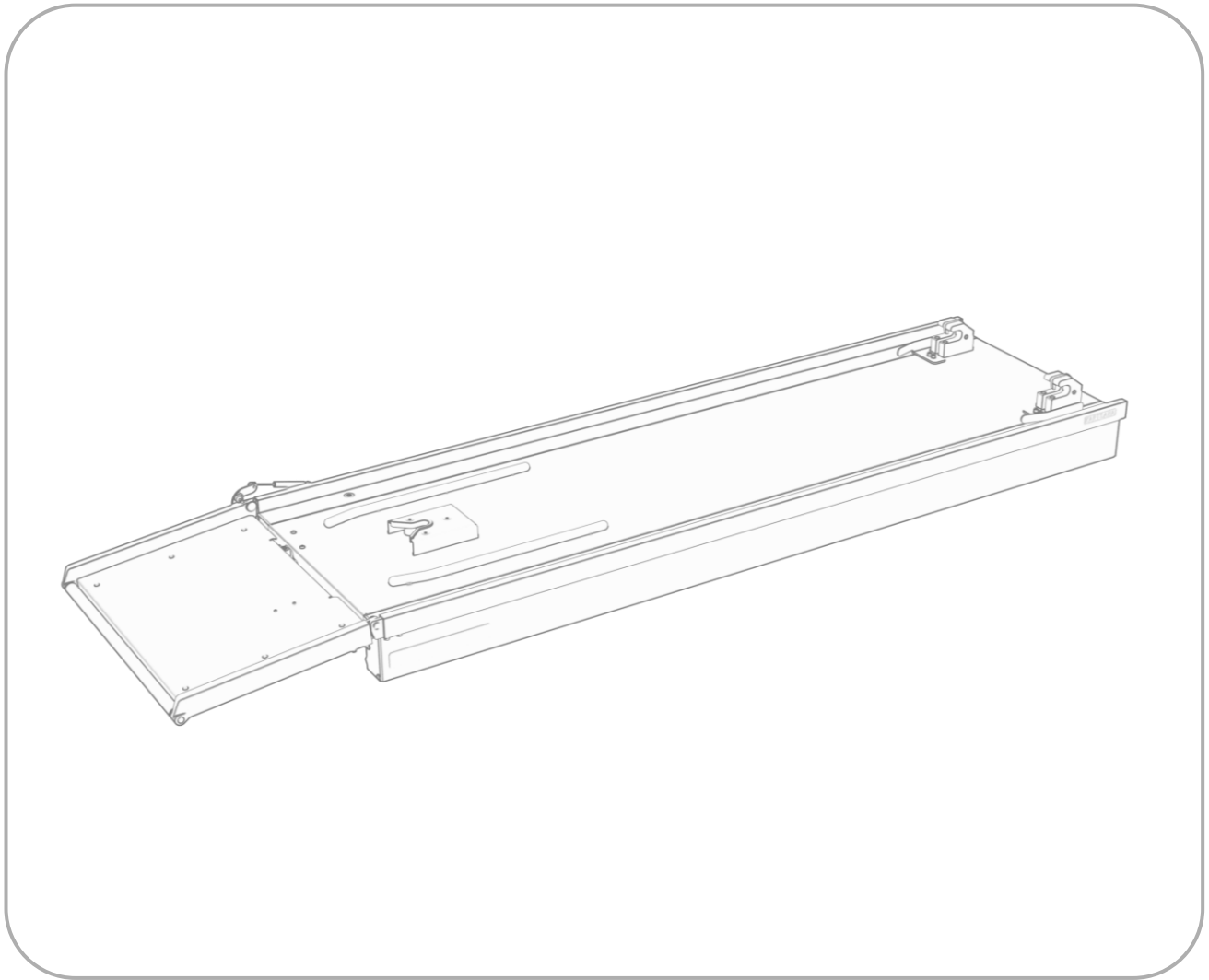
KARTSANA

Life-moving innovation

OPERATION AND HANDLING
RAÍL R-419 / R-419-L

5.- SPARE PARTS

For spare parts for this product, please visit the following link:

https://www.kartsana.com/global/Spare_parts_R419.pdf

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